


Ethos Backrest Mount and Hardware

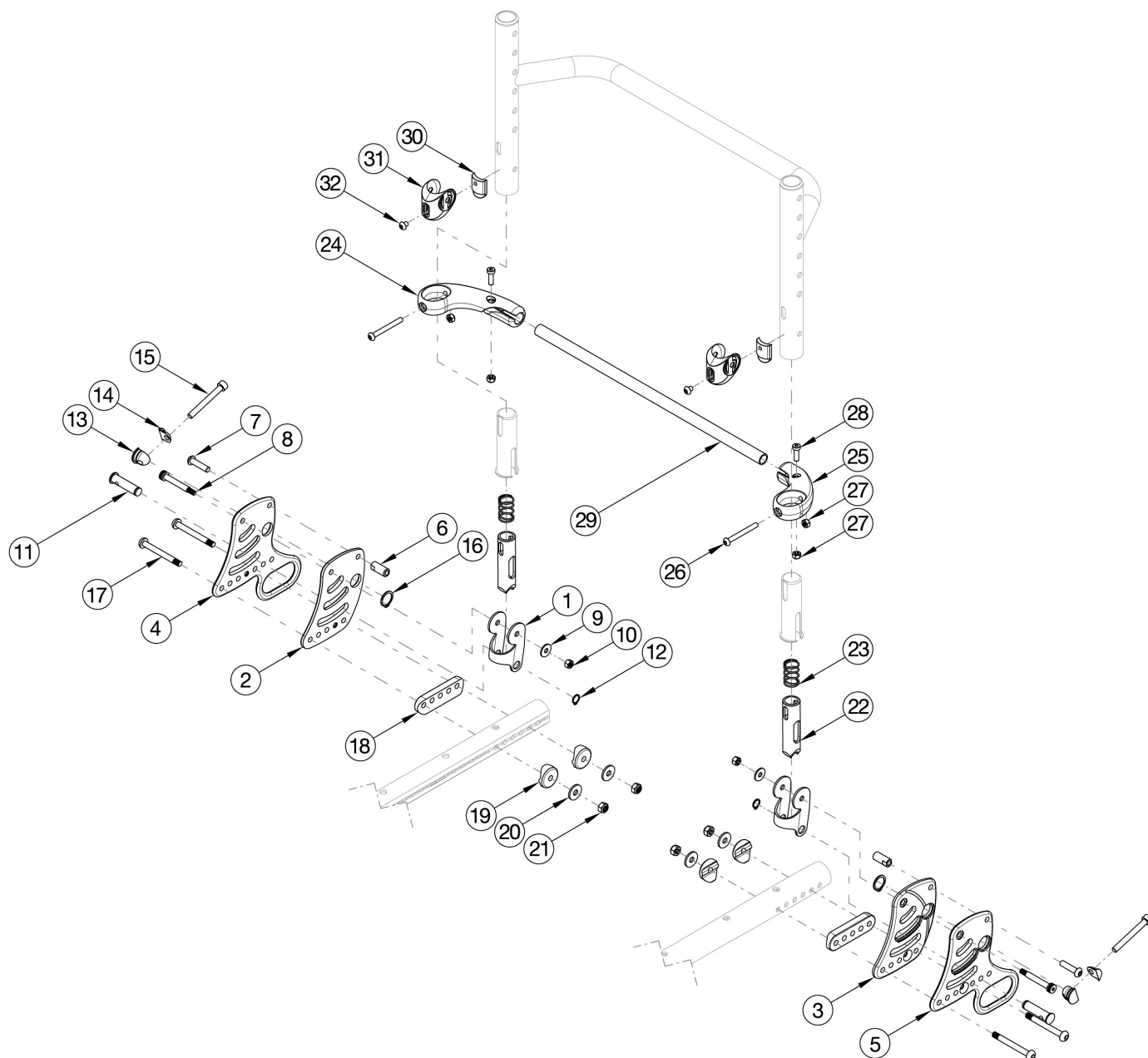

Utilize this page if chair has Rigid Locking Fold Down Back. For a complete retrofit, select a Back Plate and Hardware Kit PAIR and Backrest Actuation Bar kit based on width. If retrofitting, new back canes will be needed and if swing away arms are present new receivers are needed.





Item Number	Part Number	Description	Quantity
1, 2a, 3a, 6 - 23, 30 - 32	116890	Back Plate and Hardware, Non-Transit, Black, Ethos, PAIR	1
1, 2a, 6 - 21	116888	Back Plate and Hardware, Non-Transit, Black, Right, Ethos	1

Item Number	Part Number	Description	Quantity
1, 3a, 6 - 21	116889	Back Plate and Hardware, Non-Transit, Black, Left, Ethos	1
1, 2b, 3b, 6 - 23, 30 - 32	117348_ _ _	Back Plate and Hardware Non-Transit Color Anodized Ethos PAIR	1
1, 2b, 6 - 21	117296_ _ _	Back Plate and Hardware Non-Transit Color Anodized Ethos Right	1
1, 3b, 6 - 21	117297_ _ _	Back Plate and Hardware Non-Transit Color Anodized Ethos Left	1
1, 4 - 23, 30 - 32	117347	Back Plate and Hardware, Transit, Black, Ethos, PAIR	1
1, 4, 6 - 21	116892	Back Plate and Hardware, Transit, Black, Right, Ethos	1
1, 5 - 6 - 21	116893	Back Plate and Hardware, Transit, Black, Left, Ethos	1
30 - 32	116876	Relax Subassembly	1
24, 26 - 28	116877	Release Lever with Fasteners, Right	1
25 - 28	116878	Release Lever with Fasteners, Left	1
24 - 28, 29a	116879	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 12" Wide, Black	1
24 - 28, 29b	116880	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 13" Wide, Black	1
24 - 28, 29c	116881	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 14" Wide, Black	1
24 - 28, 29d	116882	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 15" Wide, Black	1
24 - 28, 29e	116883	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 16" Wide, Black	1
24 - 28, 29f	116884	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 17" Wide, Black	1
24 - 28, 29g	116885	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 18" Wide, Black	1
24 - 28, 29h	116886	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 19" Wide, Black	1
24 - 28, 29i	116887	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 20" Wide, Black	1
24 - 28, 29j	117832_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 12" Wide, Painted	1
24 - 28, 29k	117833_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 13" Wide, Painted	1
24 - 28, 29l	117913_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 14" Wide, Painted	1
24 - 28, 29m	117914_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 15" Wide, Painted	1
24 - 28, 29n	117915_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 16" Wide, Painted	1
24 - 28, 29o	117916_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 17" Wide, Painted	1
24 - 28, 29p	117917_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 18" Wide, Painted	1
24 - 28, 29q	117918_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 19" Wide, Painted	1

Item Number	Part Number	Description	Quantity
24 - 28, 29r	117919_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 20" Wide, Painted	1
2a	115550	Back Plate, Right, Black	1
2b	115550_ _ _	Color Anodized/Color Accent Back Plate, Right	1
3a	115551	Back Plate, Left, Black	1
3b	115551_ _ _	Color Anodized/Color Accent Back Plate, Left	1
4	115552	Back Plate, Transit, Right, Black	1
5	115553	Back Plate, Transit, Left, Black	1
6	115546	Strike, Ø.364	2
7	115573	M6 x 25, BHCS, CLS 10.9, NYL, BLZ	2
8	103284	M6 x 35 Shoulder Bolt BZ	2
9	103022	M5 Washer, Blk Zn	2
10	103283	M5 Top Lock Nut BZ	2
12	115570	Eternal Retaining Ring, 3/8"	2
15	115545	M6 x 1.0 x 50 SHCS, FT, NYL, BLZ	2
16	115571	Eternal Retaining Ring, 5/8"	2
17	003846	M6 x 50 BHCS, Custom, Blk Zc	4
18	003848	Back Bracket Saddle 1 1/8"	2
19	003849	Saddle For Dimple 1 1/8"	4
20	101853	M6 Washer Blk Zn	4
21	100658	M6 Nylock Nut Blk Zn	4
22	115539	Release Pin	2
23	115547	Compression Spring, .66" OD x 1" LG	2
26	000142	M5 x 45 BHCS, P/THD, BLZ	1
27	100657	M5 Nylock Nut Blk Zn	2
28	102801	M5 x 16, Low SHCS, Blk Zn	4
29a	115554	Backrest Actuation Bar, 12" Wide, Black	2
29b	115555	Backrest Actuation Bar, 13" Wide, Black	1
29c	115556	Backrest Actuation Bar, 14" Wide, Black	1

Item Number	Part Number	Description	Quantity
29d	115557	Backrest Actuation Bar, 15" Wide, Black	1
29e	115558	Backrest Actuation Bar, 16" Wide, Black	1
29f	115559	Backrest Actuation Bar, 17" Wide, Black	1
29g	115560	Backrest Actuation Bar, 18" Wide, Black	1
29h	115561	Backrest Actuation Bar, 19" Wide, Black	1
29i	115562	Backrest Actuation Bar, 20" Wide, Black	1
29j	117921_ _ _	Backrest Actuation Bar, 12" Wide, Painted	1
29k	117922_ _ _	Backrest Actuation Bar, 13" Wide, Painted	1
29l	117923_ _ _	Backrest Actuation Bar, 14" Wide, Painted	1
29m	117924_ _ _	Backrest Actuation Bar, 15" Wide, Painted	1
29n	117925_ _ _	Backrest Actuation Bar, 16" Wide, Painted	1
29o	117926_ _ _	Backrest Actuation Bar, 17" Wide, Painted	1
29p	117927_ _ _	Backrest Actuation Bar, 18" Wide, Painted	1
29q	117928_ _ _	Backrest Actuation Bar, 19" Wide, Painted	1
29r	117929_ _ _	Backrest Actuation Bar, 20" Wide, Painted	1
30	115549	Relax Bumper	2
31	115548	Relax Saddle	2
32	115572	M5 x 0.8 x 6 BHCS BLZ, CLS 10.9	2

Color Anodizing Samples

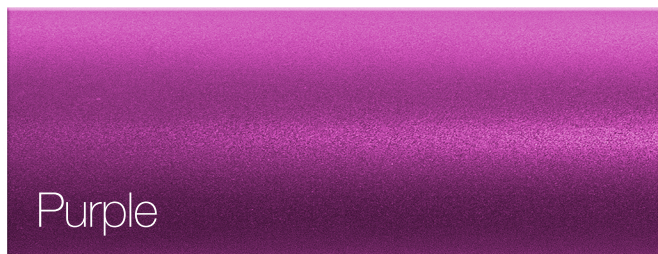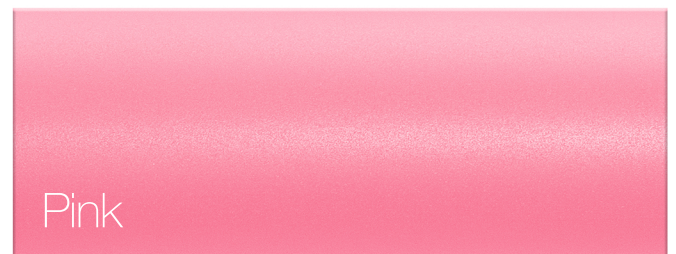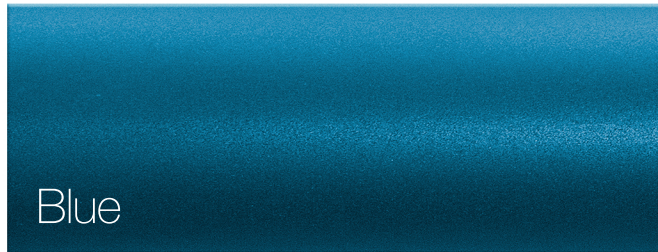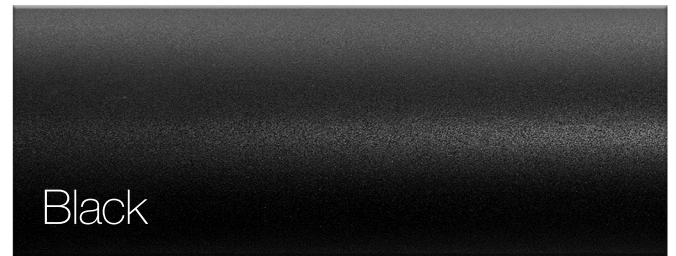

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
BRD	Red
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer