


# Flip for x:panda Base Frame

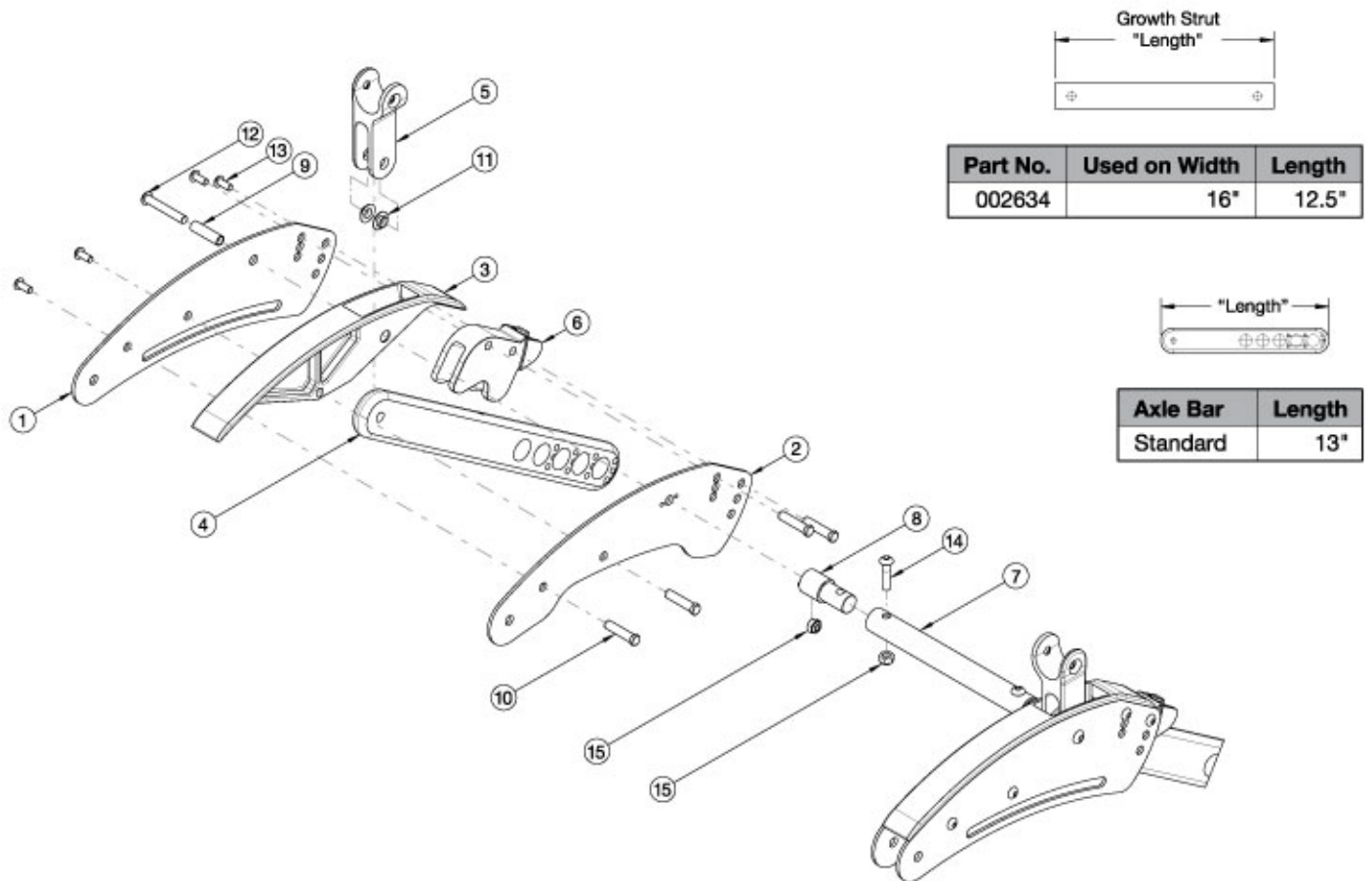

Please Note: The color for the Base Frame Assemblies and Frame Plates is the "Accent Color" for the Flip.

Item Number	Part Number	Description	Quantity
1, 2a, 3, 4, 5, 6, 8-15	502791_ _ _	Base Frame Assembly Std, Right	1
1, 2b, 3, 4, 5, 6, 8-15	502792_ _ _	Base Frame Assembly Std, Left	1
1	001818_ _ _	Frame Plate, Outer	1
2a	002624_ _ _	Frame Plate, Right Inner, FLIP	1
2b	003240_ _ _	Frame Plate, Left Inner, FLIP	1
3	001819	Cover, Frame Plate	1
4	001838	Axle Bar, Standard	1
5	001878	Tilt Yoke	1
6	001837	Saddle, Axle Bar	1

Item Number	Part Number	Description	Quantity
7	002634	Frame Growth Strut 12.5"	1
8	002626	Strut Mount, Rear	1
9	001820	Spacer, Frame	1
10	000987	Threaded Fastener	4
11	102786	Iglide Flange Bushing	2
12	100670	M6 x 45 BTN SHCS BLK ZC	1
13	100669	M5 x 10 BTN SHCS BLKZC w/Patch	4
14	102792	M6 x 25, HHCS, Blk Zn	1
15	100658	M6 Nylock Nut Blk Zn	2