


# Arc Axle Plate

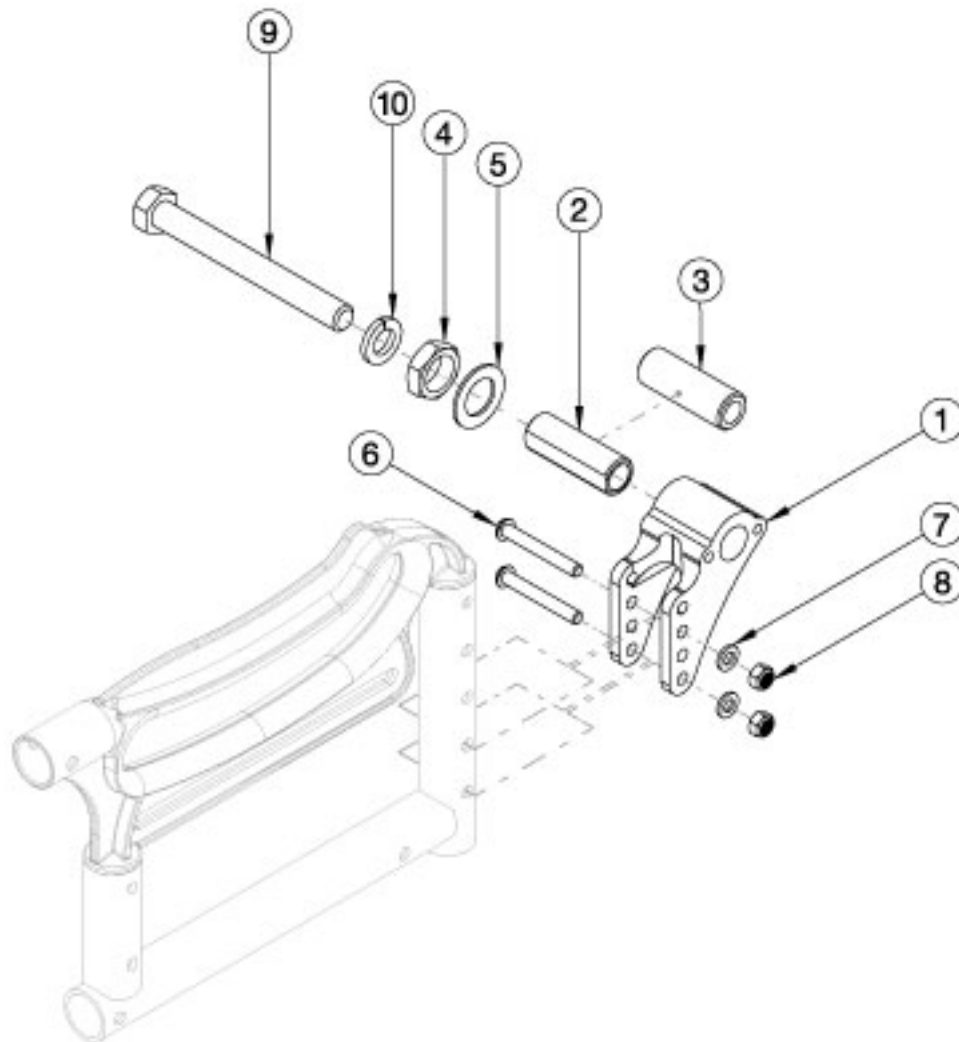

If a drum brake is present, please navigate to the "Arc Drum Brake" parts manual page.

Item Number	Part Number	Description	Quantity
1, 2, 4-8	113285	Axle Plate Assembly, Quick Release, Arc <i>Used with Quick Release axles. Contains parts for 1 side.</i>	2
1-8, 10	113286	Axle Plate Assembly, Fixed, Arc <i>Used with Fixed axles. Contains parts for 1 side.</i>	2
1	110186	Axle Plate, Standard	1
2	000985	Axle Receiver with Flats M18 x 1/2" ID <i>Used with Quick Release axles</i>	1
3	003239	Axle Receiver, Threaded Axle <i>Used with Fixed axles.</i>	1
4	100514	Axle Receiver Nut M18 x D12 x 8mm	1

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
5	004445	Axle Plate Flat Washer	1
6	100670	M6 x 45 BTN SHCS BLK ZC	2
7	100746	M6 Flat Washer Blk Zn	2
8	100658	M6 Nylock Nut Blk Zn	2
9	110293	HHB, 1/2-13 x 4.5, Grade 8, BLZ	1
10	110191	Washer, Lock, 1/2", ZC	1