
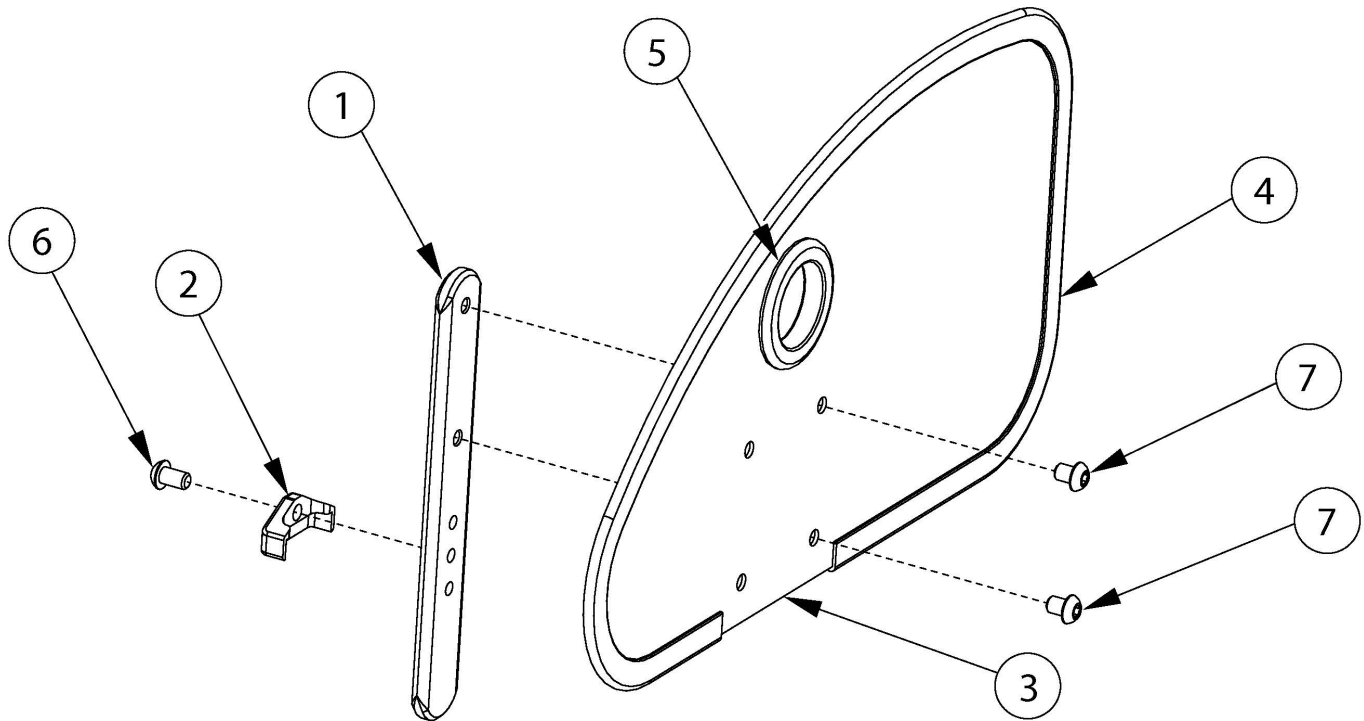


Aluminum Straight Side Guard

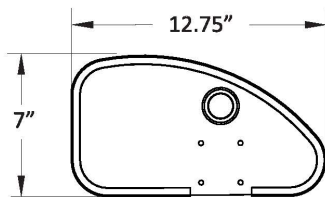

If retrofitting, side guard receivers may be required

Item Number	Part Number	Description	Quantity	Retail
1, 2, 3a, 4-7	108346	Aluminum Straight Side Guard Assembly Left, Black Matte	1	\$96.52
1, 2, 3a, 4-7	108347	Aluminum Straight Side Guard Assembly Right, Black Matte	1	\$96.52
1, 2, 3a, 4-7	108348	Aluminum Straight Side Guard Assembly (PAIR), Black Matte	1	\$193.05
1, 2, 3b, 4-7	108349_ _ _	Aluminum Straight Side Guard Assembly Left, Match Frame Color	1	\$161.80
1, 2, 3b, 4-7	108350_ _ _	Aluminum Straight Side Guard Assembly Right, Match Frame Color	1	\$161.80
1, 2, 3b, 4-7	108351_ _ _	Aluminum Straight Side Guard Assembly (PAIR), Match Frame Color	1	\$323.61
1	000140	Side Guard Post	1	\$25.03
2	001342	Side Guard Post Stop	1	\$21.87

Item Number	Part Number	Description	Quantity	Retail
3a, 4, 5	108342	Aluminum Straight Side Guard Left, Black Matte	1	\$44.88
3a, 4, 5	108343	Aluminum Straight Side Guard Right, Black Matte	1	\$44.88
3b, 4, 5	108344_ _ _	Aluminum Straight Side Guard Left, Match Frame Color	1	\$110.16
3b, 4, 5	108345_ _ _	Aluminum Straight Side Guard Right, Match Frame Color	1	\$110.16
6	101184	M6 x 10 BTN SHCS BLKZC w/Patch	1	\$1.58
7	101700	M6 x 8 BTN SHCS BLK ZC	2	\$1.58

Matte Finish	
Code	Color Name
AGM	Army Green
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia
GSM	Granny Smith
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer

* Discontinued Color
10-05-2021