


Liberty Rigid Cross Braces

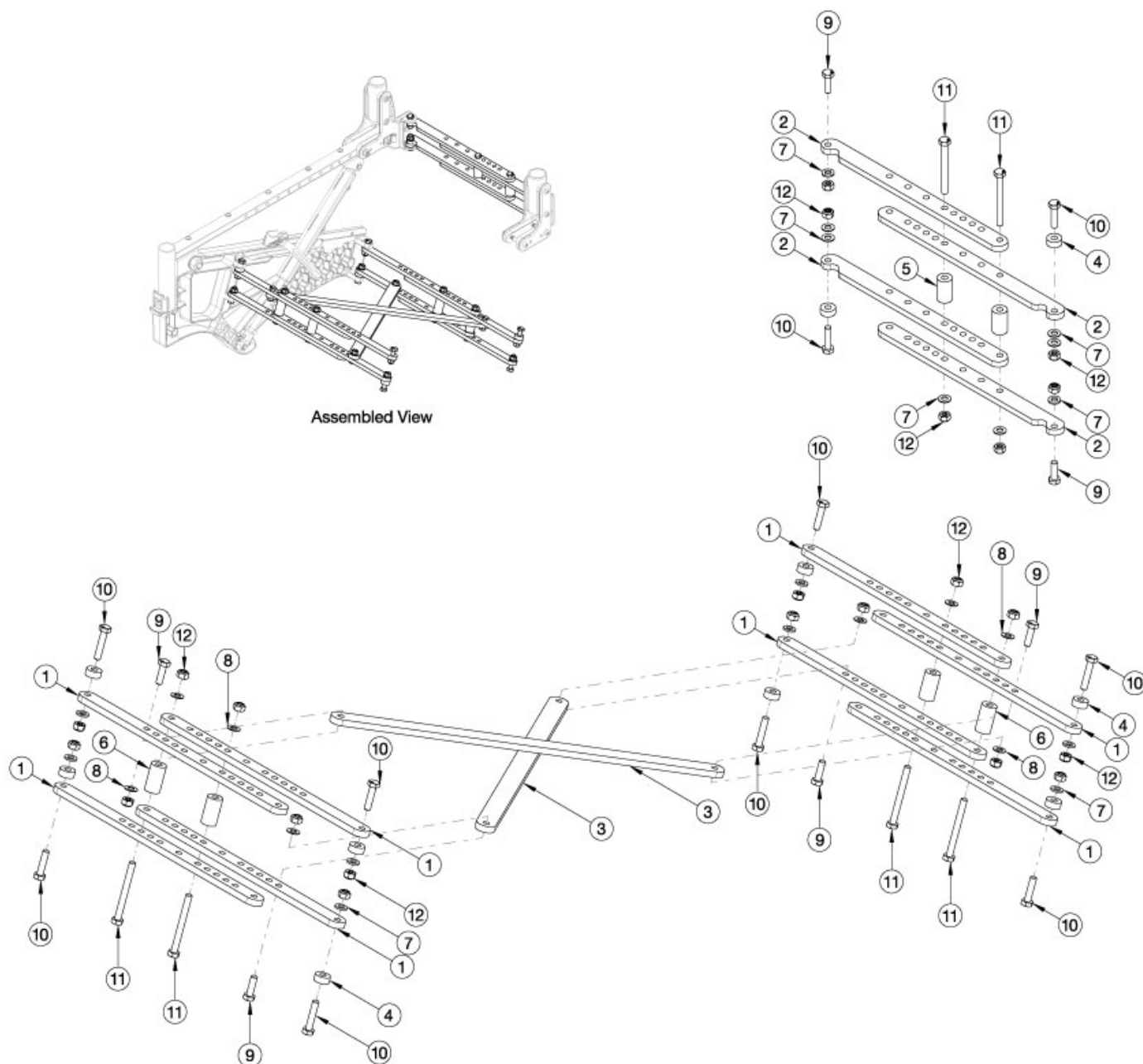

These parts are Not Retrofittable. If you have a Folding Liberty Frame, please see the "Liberty Non-Folding Cross Brace" page.

Item Number	Part Number	Description	Quantity
1a, 4, 6-12	118066	Rigid Lower Crossbrace Kit with Hardware, Small, Liberty <i>Used with widths 14"-18"</i>	1
1b, 4, 6-12	118067	Rigid Lower Crossbrace Kit with Hardware, Medium, Liberty <i>Used with widths 16"-20"</i>	1

Item Number	Part Number	Description	Quantity
1c, 4, 6-12	118068	Rigid Lower Crossbrace Kit with Hardware, Large, Liberty <i>Used with Widths 18"-22"</i>	1
2a, 4, 5, 7, 9-12	118069	Rigid Upper Crossbrace Kit with Hardware, Small, Liberty <i>Used with widths 14"-18"</i>	1
2b, 4, 5, 7, 9-12	118070	Rigid Upper Crossbrace Kit with Hardware, Medium, Liberty <i>Used with widths 16"-20"</i>	1
2c, 4, 5, 7, 9-12	118071	Rigid Upper Crossbrace Kit with Hardware, Large, Liberty <i>Used with Widths 18"-22"</i>	1
3, 8, 9, 12	118072	Rigid Stabilizer Link Kit, Liberty <i>Reusable part - Not required if chair is already a Rigid version</i>	1
4-12	118073	Rigid Hardware Kit, Full Package, Liberty	1
1a	116433	Cross Link, Base, Small, Liberty <i>Used with widths 14"-18"</i>	1
1b	116434	Cross Link, Base, Medium, Liberty <i>Used with widths 16"-20"</i>	1
1c	116435	Cross Link, Base, Large, Liberty <i>Used with Widths 18"-22"</i>	1
2a	116436	Cross Link, Backrest, Small, Liberty <i>Used with widths 14"-18"</i>	1
2b	116437	Cross Link, Backrest, Medium, Liberty <i>Used with widths 16"-20"</i>	1
2c	116438	Cross Link, Backrest, Large, Liberty <i>Used with Widths 18"-22"</i>	1
3	116439	Stabilizer Link, Liberty	1
4	116440	Spacer, .59 OD x .251 ID x .250 L	1
5	116441	Spacer, .59 OD x .251 ID x .998 L	1
6	116442	Spacer, .59 OD x .251 ID x 1.073 L	1
7	100746	M6 Flat Washer Blk Zn	1
8	102803	Washer Flat .250 X .500 X .027 Hx Blz	1
9	101823	M6 X 20 Hhcs Blk Zn	1
10	116854	HHCS, M6x1, 0x30, CLS10.9, PT, BLZ, DIN931	1
11	100677	M6 x 60 CLS10.9 DIN931 HHCS ZC	1

Item Number	Part Number	Description	Quantity
12	100658	M6 Nylock Nut Blk Zn	1