



(Discontinued 1) Rogue Fixed Height Backrest with Adjustable Height Rigidizer Bar

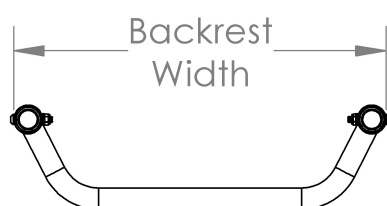

Backrest 10"	10.3"
Backrest 11"	11.3"
Backrest 12"	12.3"
Backrest 13"	13.3"
Backrest 14"	14.3"
Backrest 15"	15.3"
Backrest 16"	16.3"
Backrest 17"	17.3"
Backrest 18"	18.3"
Backrest 19"	19.3"
Backrest 20"	20.3"
Backrest 21"	21.3"
Backrest 22"	22.3"

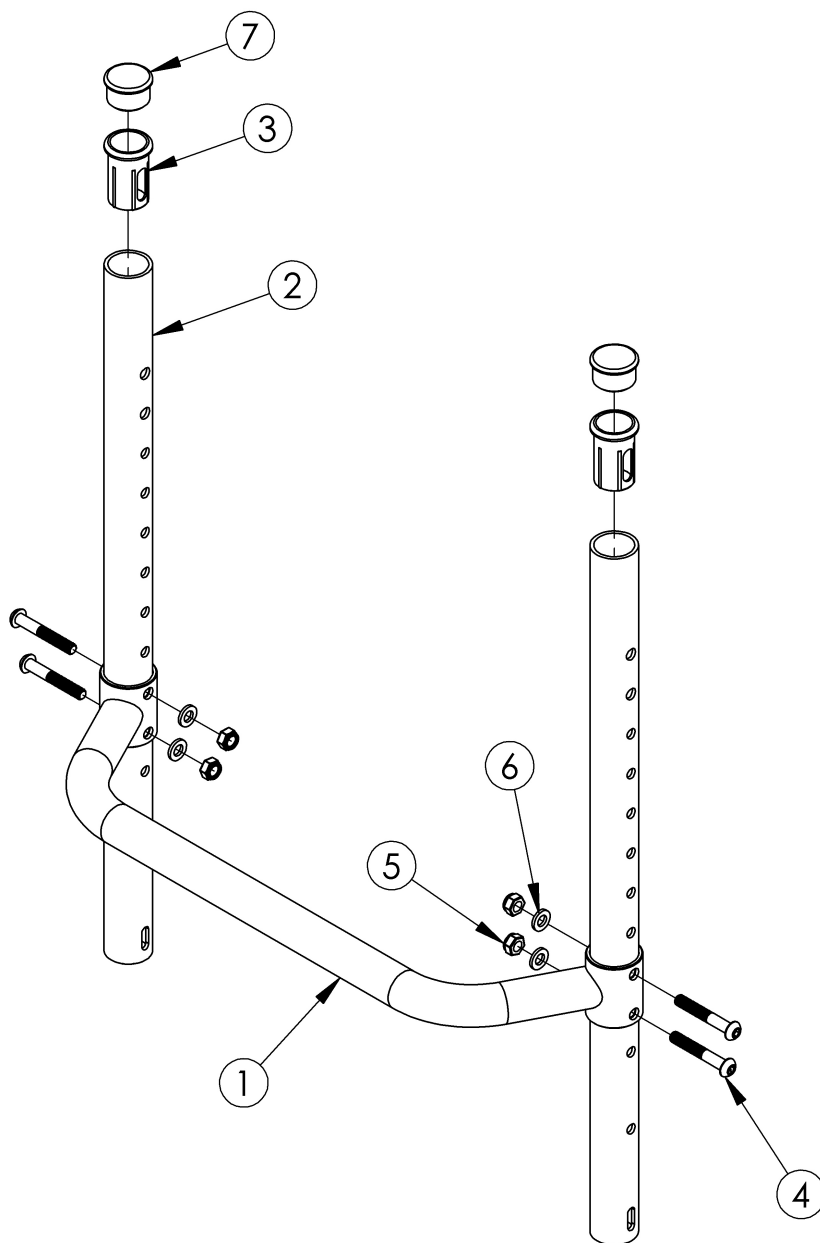

This Rogue Fixed Height Backrest with Adjustable Height Rigidizer Bar is discontinued and was used on serial numbers prior to RG0016612.

Item Number	Part Number	Description	Quantity
1a	000527	Adjustable Height Rigidizer Bar 12" Wide	1
1b	000528	Adjustable Height Rigidizer Bar 13" Wide	1
1c	000185	Adjustable Height Rigidizer Bar 14" Wide	1

Item Number	Part Number	Description	Quantity
1d	000186	Adjustable Height Rigidizer Bar 15" Wide	1
1e	000187	Adjustable Height Rigidizer Bar 16" Wide	1
1f	000188	Adjustable Height Rigidizer Bar 17" Wide	1
1g	000189	Adjustable Height Rigidizer Bar 18" Wide	1
1h	000190	Adjustable Height Rigidizer Bar 19" Wide	1
1i	000191	Adjustable Height Rigidizer Bar 20" Wide	1
2	000698	Backrest Tube Fixed Height, 17.25"	2
3	000034	Sleeve 3/4" x 1 1/2"	2
4	100797	M6 x 40 BHCS BLK ZC	4
5	100658	M6 Nylock Nut Blk Zn	4
6	100746	M6 Flat Washer Blk Zn	4
7	100804	Tube End Plug 1"	2