


Arc Front Side Frame

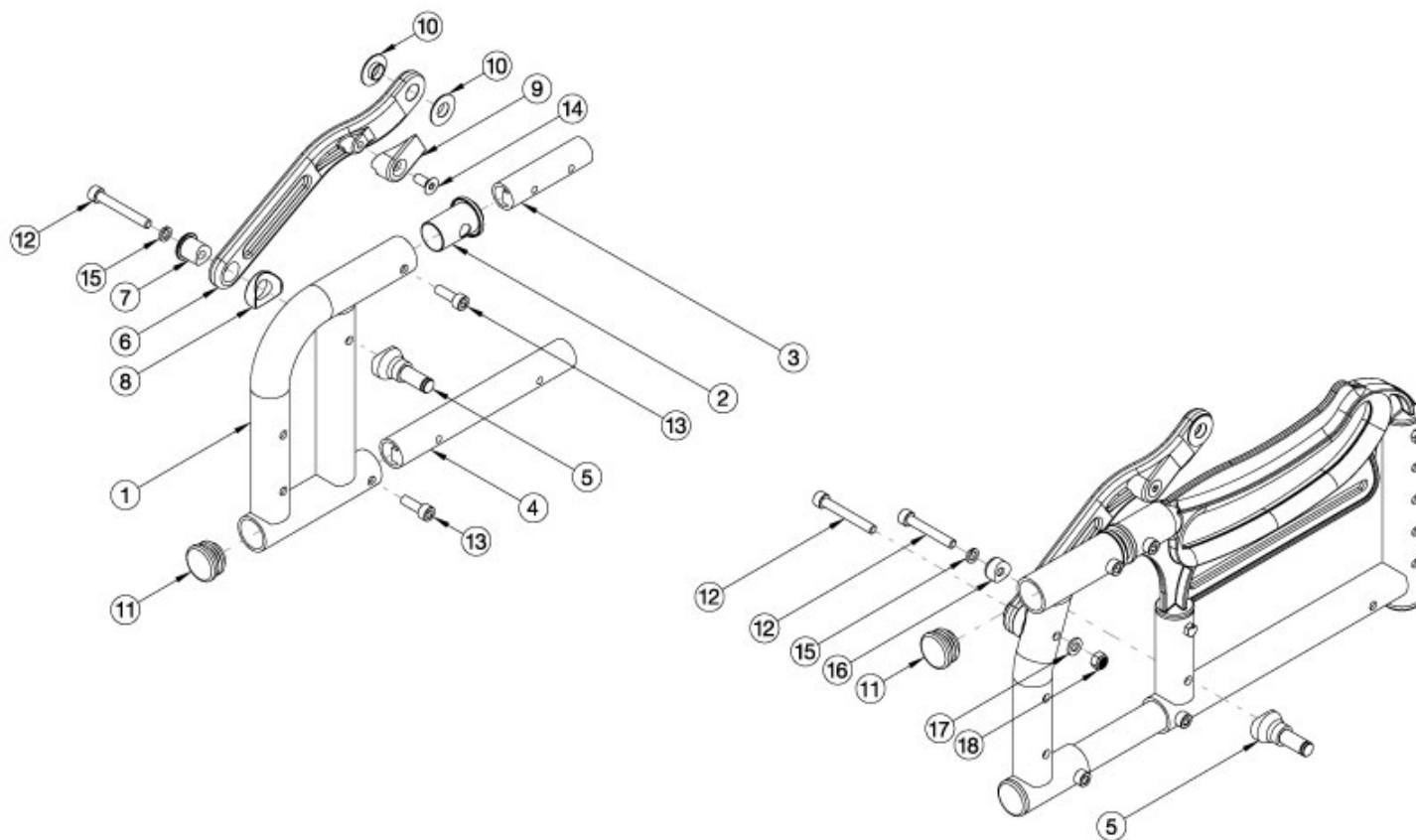

Extra Short

Select color of parts based on Base Frame color of the chair. When changing frame depth ranges, new seat frame assemblies are needed. If quoting a Left Front Frame for Australia or New Zealand, 122921 is required when transit is not present.

Item Number	Part Number	Description	Quantity
1a, 2, 11	113109_ _ _	Front Side Frame, Extra Short, Right, Arc <i>Accommodates 12-15" Depths</i>	1
1b, 2, 11	113110_ _ _	Front Side Frame, Extra Short, Left, Arc <i>Accommodates 12-15" Depths</i>	1
1c, 2, 11	113111_ _ _	Front Side Frame, Short, Right, Arc <i>Accommodates 14-17" Depths</i>	1
1d, 2, 11	113112_ _ _	Front Side Frame, Short, Left, Arc <i>Accommodates 14-17" Depths</i>	1
1e, 2, 11	113113_ _ _	Front Side Frame, Medium, Right, Arc <i>Accommodates 16-19" Depths</i>	1
1f, 2, 11	113114_ _ _	Front Side Frame, Medium, Left, Arc <i>Accommodates 16-19" Depths</i>	1

Item Number	Part Number	Description	Quantity
1g, 2, 11	113115_ _ _	Front Side Frame, Long, Right, Arc <i>Accommodates 18-20" Depths</i>	1
1h, 2, 11	113116_ _ _	Front Side Frame, Long, Left, Arc <i>Accommodates 18-20" Depths</i>	1
2	110149	Bushing, Pivot, X-Hinge	1
3	110162	Side Frame Pivot Tube, Upper	1
4	110163	Side Frame Pivot Tube, Lower	1
5	110210	Mount, Mechlock, Lower	1
6	110212	Link Arm	1
7	110214	Link Arm Lower Bushing	1
8	110215	Saddle, Link Arm	1
9	110270	Bumper, Tilt Stop	1
10	110253	Bushing, Flg, .397IDx.87ODx.175Lg	2
11	100804	Tube End Plug 1"	1
12	101472	M6 x 45 SHCS BLK ZC	1
13	101179	M6 X 10 Shcs Blk Zc W/Patch	2
14	101594	M6 x 14 CLS10.9 DIN7991 FSHCS BLKZC w/Patch	1
15	102810	M6 Lockwasher, Hi-Collar, ZC	1
16	110216	Mechlock Mount Saddle	1
17	100746	M6 Flat Washer Blk Zn	1
18	100658	M6 Nylock Nut Blk Zn	1
	122921	Not AS/NZS 3696.19 Compliant On-Chair Label with Instruction	1