


Clik / Ethos Positioning Belt Clamp

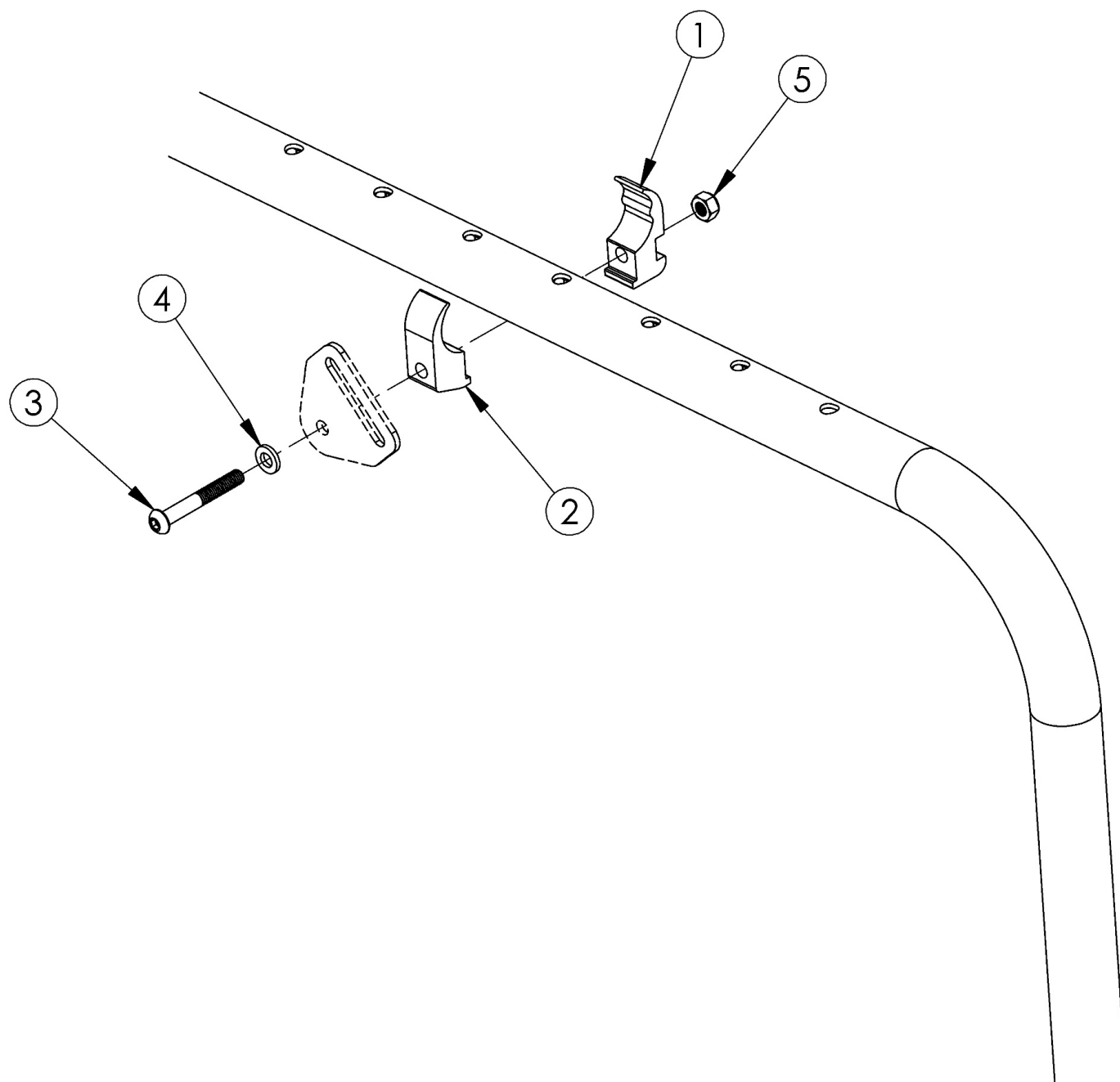

Item Number	Part Number	Description	Quantity
1-5	503234	Ki Mobility Belt Mounting Clamp Kit (PAIR) Clik / Ethos	1
1	002792	Seat Belt Clamp A, 1.125" Tubing	2
2	002793	Seat Belt Clamp B, 1.125" Tubing	2

Item Number	Part Number	Description	Quantity
3	100670	M6 x 45 BTN SHCS BLK ZC	2
4	100746	M6 Flat Washer Blk Zn	2
5	100658	M6 Nylock Nut Blk Zn	2