


(Discontinued) Catalyst Height Adjustable Depth Adjustable Backrest with Hardware

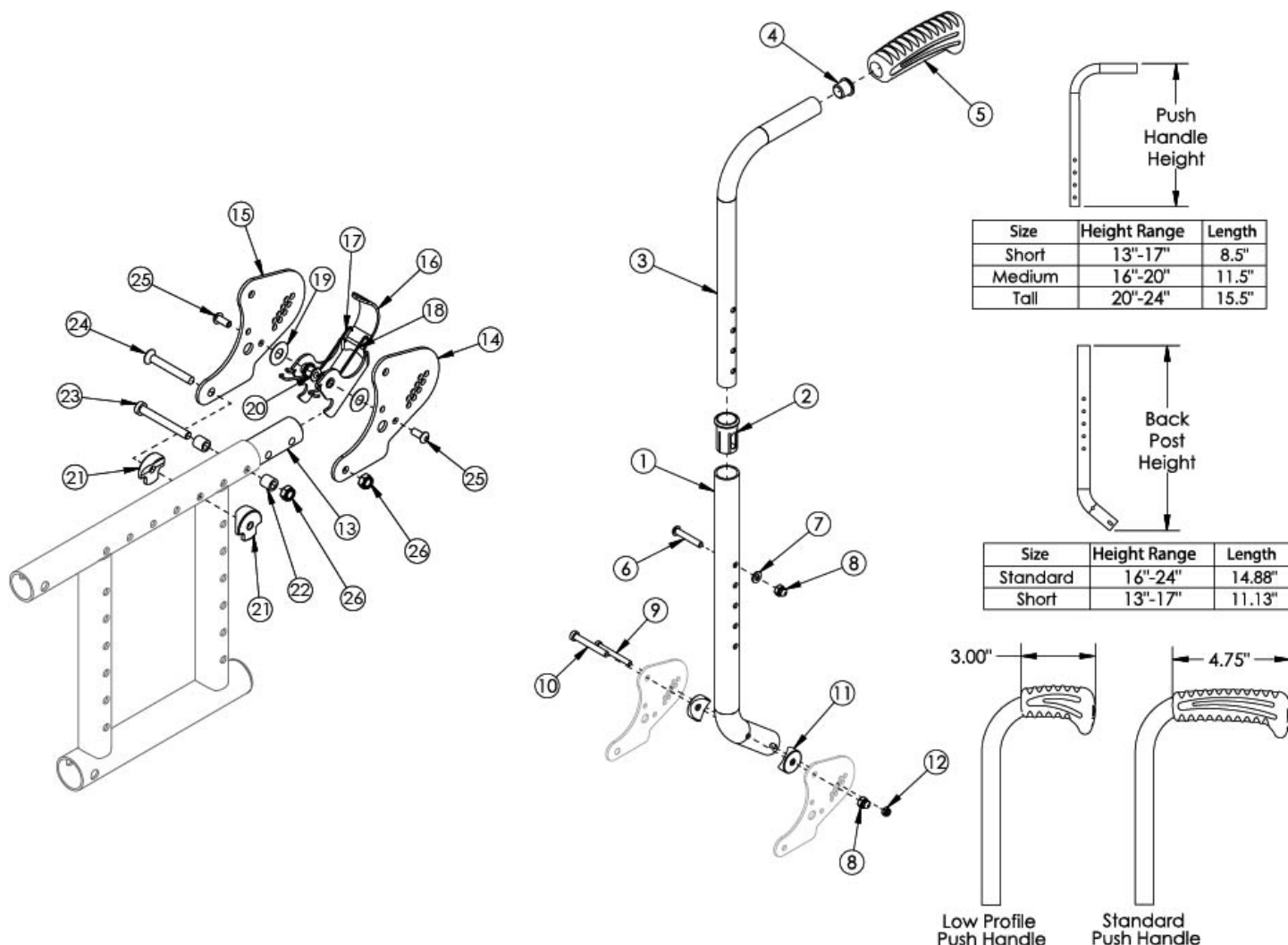

This backrest style discontinued February 8th, 2021 Not compatible with Drum Brake. A backrest retrofit to the Catalyst Height Adjustable Straight with Push Handle (Depth Adjustable) is required if retrofitting Drum Brake.

Item Number	Part Number	Description	Quantity
1a, 2, 3b, 4, 5a - 13, 14b, 15-26	111320	Backrest Assembly Depth Adjustable Height Adjustable Medium w/ Standard Grip w/ Attaching Hardware - Right Side	1
1a, 2, 3c, 4, 5a - 13, 14b, 15-26	111322	Backrest Assembly Depth Adjustable Height Adjustable Tall w/ Standard Grip w/ Attaching Hardware - Right Side	1
1a, 2, 3b, 4, 5b - 13, 14b, 15-26	111326	Backrest Assembly Depth Adjustable Height Adjustable Medium w/ Low Profile Grip w/ Attaching Hardware - Right Side	1
1a, 2, 3c, 4, 5b - 13, 14b, 15-26	111328	Backrest Assembly Depth Adjustable Height Adjustable Tall w/ Low Profile Grip w/ Attaching Hardware - Right Side	1

Item Number	Part Number	Description	Quantity
1a, 2, 3b, 4, 5a - 8	501257	Backrest Assembly Depth Adjustable Height Adjustable Medium w/ Standard Grip	1
1a, 2, 3c, 4, 5a - 8	501258	Backrest Assembly Depth Adjustable Height Adjustable Tall w/ Standard Grip	1
1a, 2, 3b, 4, 5b - 8	503296	Backrest Assembly Depth Adjustable Height Adjustable Medium w/ Low Profile Grip	1
1a, 2, 3c, 4, 5b - 8	503297	Backrest Assembly Depth Adjustable Height Adjustable Tall w/ Low Profile Grip	1
3a, 4, 5a	501654	Depth Adjustable Height Adjustable Upper Push Handle Short Assembly w/ Standard Grip	1
3b, 4, 5a	501655	Depth Adjustable Height Adjustable Upper Push Handle Medium Assembly w/ Standard Grip	1
3c, 4, 5a	501656	Depth Adjustable Height Adjustable Upper Push Handle Tall Assembly w/ Standard Grip	1
3a, 4, 5b	002762	Depth Adjustable Height Adjustable Upper Push Handle Short Assembly w/ Low Profile Grip	1
3b, 4, 5b	002763	Depth Adjustable Height Adjustable Upper Push Handle Medium Assembly w/ Low Profile Grip	1
3c, 4, 5b	002764	Depth Adjustable Height Adjustable Upper Push Handle Tall Assembly w/ Low Profile Grip	1
13, 14b, 15-26	500739	Depth Adjustable Backrest Hardware Assm Right CAT	1
13, 15-26	501227	Depth Adjustable Backrest Hardware Assm SPARK/ OPEN SEAT	1
8-12	111164	Depth Adjustable Backrest Attaching Hardware Kit	1
1a	000986	Depth Adjustable Straight Back Tube Medium/Tall CAT/ SPARK	1
2	000034	Sleeve 3/4" x 1 1/2"	1
4	100638	Tube End Plug 3/4"	1
5a	109432	Ergo Push Handle Grip 3/4" Standard with Instruction	1
5b	109430	Ergo Push Handle Grip 3/4" Low Profile with Instruction	1
6	101811	M6 x 35 BHCS Blk Zn	1
7	100746	M6 Flat Washer Blk Zn	1
8	100558	M6 Nyl Ins Domed Cap L/N Zc	1
9	101186	M5 X 45 CLS12.9 DIN912 SHCS P/THD BLZ	1
10	101123	M6 x 45 Low SHCS BLK ZC	1
11	000055	Saddle 1"	2
12	100657	M5 Nylock Nut Blk Zn	1
13	000605	Depth Adjustable Frame Extension Tube, Catalyst/Spark	1
14b	000604R	Depth Adjustable Backrest Plate Inside Right CAT/ SPARK	1

Item Number	Part Number	Description	Quantity
15	000611	Depth Adjustable Backrest Side Plate CAT/ SPARK	2
16	000670	Depth Adjustable Backrest Latch CAT/ SPARK	1
17	000613R	Depth Adjustable Backrest Latch Spring Right CAT/ SPARK	1
18	000613L	Depth Adjustable Backrest Latch Spring Left CAT/ SPARK	1
19	000428	Wear Washer	2
20	000420	Backrest Latch Pivot Bushing	2
21	000612	1" Half Saddle Washer	2
22	000240	Backrest Latch Strike Bushing / Close Mount Handrim Spacer	2
23	101124	M6 x 50 Low SHCS BLK ZC	1
24	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	1
25	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2
26	100658	M6 Nylock Nut Blk Zn	2