

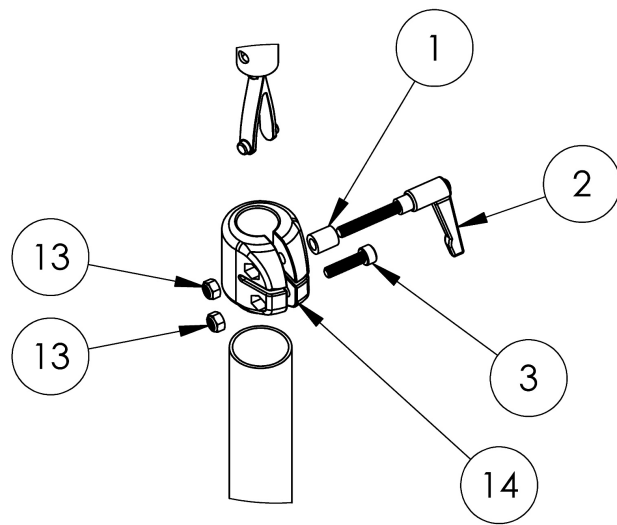



Catalyst IV Pole

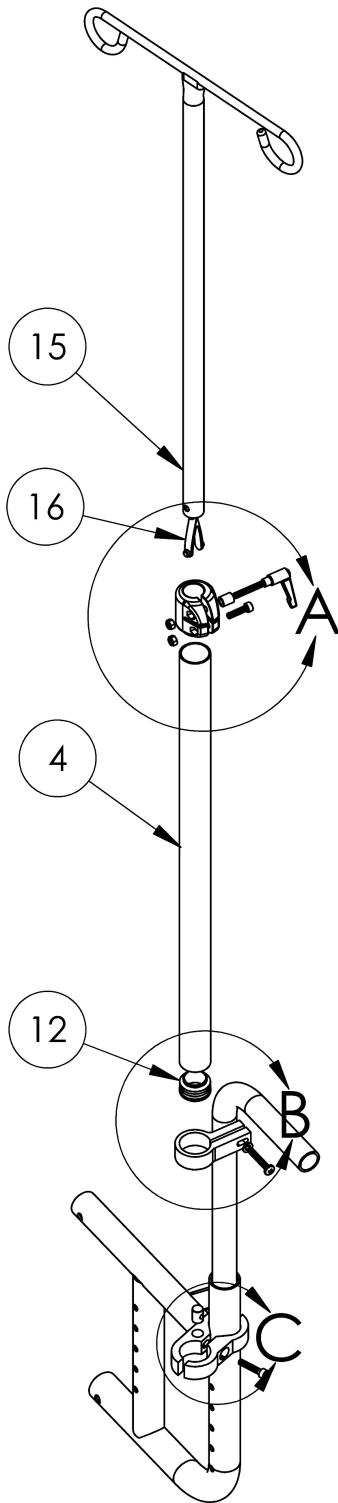

DETAIL A

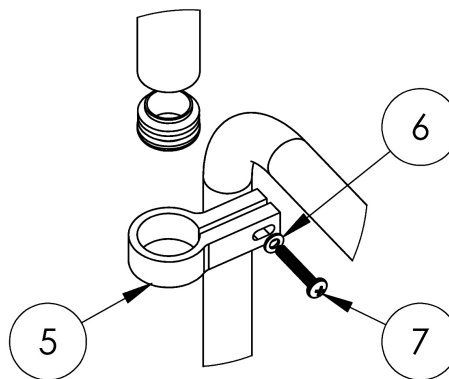

DETAIL B

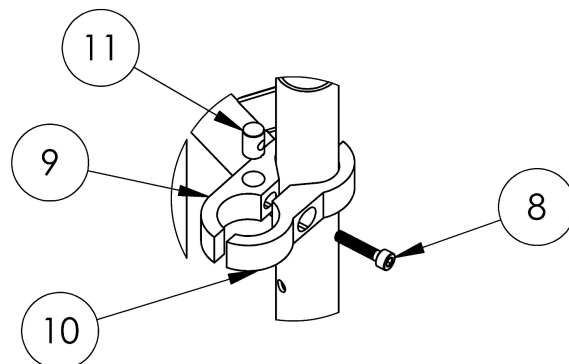

DETAIL C

Item Number	Part Number	Description	Quantity	Retail
1-3, 4a, 5-7, 8a, 8b, 9-16	502404	IV Pole Assembly 9-12" Back Height, Catalyst	1	\$275.00
1-3, 4b, 5-7, 8a, 8b, 9-17	502405	IV Pole Assembly 13-16" Back Height, CAT/LFT	1	\$275.00
1-3, 4c, 5-7, 8a, 8b, 9-18	502406	IV Pole Assembly 17-20" Back Height, CAT/LFT	1	\$275.00
1, 2, 3, 13, 14	502401	IV Pole Top Clamp Assembly	1	\$44.49
5, 6, 7	502402	IV Pole Middle Clamp Assembly, Catalyst	1	\$39.13
8a, 8b, 9, 10, 11	502403	IV Pole Bottom Clamp Assembly, Catalyst	1	\$77.68
1	102766	Spacer 3/8R. X 7/16 x .218 ID.	1	\$7.07
2	102767	Handle wM5 x 32L Threaded Stud	1	\$29.48
3	101843	M5 x 20 SHCS Blk Zn	1	\$0.53
4a	002670	IV Pole Outer Tube 20" Height	1	\$30.02
4b	002669	IV Pole Outer Tube 24" Height	1	\$32.16
4c	002581	IV Pole Outer Tube 28" Height	1	\$34.30
5	002707	IV Pole Clamp Single 1-1/8"	1	\$37.52
6	101856	Flat Washer, M5, Blk Zn	1	\$0.53
7	100696	M5 x 30mm PPH Capscrew	1	\$1.58
8a	100780	M6 x 25 SHCS ZC <i>Used on Catalyst 5 chairs</i>	1	\$2.10
8b	102627	M6 X 30 SHCS BLK ZC <i>Used on Catalyst 4 chairs</i>	1	\$1.05
9	002685	IV Pole Clamp 1-1/8" 1" Thru 1-1/4" w/ Cross Dowel Hole	1	\$37.52
10	002686	IV Pole Clamp 1-1/8" 1" Thru 1-1/4"	1	\$37.52
12	100532	Tube End Plug 1 1/8"	1	\$2.63
13	100657	M5 Nylock Nut Blk Zn	2	\$0.53
14	002395	Footrest Clamp Left LW	1	\$6.97
15	002585	IV Pole Upper	1	\$26.81
16	102769	Snap Button Plastic	1	\$3.49