


Flip Drum Brake - Growth

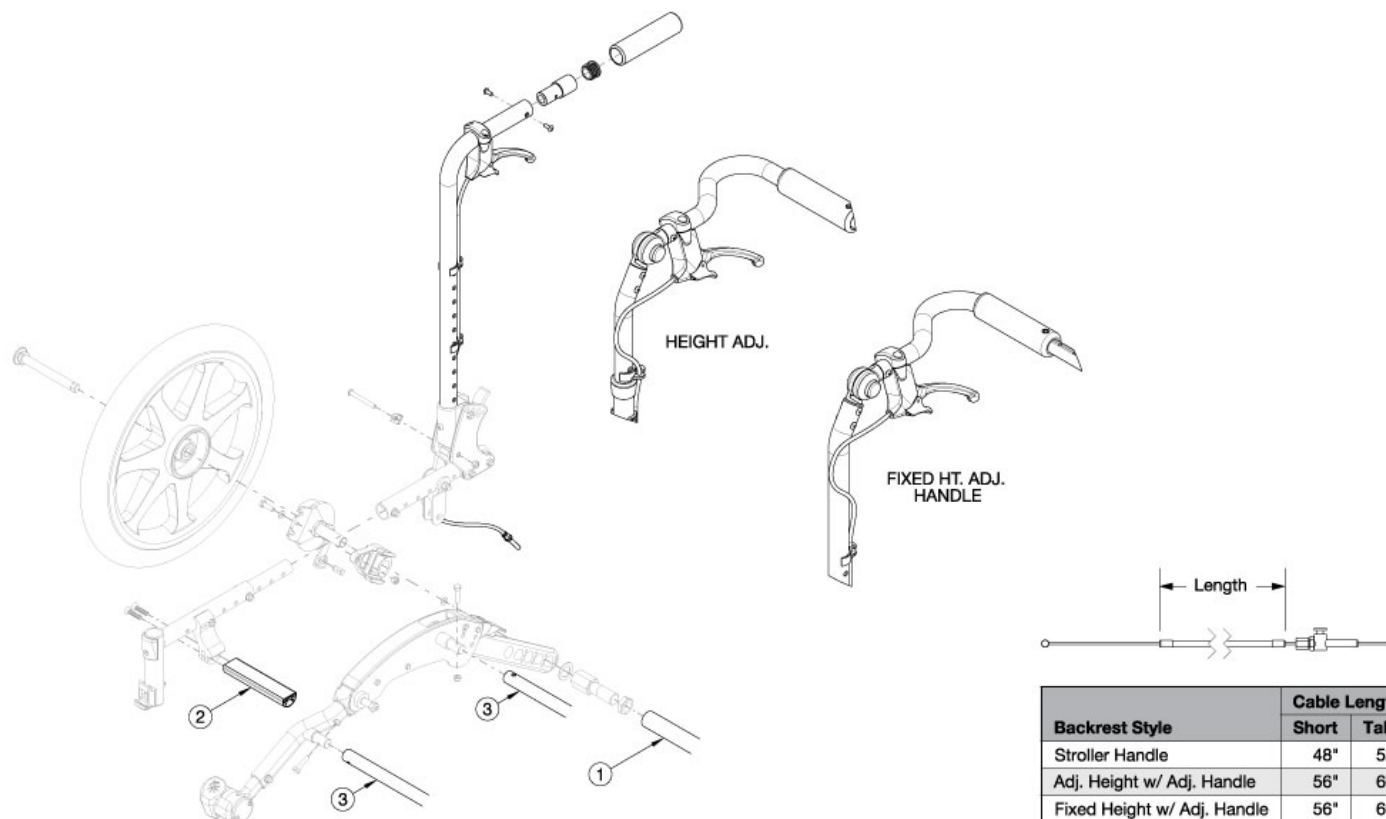

Has your width changed? If yes, please select the appropriate strut kit.

Item Number	Part Number	Description	Quantity
1a, 2a, 3a, 3d	502868	Strut Kit 10" Wide Flip w/ Drum Brake	1
1b, 2b, 3b, 3e	502869	Strut Kit 11" Wide Flip w/ Drum Brake	1
1c, 2c, 3c, 3f	502870	Strut Kit 12" Wide Flip w/ Drum Brake	1
1d, 2d, 3d, 3g	502871	Strut Kit 13" Wide Flip w/ Drum Brake	1
1e, 2e, 3e, 3h	502872	Strut Kit 14" Wide Flip w/ Drum Brake	1
1f, 2f, 3f, 3i	502873	Strut Kit 15" Wide Flip w/ Drum Brake	1
1g, 2g, 3g, 3j	502874	Strut Kit 16" Wide Flip w/ Drum Brake	1
1h, 2h, 3h, 3k	502875	Strut Kit 17" Wide Flip w/ Drum Brake	1
1i, 2i, 3i, 3l	502876	Strut Kit 18" Wide Flip w/ Drum Brake	1
1a	002565	Strut, Drum Brake, Ped, 10W	1
1b	002566	Strut, Drum Brake, Ped, 11W	1

Item Number	Part Number	Description	Quantity
1c	002567	Strut, Drum Brake, Ped, 12W	1
1d	002568	Strut, Drum Brake, Ped, 13W	1
1e	002569	Strut, Drum Brake, Ped, 14W	1
1f	002570	Strut, Drum Brake, Ped, 15W	1
1g	002571	Strut, Drum Brake, Ped, 16W	1
1h	002572	Strut, Drum Brake, Ped, 17W	1
1i	002573	Strut, Drum Brake, Ped, 18W	1
2a	001879	Strut Tube Seat 10 Wide	1
2b	001880	Strut Tube Seat 11 Wide	1
2c	001881	Strut Tube Seat 12 Wide	1
2d	001882	Strut Tube Seat 13 Wide	1
2e	001883	Strut Tube Seat 14 Wide	1
2f	001884	Strut Tube Seat 15 Wide	1
2g	001885	Strut Tube Seat 16 Wide	1
2h	001886	Strut Tube Seat 17 Wide	1
2i	001887	Strut Tube Seat 18 Wide	1
3a	002719	Frame Growth Strut 6.5" <i>Used on 10" wide chairs</i>	1
3b	002629	Frame Growth Strut 7.5" <i>Used on 11" wide chairs</i>	1
3c	002630	Frame Growth Strut 8.5" <i>Used on 12" wide chairs</i>	1
3d	002631	Frame Growth Strut 9.5" <i>Used on 10" and 13" wide chairs</i>	1
3e	002632	Frame Growth Strut 10.5" <i>Used on 11" and 14" wide chairs</i>	1
3f	002633	Frame Growth Strut 11.5" <i>Used on 12" and 15" wide chairs</i>	1
3g	002634	Frame Growth Strut 12.5" <i>Used on 13" and 16" wide chairs</i>	1
3h	002635	Frame Growth Strut 13.5"	1

Item Number	Part Number	Description	Quantity
		<i>Used on 14" and 17" wide chairs</i>	
3i	002636	Frame Growth Strut 14.5" <i>Used on 15" and 18" wide chairs</i>	1
3j	002637	Frame Growth Strut 15.5" <i>Used on 16" wide chairs</i>	1
3k	002638	Frame Growth Strut 16.5" <i>Used on 17" wide chairs</i>	1
3l	002639	Frame Growth Strut 17.5" <i>Used on 18" wide chairs</i>	1