


# Rogue / Rogue XP Height Adjustable Low T-Arm

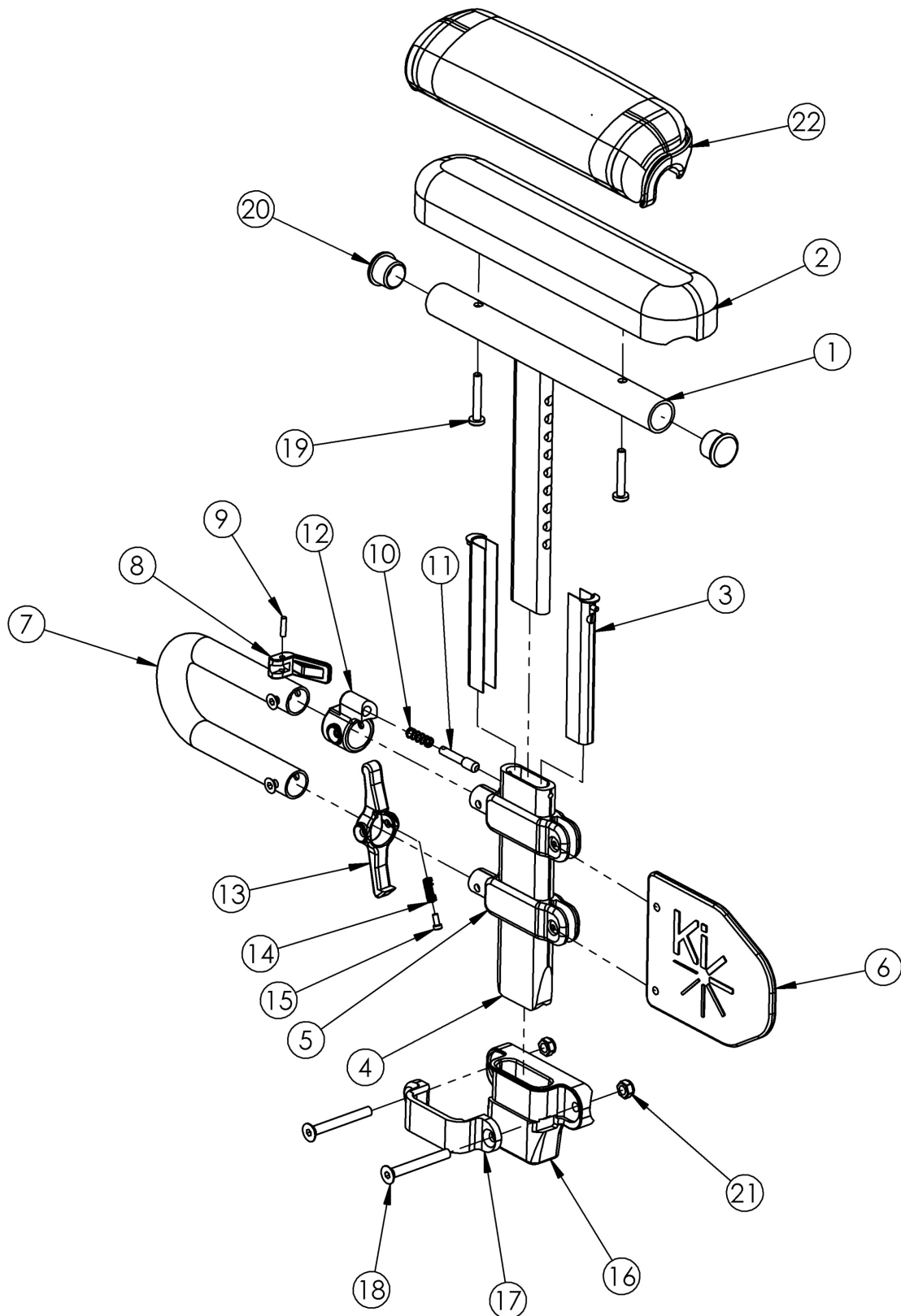

Height Adjustable Low T-Arms cannot be retrofitted

Item Number	Part Number	Description	Quantity	Retail
1a, 2b, 3-18, 19a, 20-21	113460	Height Adj T-Arm Assm Low, Desk w/ Std Pad and TSU/RG Receiver PAIR	1	\$419.56
1a, 2b, 3-18, 19a, 20-21	113444	Height Adj T-Arm Assm Low, Desk w/ Std Pad and TSU/RG Receiver Right	1	\$209.78
1a, 2b, 3-18, 19a, 20-21	113445	Height Adj T-Arm Assm Low, Desk w/ Std Pad and TSU/RG Receiver Left	1	\$209.78
1a, 2b, 3-15, 19a, 20	503167	Height Adj T-Arm Assm Low, Desk w/ Std Pad Right	1	\$160.81
1a, 2b, 3-15, 19a, 20	503168	Height Adj T-Arm Assm Low, Desk w/ Std Pad Left	1	\$160.81
1b, 2a, 3-18, 19a, 20-21	113461	Height Adj T-Arm Assm Low, Full w/ Std Pad and TSU/RG Receiver PAIR	1	\$419.56
1b, 2a, 3-18, 19a, 20-21	113446	Height Adj T-Arm Assm Low, Full w/ Std Pad and TSU/RG Receiver Right	1	\$209.78
1b, 2a, 3-18, 19a, 20-21	113447	Height Adj T-Arm Assm Low, Full w/ Std Pad and TSU/RG Receiver Left	1	\$209.78
1b, 2a, 3-15, 19a, 20	503169	Height Adj T-Arm Assm Low, Full w/ Std Pad Right	1	\$160.81
1b, 2a, 3-15, 19a, 20	503170	Height Adj T-Arm Assm Low, Full w/ Std Pad Left	1	\$160.81
1a, 3-18, 19b, 20-21, 22b	113462	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and TSU/RG Receiver PAIR	1	\$437.88
1a, 3-18, 19b, 20-21, 22b	113448	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and TSU/RG Receiver Right	1	\$218.94
1a, 3-18, 19b, 20-21, 22b	113451	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and TSU/RG Receiver Left	1	\$218.94
1a, 3-15, 19b, 20, 23b	503183	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad Right	1	\$169.97
1a, 3-15, 19b, 20, 23b	503184	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad Left	1	\$169.97
1b, 3-18, 19b, 20-21, 22a	113463	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and TSU/RG Receiver PAIR	1	\$437.88
1b, 3-18, 19b, 20-21, 22a	113449	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and TSU/RG Receiver Right	1	\$218.94
1b, 3-18, 19b, 20-21, 22a	113450	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and TSU/RG Receiver Left	1	\$218.94
1b, 3-15, 19b, 20, 22a	503185	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad Right	1	\$169.97
1b, 3-15, 19b, 20, 22a	503186	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad Left	1	\$169.97
1a, 3-15, 20	002681	Arm T Height Adj Low, Desk Right Assembly	1	\$134.01
1a, 3-15, 20	002682	Arm T Height Adj Low, Desk Left Assembly	1	\$134.01
1b, 3-15, 20	002683	Arm T Height Adj Low, Full Right Assembly	1	\$134.01
1b, 3-15, 20	002684	Arm T Height Adj Low, Full Left Assembly	1	\$134.01
16-18, 21	501731	Height Adj T-Arm Receiver Assembly 1-3/8" TSU/RG	1	\$48.23
8-12	500541	Arm Height Adj T Upper Lever Assembly	1	\$40.56

Item Number	Part Number	Description	Quantity	Retail
1a, 20	002648	Post Weldment, T-Arm, Low, Desk	1	\$67.08
1b, 20	002649	Post Weldment, T-Arm, Low, Full	1	\$69.35
2a	100500	Arm Pad Full	1	\$23.64
2b	100501	Arm Pad Desk	1	\$23.64
3	000090	Height Adj T Armrest Shim	2	\$2.55
4	002105	Tube, Outer, T-Arm, Low	1	\$29.62
5	000092	Height Adj T Armrest Clamp	2	\$8.97
6	002107	Sideguard, T-Arm, Low	1	\$8.10
7	002104	Transfer Tube, T-Arm, Low	1	\$16.81
8	000098	Height Adj Armrest Lever	1	\$3.58
9	200257	Dowel Pin 1/8 x 1/2	1	\$2.63
10	000134	Height Adj Armrest Lever Spring	1	\$1.11
11	000099	Height Adj Armrest Lever Pin	1	\$19.13
12	000096	Height Adj T Armrest Pin Lock Housing	1	\$14.71
13	002106	Lever, T-Arm, Low	1	\$3.47
14	000135	Height Adj T Armrest Release Lever Spring	1	\$1.09
15	000152	Height Adj T Armrest Release Spring Button	1	\$1.09
16	004578	Height Adjustable T-Arm Receiver 1.375"	1	\$15.25
17	000088	Height Adj T Armrest Bracket	1	\$27.42
18	101130	M6 x 55 FSHCS P/THD BLKZC	2	\$2.10
19a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2	\$1.58
19b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2	\$1.58
20	100803	Tube End Plug 7/8"	2	\$1.58
21	100658	M6 Nylock Nut Blk Zn	2	\$1.05
22a	102452	Waterfall Arm Pad Full	1	\$32.80
22b	102521	Waterfall Arm Pad Desk	1	\$32.80

ROGUE/ ROGUE XP		REAR SEAT HEIGHT																	
		11.5	12	12.5	13	13.5	14	14.5	15	15.5	16	16.5	17	17.5	18	18.5	19	19.5	20
WHEEL DIAMETER	18	X	X	X	X	X	X	X	X	X	X	X	X	X					
	20				X	X	X	X	X	X	X	X	X	X	X	X	X	X	
	22								X	X	X	X	X	X	X	X	X	X	X
	24												X	X	X	X	X	X	X
	25														X	X	X	X	X
	26																X	X	X

X = Low T-Arms are Available

Low T-Arm availability may change depending on chair configuration

See Order Form for Additional No-Go's