


Rogue ALX Aluminum Caster Mount (Formerly Tsunami)

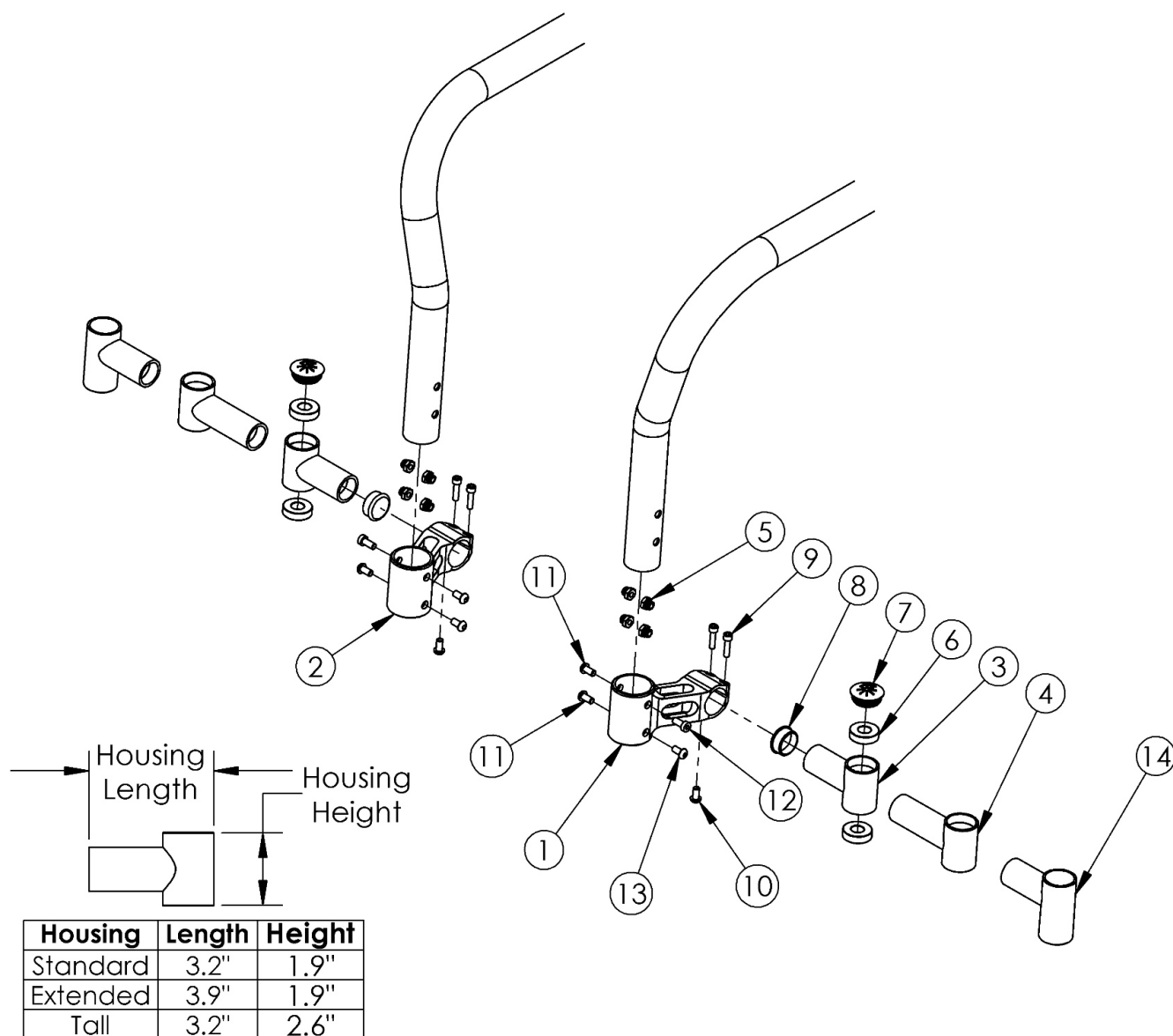

If quoting replacement frame, caster mounts are included with frame.

Item Number	Part Number	Description	Quantity
1, 5, 9, 11a, 11b, 12a, 12b, 13	502648	Caster Mount Assembly With Hardware Left, RG ALX/LW XP	1
2, 5, 9, 11a, 11b, 12a, 12b, 13	502649	Caster Mount Assembly With Hardware Right, RG ALX/LW XP	1
1, 3, 5 - 13	501178	Caster Mount Assembly With Standard Housing Left, RG ALX/LW XP	1
2, 3, 5 - 13	501179	Caster Mount Assembly With Standard Housing Right, RG ALX/LW XP	1

Item Number	Part Number	Description	Quantity
1, 2, 3, 5 - 13	501180	Caster Mount Assembly With Standard Housing Pair, RG ALX/LW XP	1
1, 4 - 13	501181	Caster Mount Assembly With Ext Housing Left, RG ALX/LW XP	1
2, 4 - 13	501182	Caster Mount Assembly With Ext Housing Right, RG ALX/LW XP	1
1, 2, 4 - 13	501183	Caster Mount Assembly With Ext Housing Pair, RG ALX/LW XP	1
3, 6 - 10	502650	Caster Housing Assembly With Hardware Standard Left, RG ALX/LW XP	1
3, 6 - 10	505239	Caster Housing Assembly With Hardware Standard Right, RG ALX/LW XP	1
4, 6 - 10	502651	Caster Housing Assembly With Hardware Extended Left, RG ALX/LW XP	1
4, 6 - 10	505240	Caster Housing Assembly With Hardware Extended Right, RG ALX/LW XP	1
1	000390L	Caster Mount AL Left, RG ALX/LW XP	1
2	000390R	Caster Mount AL Right, RG ALX/LW XP	1
3, 6	000251	Caster Housing, RG ALX/LW XP	2
4, 6	000458	Caster Housing Extended, RG ALX/LW XP	2
5	000491	Caster T-Nut, RG ALX/LW XP	8
6	100530	Bearing M28 x 1/2" 6001RS	4
7	102005	Ki Caster Housing Cap	2
8	100532	Tube End Plug 1 1/8"	2
9	101843	M5 x 20 SHCS Blk Zn	4
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
11a	101184	M6 x 10 BTN SHCS BLKZC w/Patch <i>Used when transit brackets not present</i>	4
11b	101708	M6 x 14 BTN SHCS BLKZC w/Patch <i>Used with transit brackets</i>	4
12a	101185	M6 x 14 CLS8.8 DIN7984 Low SHCS BLKZC <i>Used when footrest is not high-mounted</i>	2
12b	101184	M6 x 10 BTN SHCS BLKZC w/Patch <i>Used with high-mounted footrests</i>	2
13	101184	M6 x 10 BTN SHCS BLKZC w/Patch	2