


# Focus Footplates - Aluminum Locking Multi-Angle Adjustable Footplate Front Mount

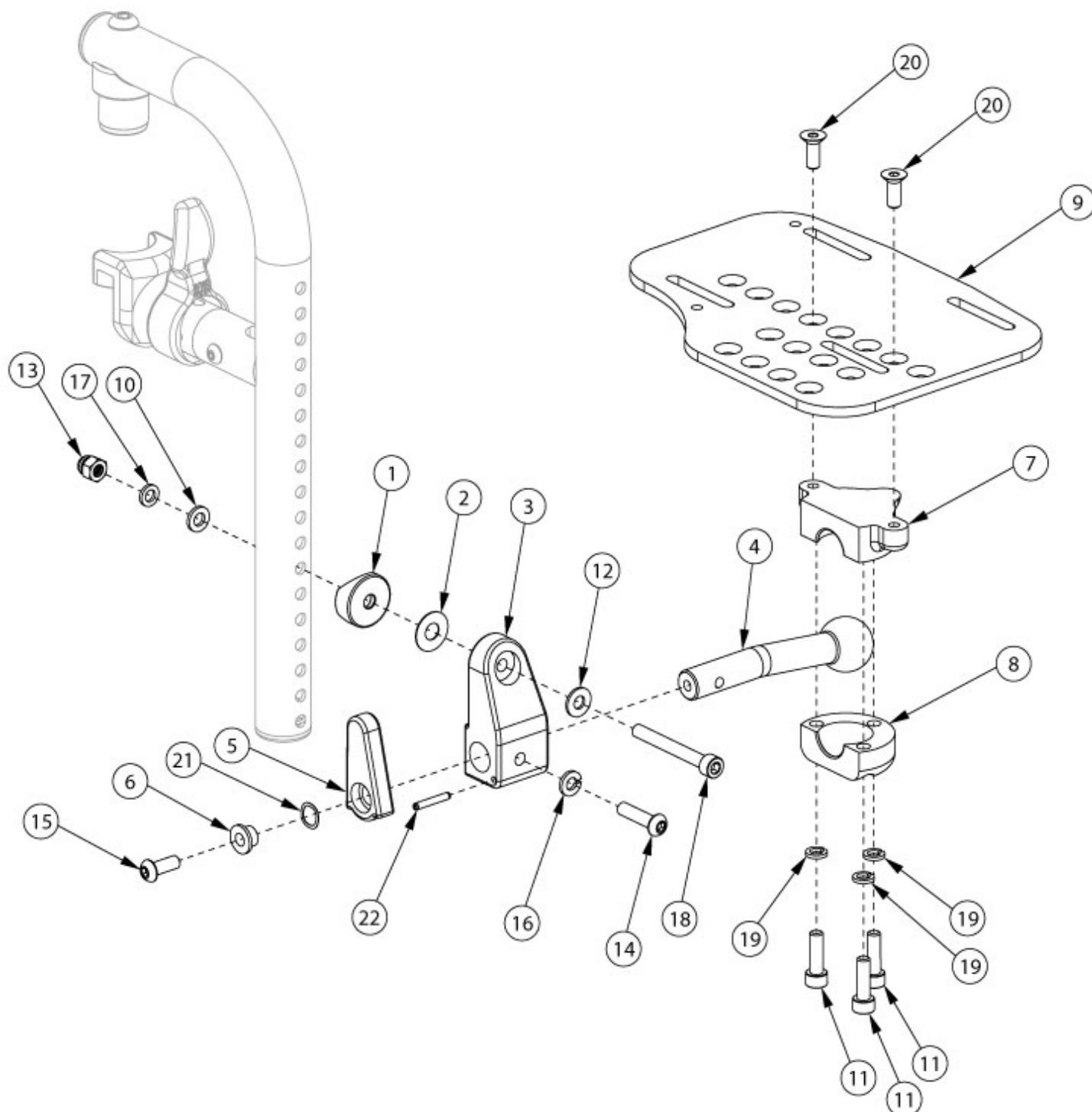

Sizes for footplates / assemblies are based on seat width with straight frame. If Focus chair has Offset Frame, add 2" to seat width.  
Only compatible with Front Mount Hangers.

| Item Number                    | Part Number | Description                                                               | Quantity |
|--------------------------------|-------------|---------------------------------------------------------------------------|----------|
| 1, 2, 3a, 4-8, 10-22           | 107618      | Footplate Hardware Kit Aluminum Locking Multi-Angle Adjustable, Right     | 1        |
| 1, 2, 3b, 4-8, 10-22           | 107619      | Footplate Hardware Kit Aluminum Locking Multi-Angle Adjustable, Left      | 1        |
| 1, 2, 3a-3b, 4-8, 10-22        | 107620      | Footplate Hardware Kit Aluminum Locking Multi-Angle Adjustable, PAIR      | 1        |
| 1, 2, 3a, 4-8, 9a, 10-22       | 107605      | Footplate Kit Aluminum Locking Multi-Angle Adjustable, 14-15" Wide, Right | 1        |
| 1, 2, 3a, 4-8, 9c, 10-22       | 107606      | Footplate Kit Aluminum Locking Multi-Angle Adjustable, 16-17" Wide, Right | 1        |
| 1, 2, 3a, 4-8, 9e, 10-22       | 107607      | Footplate Kit Aluminum Locking Multi-Angle Adjustable, 18-24" Wide, Right | 1        |
| 1, 2, 3b, 4-8, 9b, 10-22       | 107610      | Footplate Kit Aluminum Locking Multi-Angle Adjustable, 14-15" Wide, Left  | 1        |
| 1, 2, 3b, 4-8, 9d, 10-22       | 107611      | Footplate Kit Aluminum Locking Multi-Angle Adjustable, 16-17" Wide, Left  | 1        |
| 1, 2, 3b, 4-8, 9f, 10-22       | 107612      | Footplate Kit Aluminum Locking Multi-Angle Adjustable, 18-24" Wide, Left  | 1        |
| 1, 2, 3a-3b, 4-8, 9a-9b, 10-22 | 107615      | Footplate Kit Aluminum Locking Multi-Angle Adjustable, 14-15" Wide, PAIR  | 1        |
| 1, 2, 3a-3b, 4-8, 9c-9d, 10-22 | 107616      | Footplate Kit Aluminum Locking Multi-Angle Adjustable, 16-17" Wide, PAIR  | 1        |
| 1, 2, 3a-3b, 4-8, 9e-9f, 10-22 | 107617      | Footplate Kit Aluminum Locking Multi-Angle Adjustable, 18-24" Wide, PAIR  | 1        |
| 1                              | 000055      | Saddle 1"                                                                 | 1        |
| 2                              | 000428      | Wear Washer                                                               | 1        |
| 3a, 22                         | 109294      | Multi-Angle Footrest Mount 1", with Spring Pin, Right                     | 1        |
| 3b, 22                         | 109295      | Multi-Angle Footrest Mount 1", with Spring Pin, Left                      | 1        |
| 4                              | 003820      | Ball Shaft 1" Multi-Angle                                                 | 1        |
| 5                              | 003821      | Latch, Footplate, Multi-Angle                                             | 1        |
| 6                              | 003822      | Bushing, Latch, Multi-Angle                                               | 1        |
| 7                              | 003823      | Clamp, Upper, Footplate, Multi-Angle                                      | 1        |
| 8                              | 003824      | Clamp, Lower, Footplate, Multi-Angle                                      | 1        |
| 9a                             | 003829      | Footplate, Multi-Angle, 14-15W, Right                                     | 1        |
| 9b                             | 003830      | Footplate, Multi-Angle, 14-15W, Left                                      | 1        |
| 9c                             | 003831      | Footplate, Multi-Angle, 16-17W, Right                                     | 1        |
| 9d                             | 003832      | Footplate, Multi-Angle, 16-17W, Left                                      | 1        |
| 9e                             | 003833      | Footplate, Multi-Angle, 18-24W, Right                                     | 1        |
| 9f                             | 003834      | Footplate, Multi-Angle, 18-24W, Left                                      | 1        |

| Item Number | Part Number | Description                                 | Quantity |
|-------------|-------------|---------------------------------------------|----------|
| 10          | 100746      | M6 Flat Washer Blk Zn                       | 1        |
| 11          | 101180      | M6 x 20 SHCS BLK ZC                         | 3        |
| 12          | 101455      | M6 Belleville Washer (.252 x .551 x .050)   | 1        |
| 13          | 101456      | M6 Ovl Top L/N ZC                           | 1        |
| 14          | 101593      | M6 x 25 BTN SHCS BLKZC                      | 1        |
| 15          | 101721      | M6 X 16 Btn Shcs W/ Patch                   | 1        |
| 16          | 101835      | M6 Lock Washer Blk Zn                       | 1        |
| 17          | 102072      | Washer Nylon .234 ID x .406 OD x .062 Thick | 1        |
| 18          | 102473      | M6 X 50 CLS12.9 SHCS BLK ZC                 | 1        |
| 19          | 102810      | M6 Lockwasher, Hi-Collar, ZC                | 3        |
| 20          | 102936      | M6 X 16 FHCS, Blk Zn                        | 2        |
| 21          | 103539      | Wave Spring, 3/8IDx1/2ODx.009T, SS          | 1        |