


# Discontinued Catalyst Loop Footplate

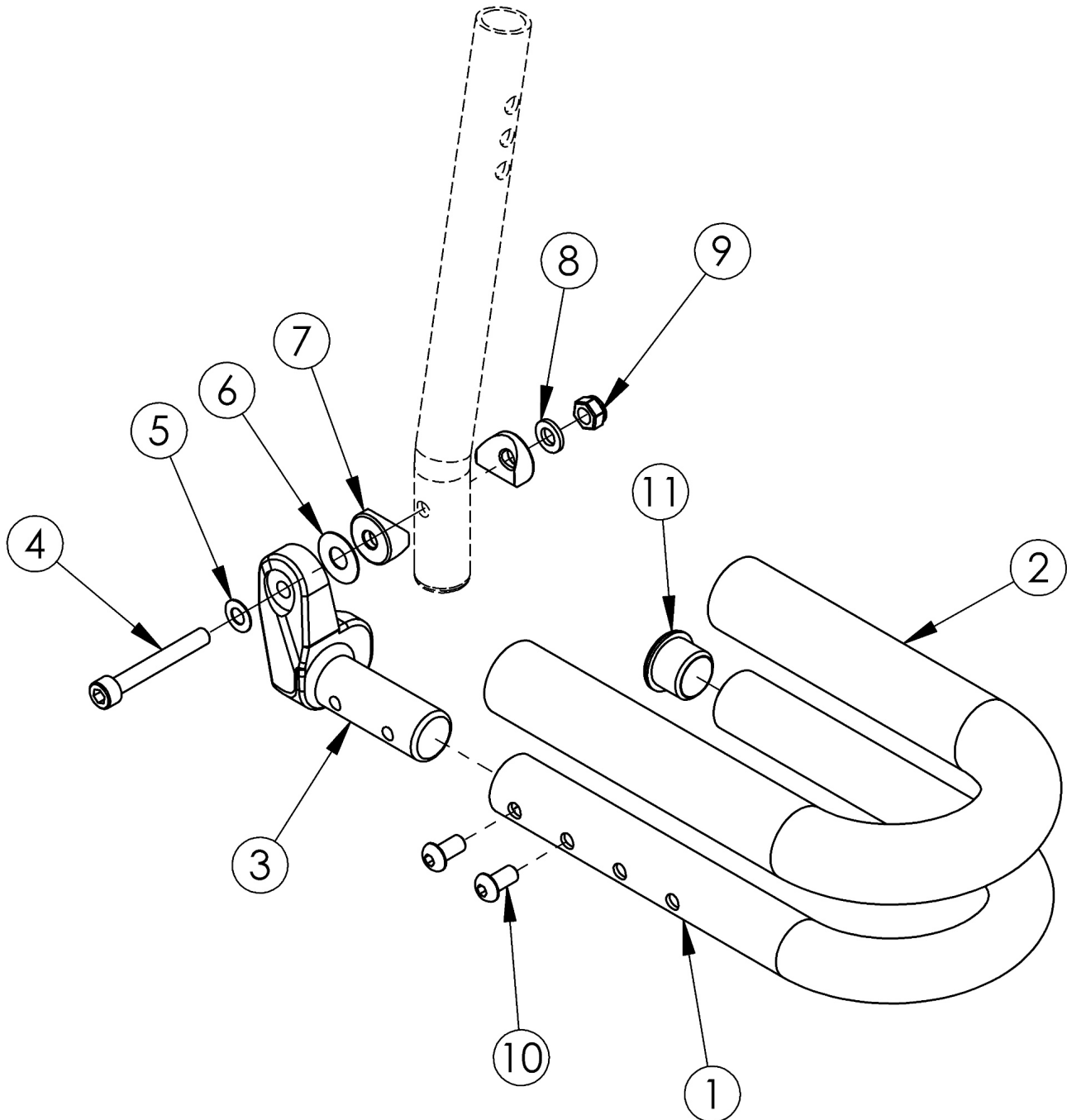

*This Catalyst Loop Footplate is discontinued as of November 30th, 2015*

Item Number	Part Number	Description	Quantity
1a, 2a, 3a, 4-11	500740	Footplate Loop Assembly 14/15 Right	1

Item Number	Part Number	Description	Quantity
1b, 2b, 3a, 4-11	500741	Footplate Loop Assembly 16/17 Right	1
1c, 2c, 3a, 4-11	500742	Footplate Loop Assembly 18-22 Right	1
1a, 2a, 11	100538	Footplate Loop w/ Grip 14/15	1
1b, 2b, 11	100541	Footplate Loop w/ Grip 16/17	1
1c, 2c, 11	100543	Footplate Loop w/ Grip 18-22	1
3a	000067	Footplate Loop Casting 3/4" Right	1
4	101472	M6 x 45 SHCS BLK ZC	1
5	101455	M6 Belleville Washer (.252 x .551 x .050)	1
6	000428	Wear Washer	1
7	000053	Saddle 3/4"	2
8	100746	M6 Flat Washer Blk Zn	1
9	120030	M6 Top Lock Nut	1
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
11	100803	Tube End Plug 7/8"	1