


# Catalyst 5 Footplates - One Piece Flip Up

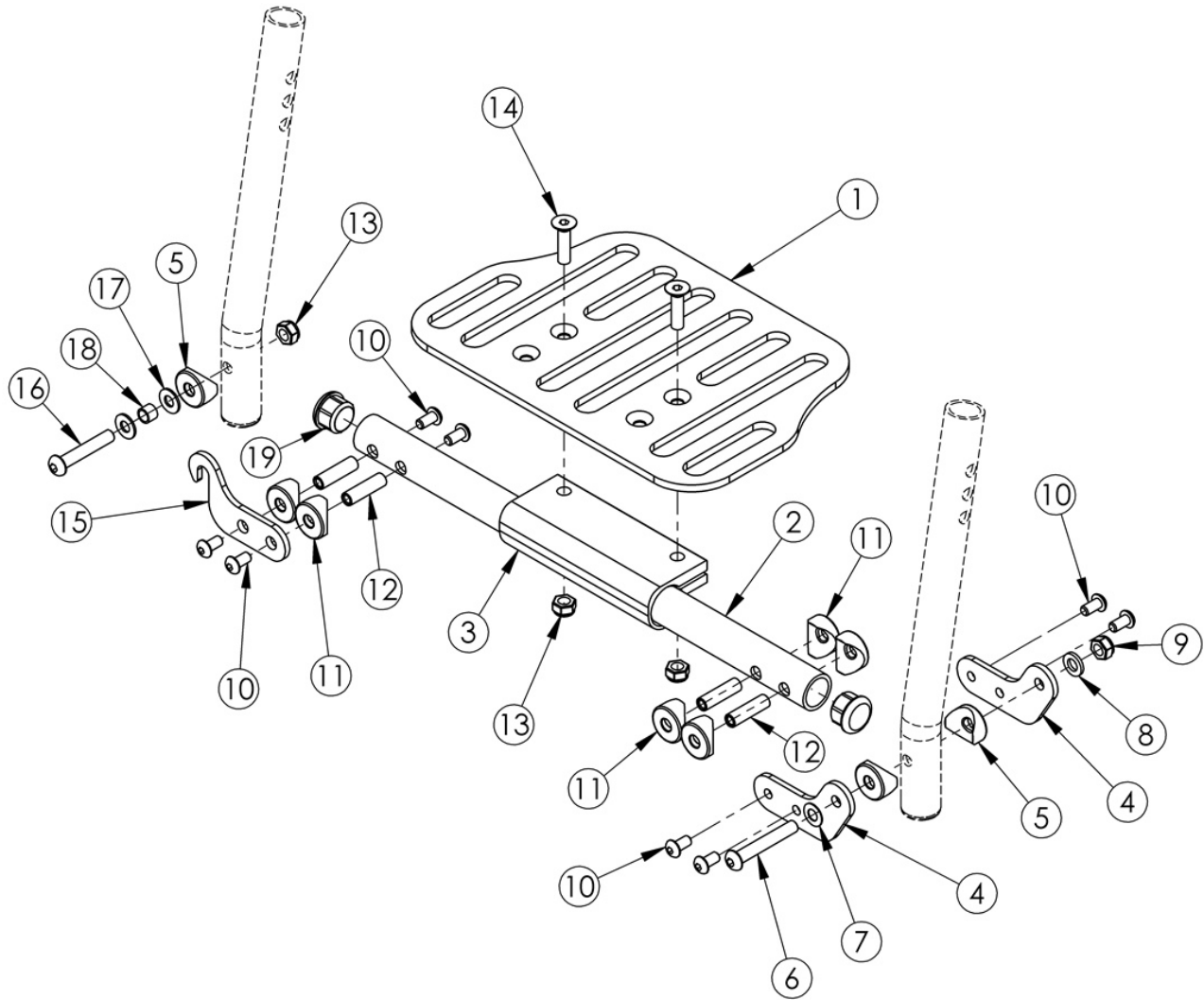

*Assemblies are based on seat width. Only compatible with extension tube hangers. If changing footplate style, new extension tubes may be needed.*

Item Number	Part Number	Description	Quantity
3-19	112894	Hardware Assembly for One Piece Flip Up Footplate	1
1a, 2a, 3-19	500517	Footrest Assembly One Piece Flip 14" Wide	1
1a, 2b, 3-19	500518	Footrest Assembly One Piece Flip 15" Wide	1
1a, 2c, 3-19	500519	Footrest Assembly One Piece Flip 16" Wide	1

Item Number	Part Number	Description	Quantity
1b, 2d, 3-19	500520	Footrest Assembly One Piece Flip 17" Wide	1
1b, 2e, 3-19	500521	Footrest Assembly One Piece Flip 18" Wide	1
1b, 2f, 3-19	500522	Footrest Assembly One Piece Flip 19" Wide	1
1b, 2g, 3-19	500523	Footrest Assembly One Piece Flip 20" Wide	1
1c, 2h, 3-19	502399	Footrest Assembly One Piece Flip 21" Wide	1
1c, 2i, 3-19	502400	Footrest Assembly One Piece Flip 22" Wide	1
1a	000226	Footrest Platform Angle Adjustable 8"	1
1b	000227	Footrest Platform Angle Adjustable 10"	1
1c	001319	Footrest Platform Angle Adjustable 12"	1
2a	000357	14" Footrest Tube, One Piece Flip Up	1
2b	000358	15" Footrest Tube, One Piece Flip Up	1
2c	000359	16" Footrest Tube, One Piece Flip Up	1
2d	000360	17" Footrest Tube, One Piece Flip Up	1
2e	000361	18" Footrest Tube, One Piece Flip Up	1
2f	000362	19" Footrest Tube, One Piece Flip Up	1
2g	000363	20" Footrest Tube, One Piece Flip Up	1
2h	002424	21" Footrest Tube, One Piece Flip Up	1
2i	002425	22" Footrest Tube, One Piece Flip Up	1
3	100789	Footrest Adjustable Clamp 5"	1
4	000355	Flip Up Footrest Pivot	2
5	000053	Saddle 3/4"	3
6	100797	M6 x 40 BHCS BLK ZC	1
7	101455	M6 Belleville Washer (.252 x .551 x .050)	1
8	100746	M6 Flat Washer Blk Zn	1
9	120030	M6 Top Lock Nut	1
10	100669	M5 x 10 BTN SHCS BLKZC w/Patch	8
11	000353	Saddle - 3/4" .266 ID	6
12	000073	M5 x 24 Threaded Barrel	4

Item Number	Part Number	Description	Quantity
13	100658	M6 Nylock Nut Blk Zn	3
14	100662	M6 x 25 FSHCS BLK ZC	2
15	000356	Flip Up Footrest Latch	1
16	101811	M6 x 35 BHCS Blk Zn	1
17	100831	1/2" Washer OD .040" Thick	2
18	000354	Flip Up Ferrule	1
19	100638	Tube End Plug 3/4"	2