



## REPLACEMENT FOR SWING-AWAY HANGER AND FOOTREST HEIGHT ADJUSTMENT



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty.

If you have any questions call Sunrise Medical Technical support at 800.333.4000

### Parts included in this Hanger Assembly Kit:

- 1x Hanger
- 2x 1/4-28 x 3/4 hex head cap screw black with patch
- 1x hanger saddle
- 1x D2 extension tube insert
- 1x Footrest Extension tube assembly

### Tools needed for installation

- 1x 7/16" open or socket wrench
- 1x Torque wrench

**NOTE-** Your new hanger/extension assembly replaces the old hanger/extension assembly that used set screws for securing the hanger and footplate. A left hand assembly is shown in (figure 1)

### I. To adjust the height of your new footrest extension I


- Using a 7/16" socket or open-end wrench, remove the two hex head cap screws (B)
- Slide the footrest extension (E) with D2 insert (D) up or down inside the hanger tube to the correct height, and align the holes.
- Install both of the hex head cap screws (B) thru the saddle (C), hanger (A), footrest extension, (E) and finally into the D2 Insert (D) and hand-thread both screws.
- Using a torque wrench snug both hex head screws, and then torque to 30 in-lbs

Reference: Swing Away 60°, 70°, 70°V hangers

For Dealer use:

Installed by: \_\_\_\_\_ Date: \_\_\_\_\_

(Please retain as a record of installation.)

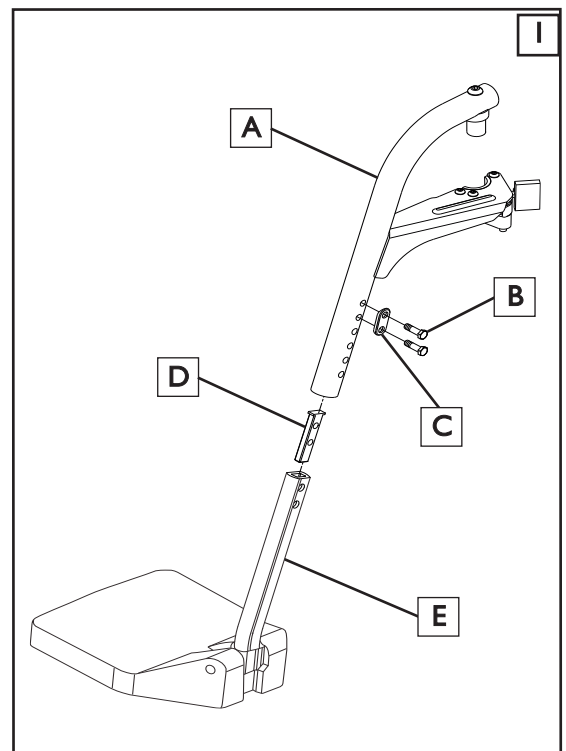

Sunrise Medical • 2842 Business Park Ave. • Fresno, CA 93727 • USA  
In Canada (800) 263-3390



Customer Service: 800.333.4000  
www.SunriseMedical.com

©2016 Sunrise Medical (US) LLC  
4.16 950112 Rev.C