


Axiom Swing Away Laterals for AXPL and AXLB

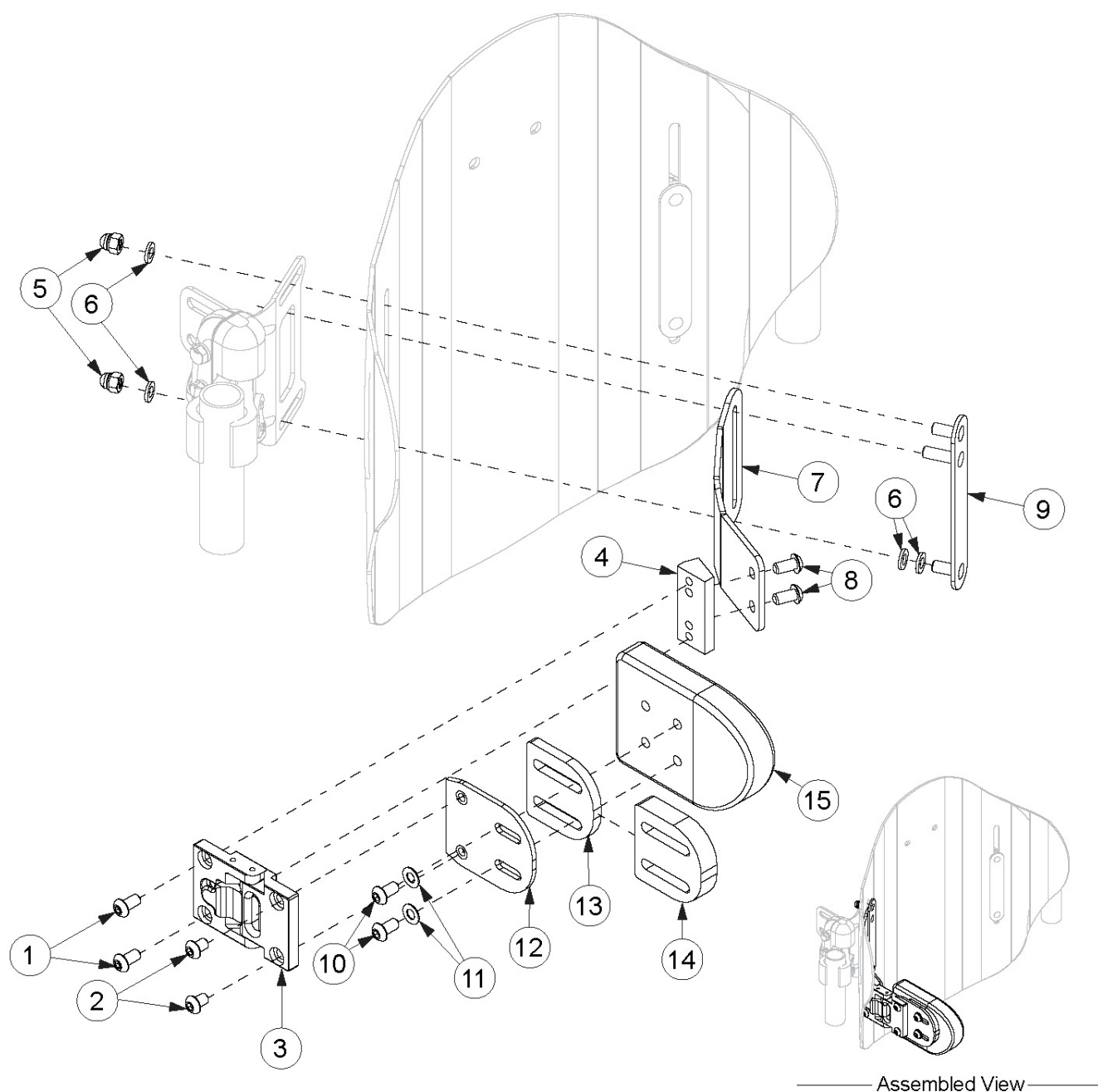

Item Number	Part Number	Description	Quantity	Retail
1	104017	1/4-20 X 1/2 Stainless Button Head Cap Screw	6	\$3.21
2	104031	1/4-20 X 3/8 Stainless Button Head Cap Screw	2	\$3.21
3	104463	Lateral, Swingaway Modular Quick Release Mount	1	\$168.68

Item Number	Part Number	Description	Quantity	Retail
4	107545	Axiom Back Spacer Block 20 Deg	1	\$85.68
5	100558	M6 NYL INS Domed Cap L/N ZC	3	\$1.10
6	100746	M6 Flat Washer Blk Zn	5	\$1.66
7a	004886	Axiom PL Back Lateral Mount, Swing Away, Right	1	\$37.49
7b	004887	Axiom PL Back Lateral Mount, Swing Away, Left	1	\$37.49
8	104017	1/4-20 X 1/2 Stainless Button Head Cap Screw	6	\$3.21
9	004889	Axiom Back Lateral Stud Plate	1	\$16.07
10a	104017	1/4-20 X 1/2 Stainless Button Head Cap Screw <i>Used when no spacers are used.</i>	6	\$3.21
10b	104028	1/4-20 X 3/4 Stainless Button Head Cap Screw <i>Used when the 1/4" spacer is used alone.</i>	2	\$5.36
10c	104014	BHCS, 1/4"-20 X 1.00, 18-8, BLK OX, ASME B18.3 <i>Used when the 1/2" spacer is used alone.</i>	2	\$5.36
10d	104024	1/4-20 X 1-1/4 Stainless Button Head Cap Screw <i>Used when the 1/4" and 1/2" spacers are used together.</i>	2	\$7.50
11	102803	Washer Flat .250 x .500 x .027 HX BLZ	2	\$1.66
12	004888	Axiom Back Lateral Adapter, Swing Away	1	\$26.78
13	107555	Axiom Back Lateral Spacer 1/4"	1	\$32.13
14	107556	Axiom Back Lateral Spacer 1/2"	1	\$32.13
13-14, 15a	108261	3x4 Lateral Pads (Pair)	1	\$192.78
13-14, 15b	108259	4x5 Lateral Pads (Pair)	1	\$192.78
13-14, 15c	108262	3x4 Lateral Pad (Each)	1	\$96.39
13-14, 15d	108260	4x5 Lateral Pad (Each)	1	\$96.39
1-6, 7a, 7b, 8-9, 10a-10d, 11-15a	LS3X4L	3 x 4 Lateral Pads with Swing Away Mounting Hardware for Axiom Posterior Lateral and Axiom Lateral Backs (Pair)	1	\$667.00
1-6, 7a, 7b, 8-9, 10a-10d, 11-15b	LS4X5L	4 x 5 Lateral Pads with Swing Away Mounting Hardware for Axiom Posterior Lateral and Axiom Lateral Backs (Pair)	1	\$667.00
1-6, 7b, 8-9, 10a-10d, 11-12	108257	Posterior Lateral - Swing Away Lateral Left <i>Pad not included.</i>	1	\$254.36
1-6, 7a, 8-9, 10a-10d, 11-12	108258	Posterior Lateral - Swing Away Lateral Right <i>Pad not included.</i>	1	\$254.36