


Catalyst Cane and Crutch Holder

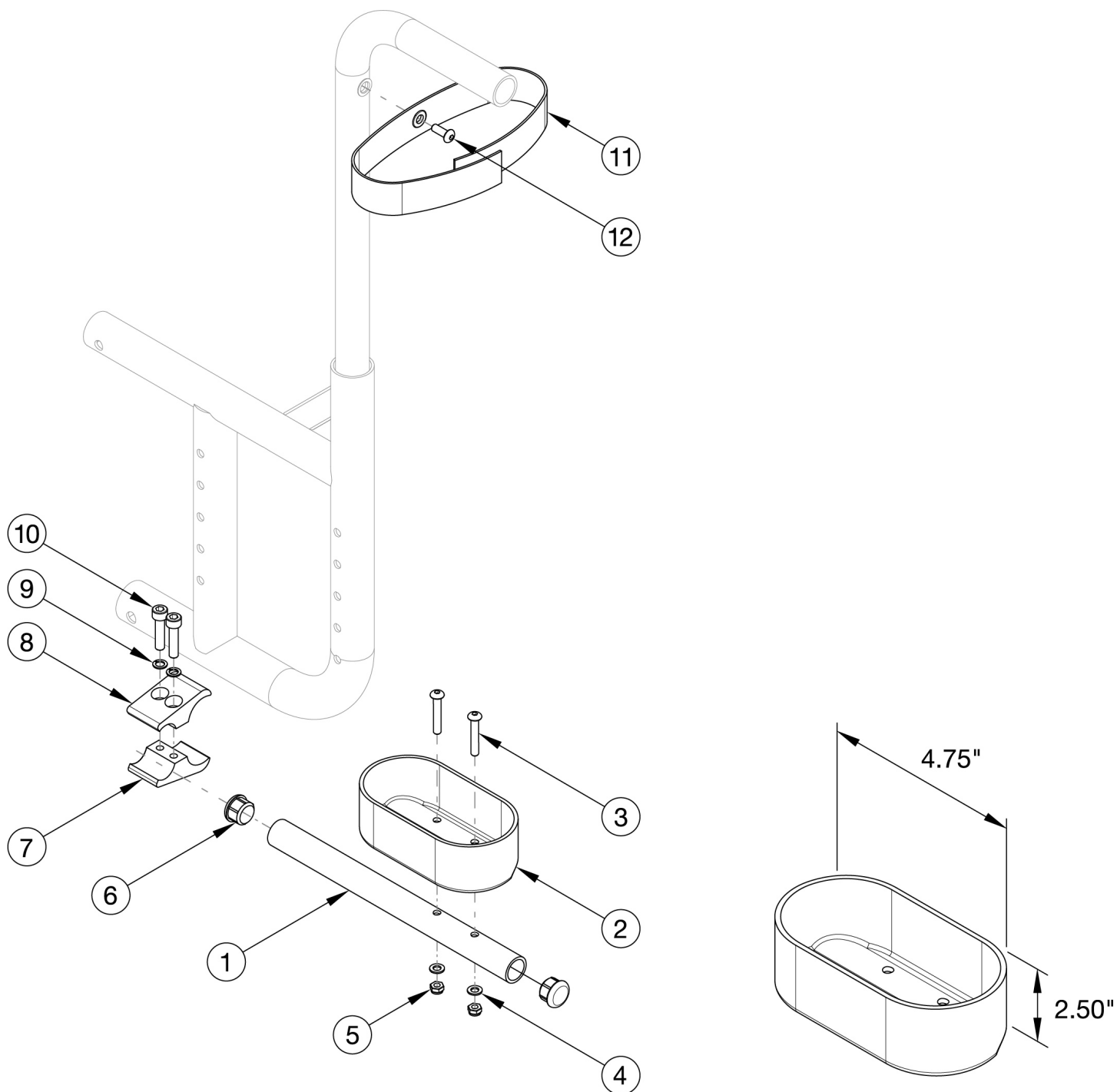

Item Number	Part Number	Description	Quantity
1 - 12	500710	Crutch Holder Assembly, Catalyst	1
1, 6	000919	Crutch Holder Tube CAT	1
2	000917	Crutch Holder Cup	1

Item Number	Part Number	Description	Quantity
3	101560	M5 x 30 BTN SHCS ZC	2
4	101856	Flat Washer, M5, Blk Zn	2
5	100657	M5 Nylock Nut Blk Zn	2
6	100638	Tube End Plug 3/4"	2
7	000379	Clamp Bottom 1.125"	1
8	000378	Clamp Top 1.125"	1
9	101835	M6 Lock Washer Blk Zn	2
10	100780	M6 x 25 SHCS ZC	2
11	101107	Crutch Holder Velcro Strap	1
12	100734	M5 x 16 PHMS ZC	1