

Fresno, California 93727-1328 (559) 292-2171 • (800) 456-8168

# INSTRUCTION SHEET ACTUATOR REPLACEMENT FOR QUICKIE S.P.O.T. TILT Kit Part Number: 102951

(Please read and understand instructions before beginning installation)

Tools required: • 3mm Hex Wrench • 5mm Hex Wrench • 13mm Wrench (2 required)	Before beginning installation, make sure that all the kit parts	DESCRIPTION S.P.O.T. TILT ACTUATOR REPLACEMENT KI	
	are included. At right is a table	ACTUATOR	1
	listing what is included in the	FRONT PIVOT BOLT	1
	Quickie S.P.O.T. Tilt Actuator	REAR PIVOT BOLT	1
	Replacement Kit.	NYLOCK NUT	2
		NYLON WASHER	2


### STEP 1

Remove seat pan by removing four bolts with 3mm hex wrench. (Fig. A)



### STEP 2

Remove the four bolts that attach the seat to the tilt (numbered 19, 20, 21, 22). Then, remove the seat from the tilt system. (Fig. B)

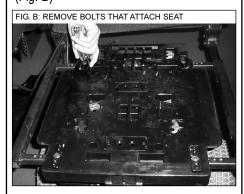

**STEP 3** Unplug molex connector from back of tilt and remove molex plug from mount. (Fig. C and Fig. D)






## STEP 4

On actuator, remove nylock nuts (with 13mm wrenches) from the pivot bolts, then remove pivot bolts. (Fig. E)

FIG. E: REMOVE NYLOCK NUTS AND PIVOT BOLTS

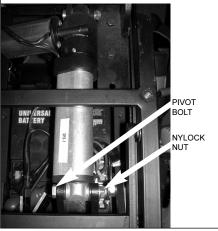

#### STEP 5

Remove and replace actuator with new actuator.

STEP 6 Reinstall pivot bolts and nuts. STEP 7 Reinstall plug into mount. (Fig. F)



STEP 8 Reinstall seat. STEP 9 Reinstall seat pan.

FOR DEALER USE Installed by:

\_ Serial #:

\_\_\_\_ (retain as record of installation)

SUNRISE MEDICAL

**JANUARY 2007** 

Date:

PAGE 1