


Discontinued Catalyst Loop Footplate

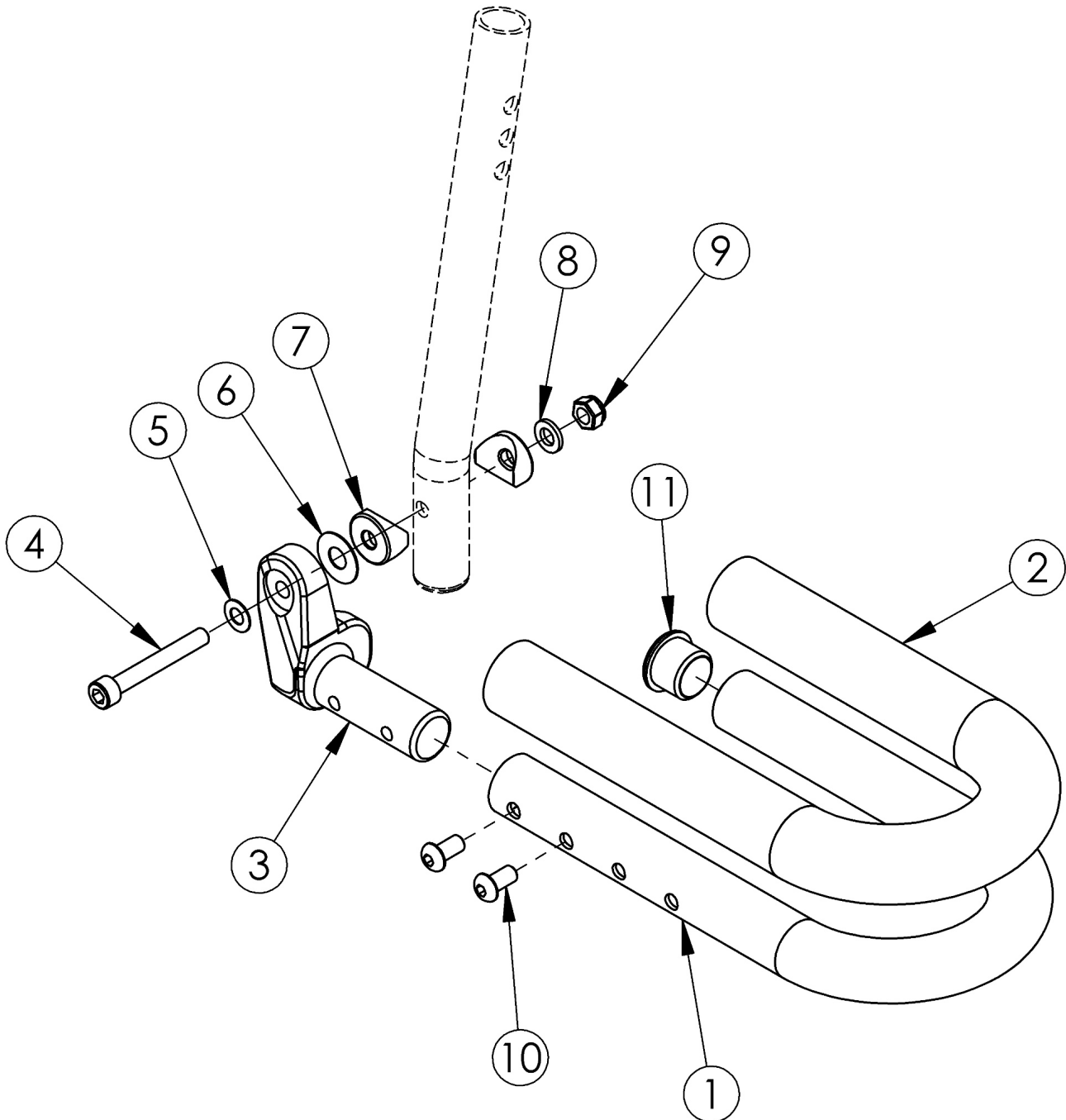

This Catalyst Loop Footplate is discontinued as of November 30th, 2015

Item Number	Part Number	Description	Quantity	Retail
1a, 2a, 3b, 4-11	501023	Footplate Loop Assembly 14/15 Left	1	\$90.76

Item Number	Part Number	Description	Quantity	Retail
1a, 2a, 3a, 4-11	500740	Footplate Loop Assembly 14/15 Right	1	\$90.76
1b, 2b, 3b, 4-11	501024	Footplate Loop Assembly 16/17 Left	1	\$90.76
1b, 2b, 3a, 4-11	500741	Footplate Loop Assembly 16/17 Right	1	\$90.76
1c, 2c, 3b, 4-11	501025	Footplate Loop Assembly 18-22 Left	1	\$90.76
1c, 2c, 3a, 4-11	500742	Footplate Loop Assembly 18-22 Right	1	\$90.76
1a, 2a, 11	100538	Footplate Loop w/ Grip 14/15	1	\$59.76
1b, 2b, 11	100541	Footplate Loop w/ Grip 16/17	1	\$59.76
1c, 2c, 11	100543	Footplate Loop w/ Grip 18-22	1	\$59.76
3a	000067	Footplate Loop Casting 3/4" Right	1	\$23.79
3b	000069	Footplate Loop Casting 3/4" Left	1	\$23.79
4	101472	M6 x 45 SHCS BLK ZC	1	\$1.58
5	101455	M6 Belleville Washer (.252 x .551 x .050)	1	\$1.58
6	000428	Wear Washer	1	\$2.63
7	000053	Saddle 3/4"	2	\$1.58
8	100746	M6 Flat Washer Blk Zn	1	\$1.58
9	101456	M6 Ovl Top L/N ZC	1	\$1.58
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58
11	100803	Tube End Plug 7/8"	1	\$1.58