


Tsunami Little Wave XP Rigid Frame With Growth

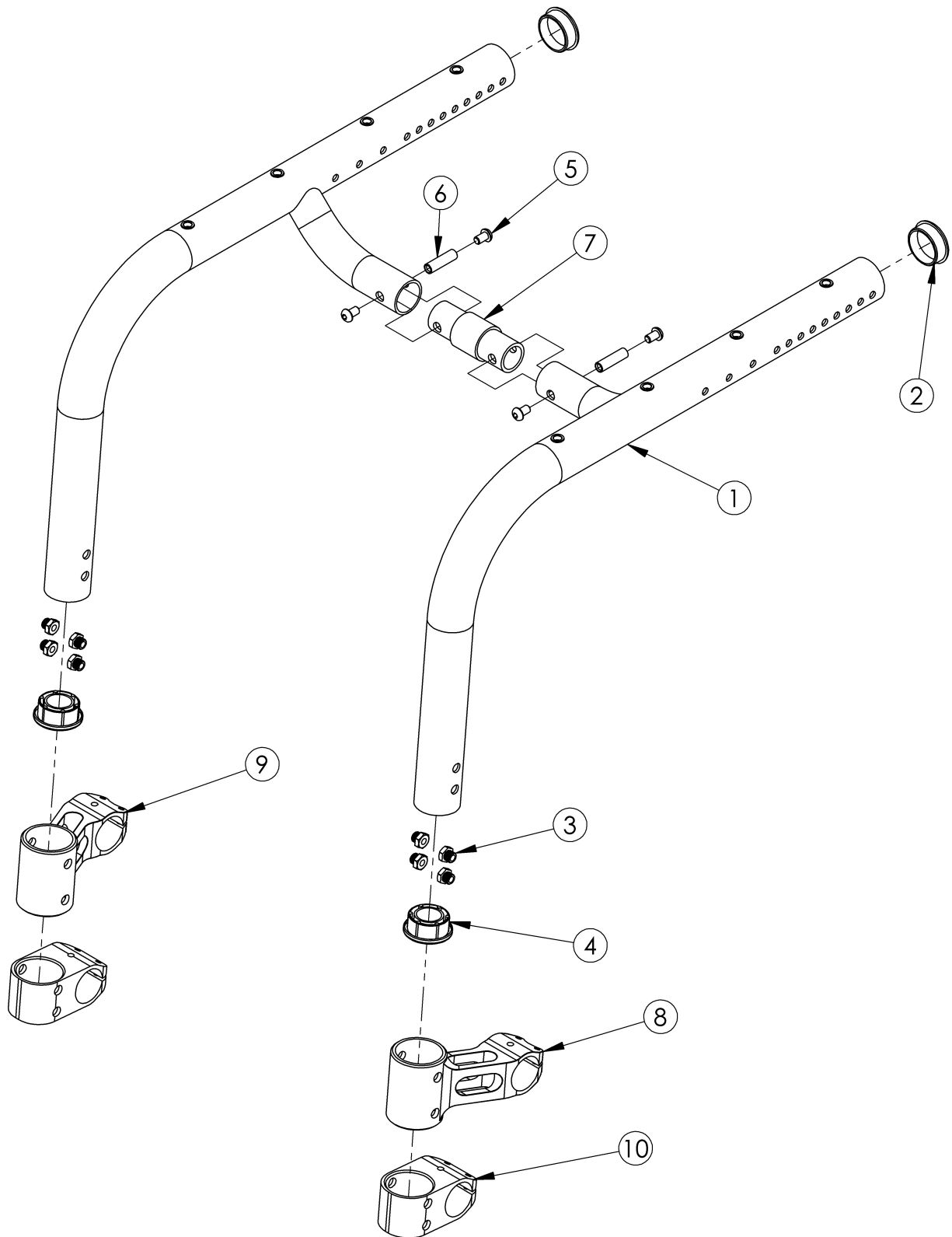

As of April 2016, the Tsunami Little Wave XP will only be offered in the XP grow-able frame <p>

Replacement rigid frames may be quoted, but must be ordered through Customer Service. Serial number required upon order <p>

Depending on configuration, new caster mounts may be required. If transit is present, new front transit brackets may be required.

Item Number	Part Number	Description	Quantity	Retail
1a, 2 - 6, 7, 8 - 10	RFTSXP	Replacement Tsunami XP Frame <i>Items 2 and 4 are included dependent upon configuration. Item 7 dependent on initial seat width. 7a is included on seat widths 10" - 16", 7b is included on 17" seat width, 7c is included on 18" seat width and 7d is included on 19" seat width. Either items 8 and 9 will be included or item 10, quantity 2, dependent upon configuration.</i>	1	\$1,042.44
1b, 2 - 6, 7, 8 - 10	RFTSXPGIT	Replacement Tsunami XP Frame w/ Graphic Image Transfer <i>Items 2 and 4 are included dependent upon configuration. Item 7 dependent on initial seat width. 7a is included on seat widths 10" - 16", 7b is included on 17" seat width, 7c is included on 18" seat width and 7d is included on 19" seat width. Either items 8 and 9 will be included or item 10, quantity 2, dependent upon configuration.</i>	1	\$1,649.34
5, 6, 7b	501331	Connector Assembly Frame 1" w/ Hardware	1	\$48.57
5, 6, 7c	501332	Connector Assembly Frame 2" w/ Hardware	1	\$48.57
5, 6, 7d	501333	Connector Assembly Frame 3" w/ Hardware	1	\$48.57
2	101113	Tube End Plug 1 3/8"	4	\$3.15
3	000491	Caster T-Nut TSU AL	8	\$6.38
4	000490	Sleeve Footrest Plastic Rigid	2	\$4.42
5	100667	M6 x 12 BTN SHCS BLKZC w/Patch	4	\$1.58
6	000980	M6 x 27.69 Threaded Barrel	2	\$5.36
7a	000585	Frame Connector 0" LW/RG	1	\$33.05
7b	000586	Frame Connector 1" LW/RG	1	\$33.05
7c	000587	Frame Connector 2" LW/RG	1	\$33.05
7d	000588	Frame Connector 3" LW/RG	1	\$33.05