


Catalyst 5Vx Footplates - One Piece Flip Up

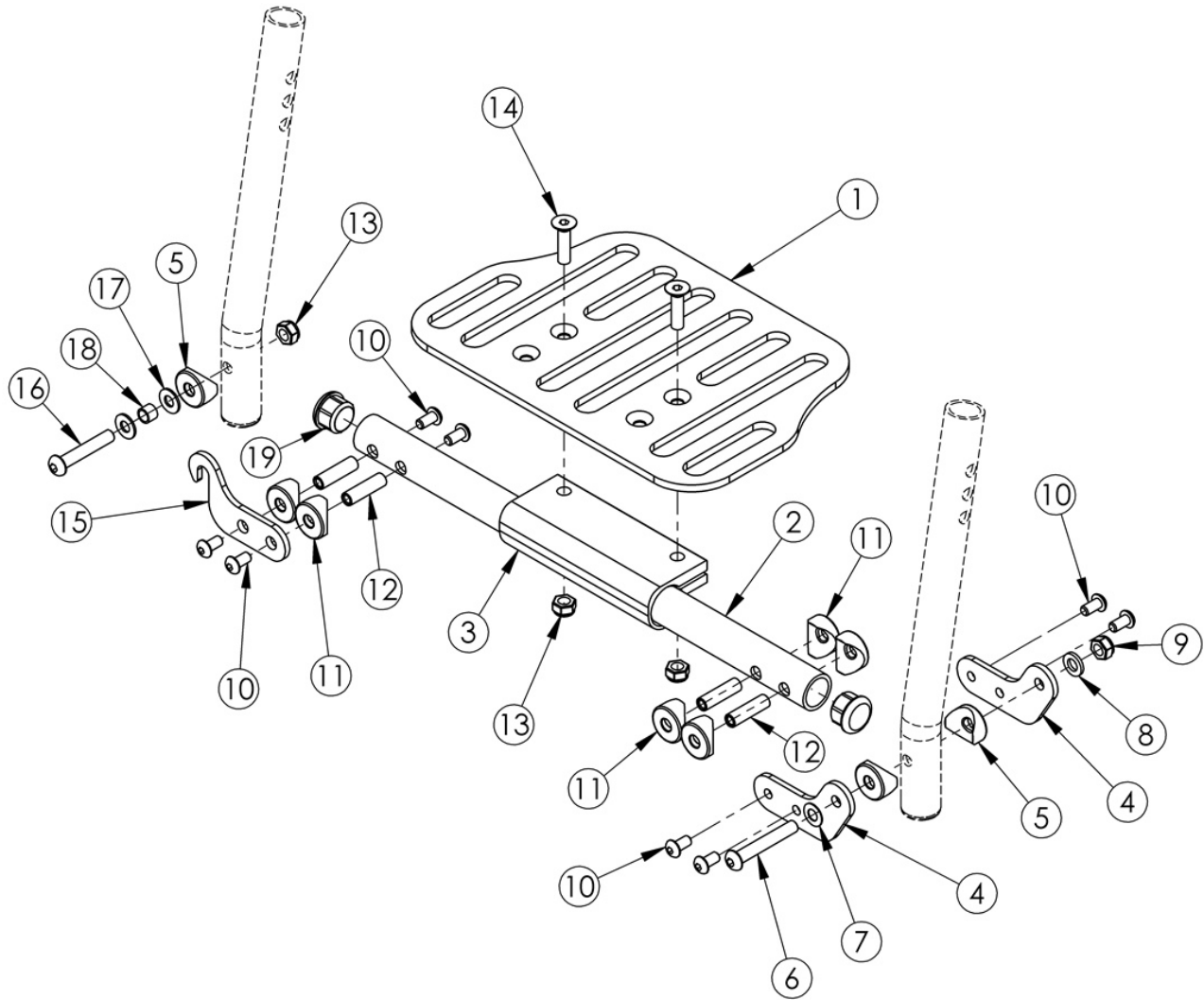

Assemblies are based on seat width. <p>
Only compatible with extension tube hangers.

Item Number	Part Number	Description	Quantity	Retail
3-19	112894	Hardware Assembly for One Piece Flip Up Footplate	1	\$142.47
1a, 2a, 3-19	500517	Footrest Assembly One Piece Flip 14" Wide	1	\$180.50
1a, 2b, 3-19	500518	Footrest Assembly One Piece Flip 15" Wide	1	\$180.50
1a, 2c, 3-19	500519	Footrest Assembly One Piece Flip 16" Wide	1	\$180.50

Item Number	Part Number	Description	Quantity	Retail
1b, 2d, 3-19	500520	Footrest Assembly One Piece Flip 17" Wide	1	\$180.50
1b, 2e, 3-19	500521	Footrest Assembly One Piece Flip 18" Wide	1	\$180.50
1b, 2f, 3-19	500522	Footrest Assembly One Piece Flip 19" Wide	1	\$192.70
1b, 2g, 3-19	500523	Footrest Assembly One Piece Flip 20" Wide	1	\$192.70
1c, 2h, 3-19	502399	Footrest Assembly One Piece Flip 21" Wide	1	\$192.70
1c, 2i, 3-19	502400	Footrest Assembly One Piece Flip 22" Wide	1	\$192.70
1a	000226	Footrest Platform Angle Adjustable 8"	1	\$85.76
1b	000227	Footrest Platform Angle Adjustable 10"	1	\$92.91
1c	001319	Footrest Platform Angle Adjustable 12"	1	\$66.30
2a	000357	14" Footrest Tube, One Piece Flip Up	1	\$35.37
2b	000358	15" Footrest Tube, One Piece Flip Up	1	\$35.37
2c	000359	16" Footrest Tube, One Piece Flip Up	1	\$35.37
2d	000360	17" Footrest Tube, One Piece Flip Up	1	\$35.37
2e	000361	18" Footrest Tube, One Piece Flip Up	1	\$35.37
2f	000362	19" Footrest Tube, One Piece Flip Up	1	\$35.37
2g	000363	20" Footrest Tube, One Piece Flip Up	1	\$35.37
2h	002424	21" Footrest Tube, One Piece Flip Up	1	\$35.37
2i	002425	22" Footrest Tube, One Piece Flip Up	1	\$35.37
3	100789	Footrest Adjustable Clamp 5"	1	\$29.23
4	000355	Flip Up Footrest Pivot	2	\$10.72
5	000053	Saddle 3/4"	3	\$1.58
6	100797	M6 x 40 BHCS BLK ZC	1	\$1.05
7	101455	M6 Belleville Washer (.252 x .551 x .050)	1	\$1.58
8	100746	M6 Flat Washer Blk Zn	1	\$1.58
9	101456	M6 Ovl Top L/N ZC	1	\$1.58
10	100669	M5 x 10 BTN SHCS BLKZC w/Patch	8	\$1.58
11	000353	Saddle - 3/4" .266 ID	6	\$1.20
12	000073	M5 x 24 Threaded Barrel	4	\$3.78

Item Number	Part Number	Description	Quantity	Retail
13	100658	M6 Nylock Nut Blk Zn	3	\$1.05
14	100662	M6 x 25 FSHCS BLK ZC	2	\$1.58
15	000356	Flip Up Footrest Latch	1	\$17.31
16	101811	M6 x 35 BHCS Blk Zn	1	\$1.05
17	100831	1/2" Washer OD .040" Thick	2	\$1.58
18	000354	Flip Up Ferrule	1	\$17.06
19	100638	Tube End Plug 3/4"	2	\$2.10