


# Catalyst Backrest - 8 Degree Bend with Rigidizer Bar

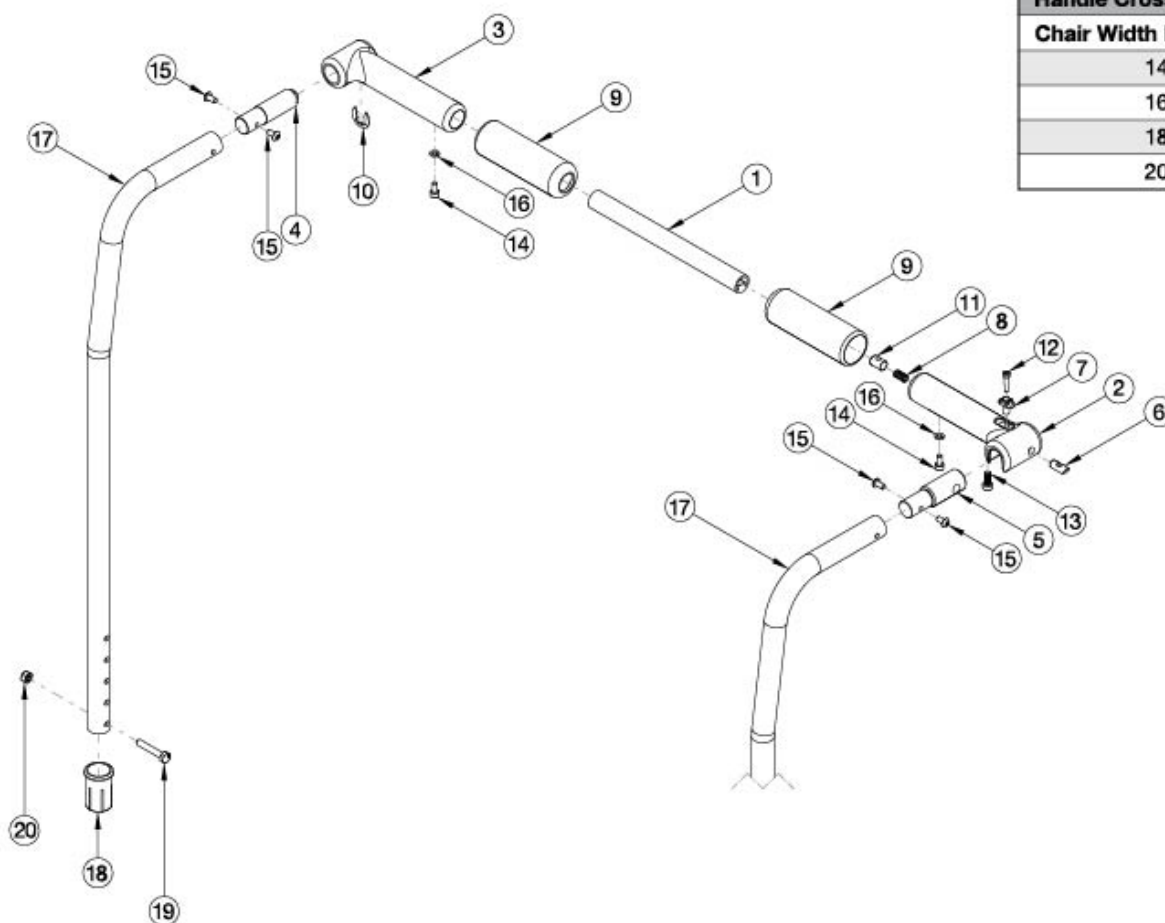

Handle Crosstube (Item 1)	
Chair Width Range	Part No.
14" - 16"	110264
16" - 18"	110265
18" - 20"	110266
20" - 22"	110267

Item Number	Part Number	Description	Quantity
1a, 2-16, 17a	116392	8 Degree Backcane, Folding Handle, Medium, 14"-16"W, CAT	1
1b, 2-16, 17a	116393	8 Degree Backcane, Folding Handle, Medium, 16"-18"W, CAT	1
1c, 2-16, 17a	116394	8 Degree Backcane, Folding Handle, Medium, 18"-20"W, CAT	1
1d, 2-16, 17a	116395	8 Degree Backcane, Folding Handle, Medium, 20"-22"W, CAT	1
1a, 2-16, 17b	116396	8 Degree Backcane, Folding Handle, Tall, 14"-16"W, CAT	1
1b, 2-16, 17b	116397	8 Degree Backcane, Folding Handle, Tall, 16"-18"W, CAT	1
1c, 2-16, 17b	116398	8 Degree Backcane, Folding Handle, Tall, 18"-20"W, CAT	1
1d, 2-16, 17b	116399	8 Degree Backcane, Folding Handle, Tall, 20"-22"W, CAT	1
1a, 2-16	116416	Folding Handle, 14"-16"W, CAT	1

Item Number	Part Number	Description	Quantity
1b, 2-16	116417	Folding Handle, 16"-18"W, CAT	1
1c, 2-16	116418	Folding Handle, 18"-20"W, CAT	1
1d, 2-16	116419	Folding Handle, 20"-22"W, CAT	1
1a	110264	Cross Tube, Rigidizer Handle, 14-16 Wide	1
1b	110265	Cross Tube, Rigidizer Handle, 16-18 Wide	1
1c	110266	Cross Tube, Rigidizer Handle, 18-20 Wide	1
1d	110267	Cross Tube, Rigidizer Handle, 20-22 Wide <i>Replaces part number 116463</i>	1
2	110261	Latch, Rigidizer Handle	1
3	110260	Pivot, Rigidizer Handle	1
4	110258	Adapter, Pivot, Rigidizer Handle	1
5	110259	Adapter, Latch, Rigidizer Handle	1
6	110263	Pin, Latch, Rigidizer Handle	1
7	110262	Button, Latch, Rigidizer Handle	1
10	110291	E-RING, M13-M18, DIN6799, ZC	1
11	102768	M6 Barrel Nut	1
12	110290	SHCS, M4 x 0.7 x 18, CLS12.9, BLZ, NYL, DIN912	1
13	101185	M6 x 14 CLS8.8 DIN7984 Low SHCS BLKZC	1
14	101766	M5 x 10 CLS12.9 DIN912 SHCS PLN	1
15	100669	M5 x 10 BTN SHCS BLKZC w/Patch	1
16	102614	M5 Split Lock Washer BLK ZC	1
17a	116071	Backtube with 8 Degree Bend Push Handle, Rigidizer, Medium	1
17b	116072	Backtube with 8 Degree Bend, Rigidizer, Tall	1
18	111965	Frame Sleeve, Angle Adjustable Backrest	1
19	100860	M6 x 35 HHCS P/THD ZC	1
20	100658	M6 Nylock Nut Blk Zn	1