


# Discontinued 2 Rogue XP Tension Adjustable Back Upholstery 17" - 18" Wide

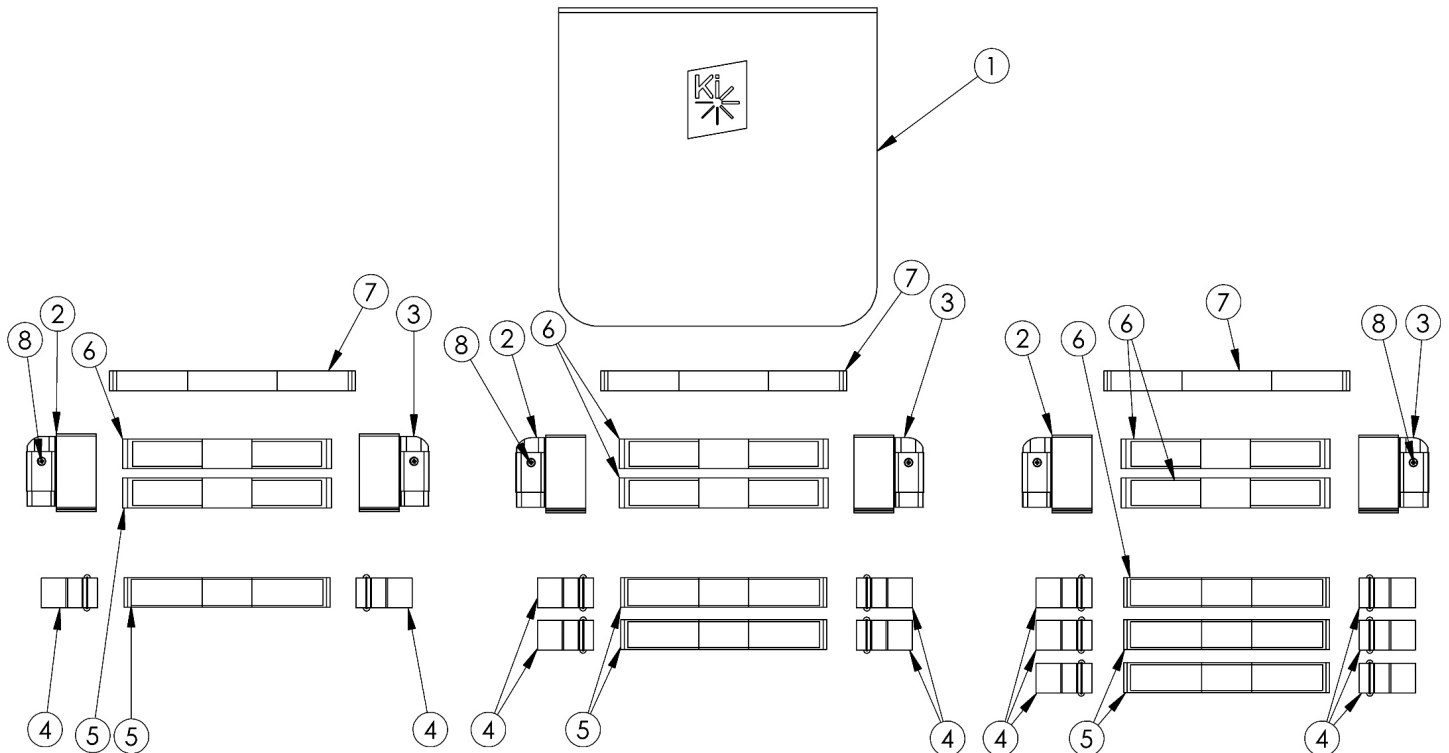

Quantity Chart			
Back Height	Item 6	Item 5	Item 4
10"-13.5"	1	2	2

Quantity Chart			
Back Height	Item 6	Item 5	Item 4
14"-16.5"	2	2	4

Quantity Chart			
Back Height	Item 6	Item 5	Item 4
17"-20"	3	2	6

Open Back Panels are used when integrated push handles are present. Closed Back Panels are when backposts without push handles are present. Tension Straps (7a-7b) are used with the combination of Fold Down Push Handle and Tension Adjustable Back Upholstery. Tension Straps 6a, 6b, 7a and 7b are based on chair width.

Item Number	Part Number	Description	Quantity
1a, 5a	504754	Backrest Cover with Two Straps 17W Rigid, 9-12H <i>Contains 003048 cover</i>	1
1b, 5a	504205	Backrest Cover with Two Straps 17W Rigid, 11-14H <i>Contains 003049 cover</i>	1
1c, 5a	504206	Backrest Cover with Two Straps 17W Rigid, 13-16H <i>Contains 003050 cover</i>	1
1d, 5a	504207	Backrest Cover with Two Straps 17W Rigid, 15-18H <i>Contains 003051 cover</i>	1
1e, 5a	504208	Backrest Cover with Two Straps 17W Rigid, 17-20H <i>Contains 003052 cover</i>	1

Item Number	Part Number	Description	Quantity
1f, 5b	504755	Backrest Cover with Two Straps 18W Rigid, 9-12H <i>Contains 003054 cover</i>	1
1g, 5b	504209	Backrest Cover with Two Straps 18W Rigid, 11-14H <i>Contains 003055 cover</i>	1
1h, 5b	504210	Backrest Cover with Two Straps 18W Rigid, 13-16H <i>Contains 003056 cover</i>	1
1i, 5b	504211	Backrest Cover with Two Straps 18W Rigid, 15-18H <i>Contains 003057 cover</i>	1
1j, 5b	504212	Backrest Cover with Two Straps 18W Rigid, 17-20H <i>Contains 003058 cover</i>	1
4	100973	Back Upholstery D-Ring	4
5a	002534	Back Upholstery Strap 21.3" <i>Used on 17" chair widths</i>	2
5b	002535	Back Upholstery Strap 23.3" <i>Used on 18" chair widths</i>	2
7a	001751	Tension Strap Folding Handle 16-17 Wide	1
7b	001752	Tension Strap Folding Handle 18-19 Wide	1
8	100734	M5 x 16 PHMS ZC	2