


Focus CR Seat Pan - Growth

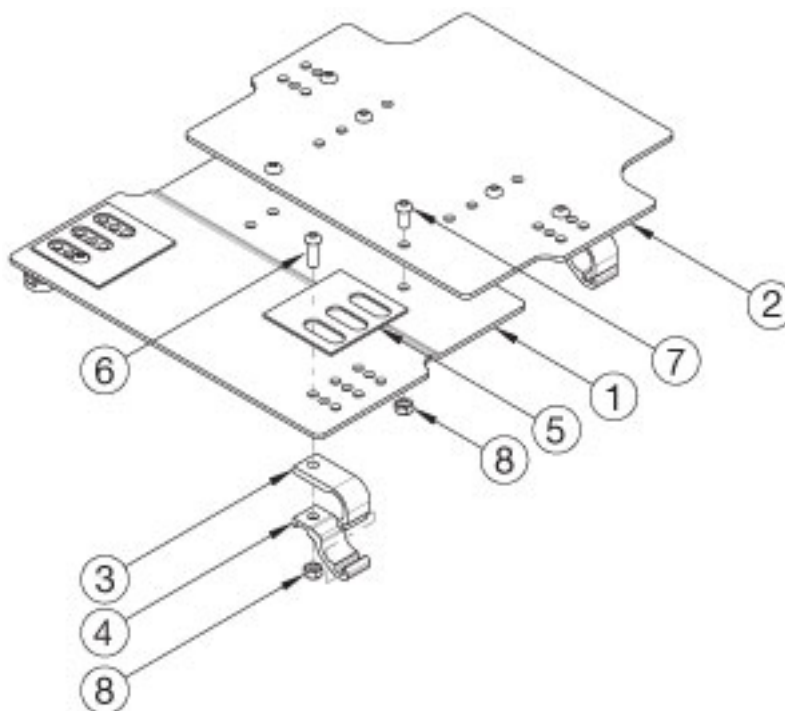

Does the growth require a new seat pan / assembly?

Item Number	Part Number	Description	Quantity
1a, 2a, 3 - 8	501144	14-16" Wide, Short, Seat Pan Assembly, Tilt	1
1b, 2b, 3 - 8	501145	16-18" Wide, Short, Seat Pan Assembly Tilt	1
1c, 2c, 3 - 8	501146	18-20" Wide, Short, Seat Pan Assembly Tilt	1
1d, 2d, 3 - 8	501147	20-22" Wide, Short, Seat Pan Assembly Tilt	1
1e, 2e, 3 - 8	501218	Seat Pan Assembly Short 22" Wide TILT	1
1f, 2a, 3 - 8	501148	Seat Pan Assembly Medium 14-16" Wide TILT	1
1g, 2b, 3 - 8	501149	Seat Pan Assembly Medium 16-18" Wide TILT	1
1h, 2c, 3 - 8	501150	Seat Pan Assembly Medium 18-20" Wide TILT	1
1i, 2d, 3 - 8	501151	Seat Pan Assembly Medium 20-22" Wide TILT	1
1j, 2e, 3 - 8	501219	Seat Pan Assembly Medium 22" Wide TILT	1
1k, 2a, 3 - 8	501152	Seat Pan Assembly Long 14-16" Wide TILT	1
1l, 2b, 3 - 8	501153	Seat Pan Assembly Long 16-18" Wide TILT	1
1m, 2c, 3 - 8	501154	Seat Pan Assembly Long 18-20" Wide TILT	1

Item Number	Part Number	Description	Quantity
1n, 2d, 3 - 8	501155	Seat Pan Assembly Long 20-22" Wide TILT	1
1o, 2e, 3 - 8	501220	Seat Pan Assembly Long 22-24" Wide TILT	1
1a	001184	Seat Pan Front Short 14-16" Wide TILT	1
1b	001185	Seat Pan Front Short 16-18" Wide TILT	1
1c	001186	Seat Pan Front Short 18-20" Wide TILT	1
1d	001187	Seat Pan Front Short 20-22" Wide TILT	1
1e	001188	Seat Pan Front Short 22" Wide TILT	1
1f	001189	Seat Pan Front Medium 14-16" Wide TILT	1
1g	001190	Seat Pan Front Medium 16-18" Wide TILT	1
1h	001191	Seat Pan Front Medium 18-20" Wide TILT	1
1i	001192	Seat Pan Front Medium 20-22" Wide TILT	1
1j	001193	Seat Pan Front Medium 22" Wide TILT	1
1k	001194	Seat Pan Front Long 14-16" Wide TILT	1
1l	001195	Seat Pan Front Long 16-18" Wide TILT	1
1m	001196	Seat Pan Front Long 18-20" Wide TILT	1
1n	001197	Seat Pan Front Long 20-22" Wide TILT	1
1o	001198	Seat Pan Front Long 22-24" Wide TILT	1
2a	001199	Seat Pan Rear 14-16" Wide TILT	1
2b	001200	Seat Pan Rear 16-18" Wide TILT	1
2c	001201	Seat Pan Rear 18-20" Wide TILT	1
2d	001202	Seat Pan Rear 20-22" Wide TILT	1
2e	001203	Seat Pan Rear 22-24" Wide TILT	1
3	001204	Seat Pan/Base Bracket TILT	4
4	001205	Seat Pan/Base Clamp TILT	4
5	001206	AB Tape 3x3 Die Cut Tilt	2
6	101814	M8 x 25 BHCS Blk Zn	4
7	101813	M8 x 18 BHCS Blk Zn	4
8	100659	M8 Nylock Nut Blk Zn	8