

Wheel Lock Setup




Zippie IRIS

25-Jul-13

- No-Go
- Setup Not Available
- *Forward
- Rearward
- Short Rearward

Attendant Wheel Lock Position

Rear Wheel Size	Roller Pos	Axle Pos							Order Kit #	*Cable & **Slide Block
		11	12	13	14	15	16	17		
12	1			2	1				108138 Rearward is Reverse (will need to flip the block)	*Short Cable and **Standard Slide Block
	2			3	2	1	1			
	3		1*	3	3	2	1	1		
	4			1*	3	3	2			
	5				1*	3	2			
16	1	1*							108139 for 20-24 Mag Wheels OR 108140 for 20-24 Spoke Wheels Rearward is Reverse (will need to flip the block)	*Long Cable and **Standard Slide Block
	2		1*			2	2			
	3		1*	1*		3	2	2		
	4			1*	1*	3	3			
	5				1*	1*	3			
20	1	2*	2*	1*	1*				108139 for 20-24 Mag Wheels OR 108140 for 20-24 Spoke Wheels Rearward is Reverse (will need to flip the block)	*Long Cable and **Standard Slide Block
	2		2*	2*	1*	1*				
	3		3*	2*	2*	1*	1*			
	4			3*	2*	2*	2*			
	5				3*	2*	2*			
22	1	3*	2*	2*	1*				108139 for 20-24 Mag Wheels OR 108140 for 20-24 Spoke Wheels Rearward is Reverse (will need to flip the block)	*Long Cable and **Standard Slide Block
	2		3*	2*	2*	1*				
	3		3*	3*	2*	2*	1*			
	4			3*	3*	2*	2*			
	5				3*	3*	2*			
24	1	3*	3*	2*	1*				108139 for 20-24 Mag Wheels OR 108140 for 20-24 Spoke Wheels Rearward is Reverse (will need to flip the block)	*Long Cable and **Standard Slide Block
	2		3*	2*	2*	1*	1*			
	3		3*	3*	3*	2*	2*	1*		
	4				3*	2*	2*			
	5					3*	3*			

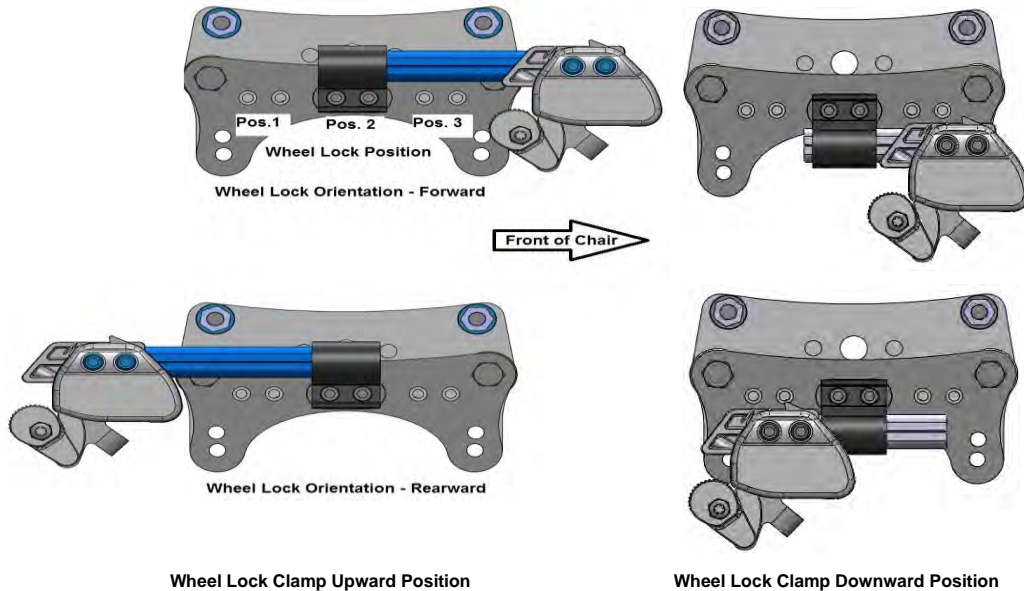

Notes:

- * When Axle Plate Orientation is in 1-4 (ENG1036), install wheel lock clamp in upward position
- * When Axle Plate Orientation is in 5-8 (ENG1036), install wheel lock clamp in downward position

Wheel Lock Setup




Zipie IRIS

25-Jul-13

- No-Go
- Setup Not Available
- *Forward
- Rearward
- Short Rearward

Short Rearward Attendant Wheel Lock Position

Rear Wheel Size	Roller Pos	Axle Pos							Order Kit #	*Cable & **Slide Block	
		11	12	13	14	15	16	17			
		12	1	3	3						
2			3								
3											
4											
5											
16	1		3	3	2				126253 Rearward is Reverse (will need to flip the block)	*Long Cable and **Short Slide Block	
	2			3	3						
	3				3						
	4										
	5										
20	1								126253 Rearward is Reverse (will need to flip the block)	*Long Cable and **Short Slide Block	
	2					3					
	3							3			
	4										
	5										
22	1										
	2										
	3										
	4										
	5										
24	1										
	2										
	3										
	4										
	5										

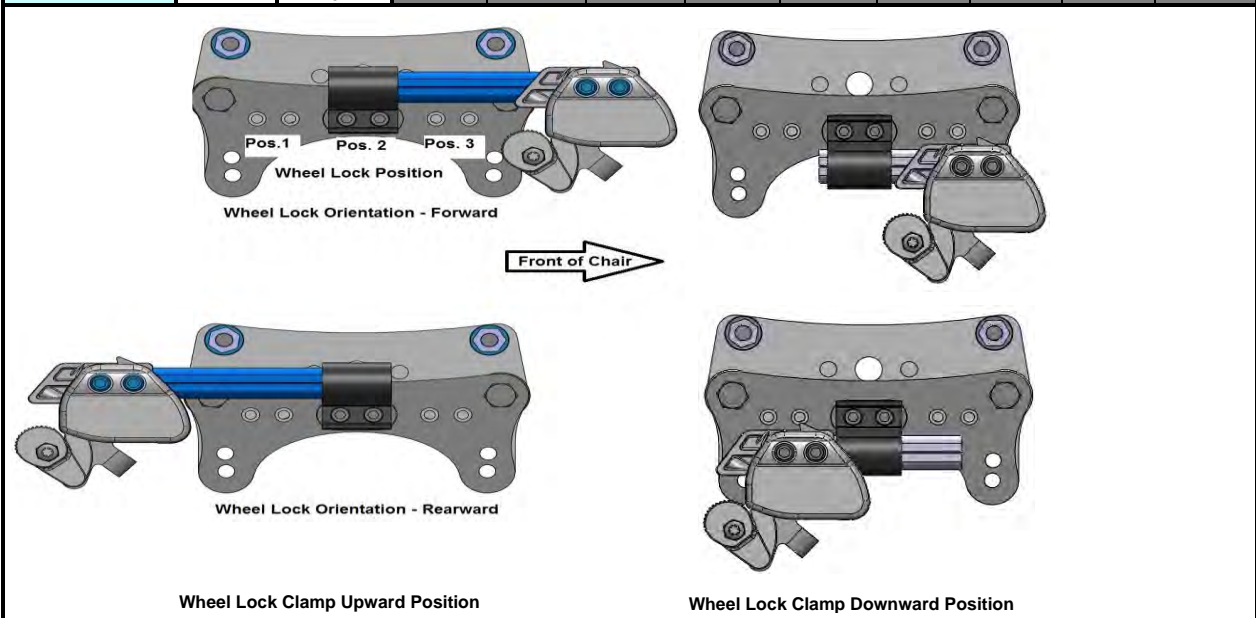

Notes:

- * When Axle Plate Orientation is in 1-4 (ENG1036), install wheel lock clamp in upward position
- * When Axle Plate Orientation is in 5-8 (ENG1036), install wheel lock clamp in downward position