


**The B118W Medial Thigh Support (Abductor Wedge) is designed to fit over the Medial Thigh Support already molded into your JAY<sup>®</sup> J2 Cushion or JAY<sup>®</sup> Cushion.**

1. Remove the cover and fluid pad from the JAY J2 Cushion or JAY Cushion. Place the Medial Thigh Support in the desired position on the cushion base. The rear of the Medial Thigh Support will face the rear of the cushion.
2. Clean the cushion base where the accessory will be located.
3. Remove the paper backing from the self-adhesive loop Velcro<sup>®</sup> strip attached to the bottom of the Medial Thigh Support. Reposition the Medial Thigh Support in the correct position on the base and press in place.
4. Remove the Medial Thigh Support and press the loop Velcro strip down firmly. Then re-attach the wedge.

### **Modifying the Medial Thigh Support (Abductor Wedge)**

The Medial Thigh Support may be shortened to fit the 110 (Junior JAY Cushion) and the 115 (Children's JAY Cushion) by cutting down the appropriate score line on the underside of the thigh support.

If a left or right side thigh support is required, the Medial Thigh Support is designed to be cut down the center score line. Additional Velcro strips are included to secure the side sections to the base. To use this additional Velcro, remove the backing from the hook Velcro and attach to the underside of the Medial Thigh Support closest to the new cut. Proceed with attachment as in steps 1-4.

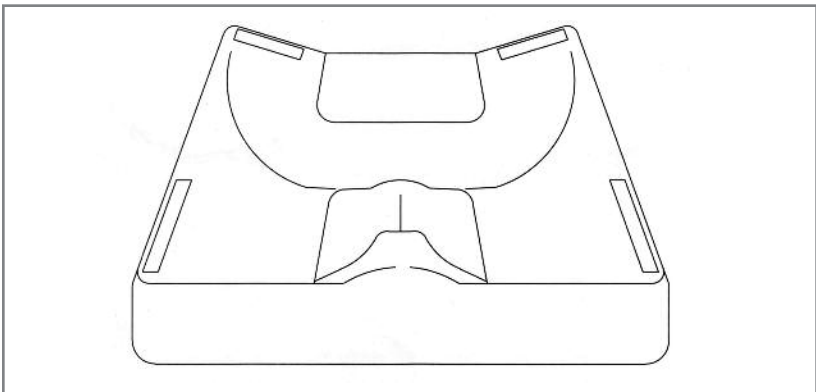

Sunrise Medical • 2842 Business Park Ave. • Fresno, CA 93727 • USA  
In Canada (800) 263-3390