


Tsunami Aluminum Caster Mount

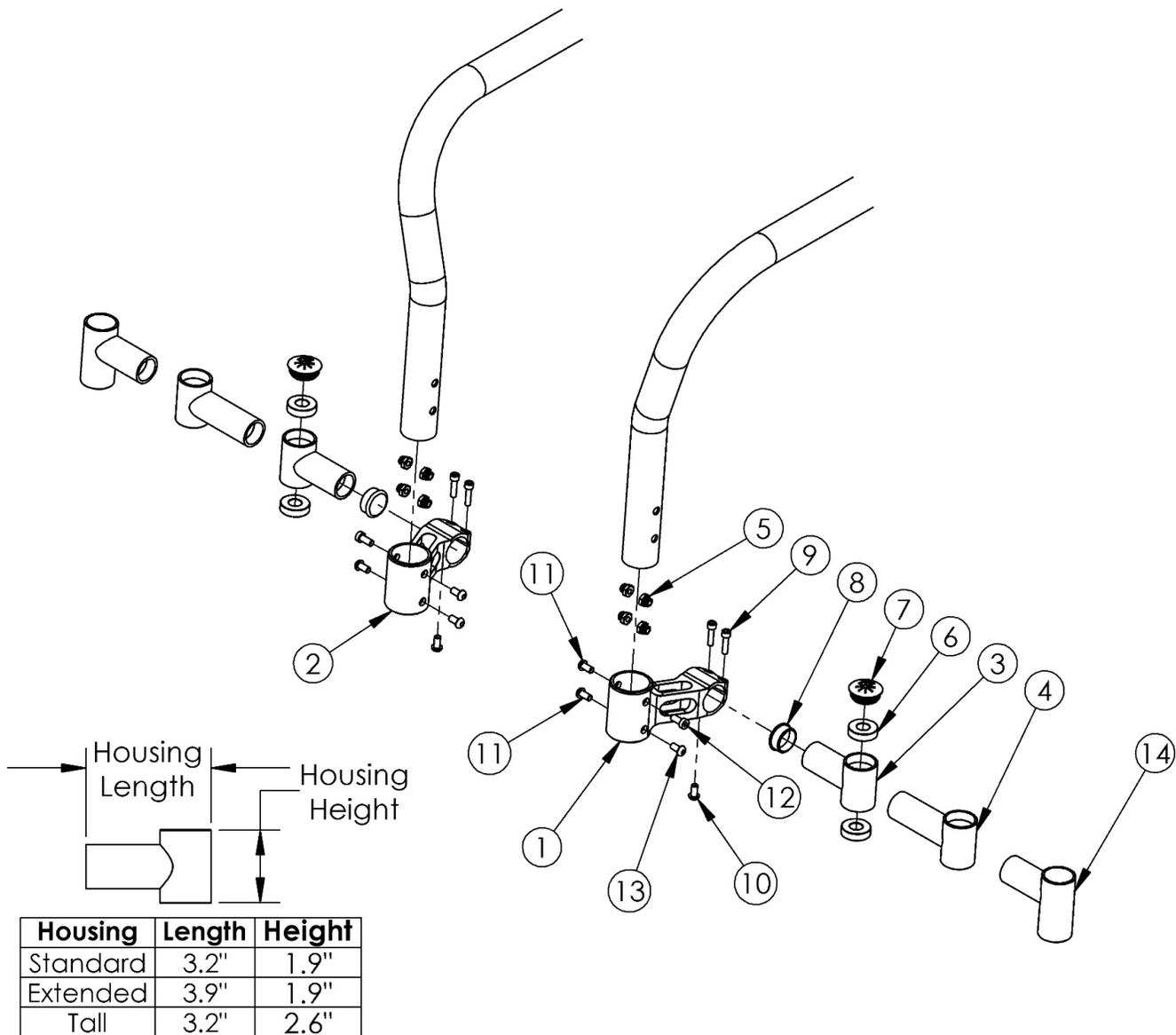

If quoting replacement frame, caster mounts are included with frame.

Item Number	Part Number	Description	Quantity	Retail
1, 5, 11a, 11b, 12a, 12b, 13	502648	Caster Mount Assembly With Hardware Left TSU AL	1	\$191.18
2, 5, 11a, 11b, 12a, 12b, 13	502649	Caster Mount Assembly With Hardware Right TSU AL	1	\$191.18
1, 3, 5 - 13	501178	Caster Mount Assembly With Standard Housing Left TSU AL	1	\$268.29
2, 3, 5 - 13	501179	Caster Mount Assembly With Standard Housing Right TSU AL	1	\$268.29

Item Number	Part Number	Description	Quantity	Retail
1, 2, 3, 5 - 13	501180	Caster Mount Assembly With Standard Housing Pair TSU AL	1	\$536.56
1, 4 - 13	501181	Caster Mount Assembly With Ext Housing Left TSU AL	1	\$268.29
2, 4 - 13	501182	Caster Mount Assembly With Ext Housing Right TSU AL	1	\$268.29
1, 2, 4 - 13	501183	Caster Mount Assembly With Ext Housing Pair TSU AL	1	\$536.56
3, 6 - 10	502650	Caster Housing Assembly With Hardware Standard Left TSU	1	\$105.35
3, 6 - 10	505239	Caster Housing Assembly With Hardware Standard Right TSU	1	\$105.35
4, 6 - 10	502651	Caster Housing Assembly With Hardware Extended Left TSU	1	\$105.35
4, 6 - 10	505240	Caster Housing Assembly With Hardware Extended Right TSU	1	\$105.35
1	000390L	Caster Mount AL Left TSU/LW	1	\$156.15
2	000390R	Caster Mount AL Right TSU/LW	1	\$156.15
3, 6	000251	Caster Housing TSU/LW	2	\$75.43
4, 6	000458	Caster Housing Extended TSU/LW	2	\$75.43
5	000491	Caster T-Nut TSU AL	8	\$6.38
6	100530	Bearing M28 x 1/2" 6001RS	4	\$12.02
7	102005	Ki Caster Housing Cap	2	\$1.17
8	100532	Tube End Plug 1 1/8"	2	\$2.63
9	101843	M5 x 20 SHCS Blk Zn	4	\$0.53
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58
11a	101184	M6 x 10 BTN SHCS BLKZC w/Patch <i>Used when transit brackets not present</i>	4	\$1.58
11b	101708	M6 x 14 BTN SHCS BLKZC w/Patch <i>Used with transit brackets</i>	4	\$1.58
12a	101185	M6 x 14 CLS8.8 DIN7984 Low SHCS BLKZC <i>Used when footrest is not high-mounted</i>	2	\$1.58
12b	101184	M6 x 10 BTN SHCS BLKZC w/Patch <i>Used with high-mounted footrests</i>	2	\$1.58
13	101184	M6 x 10 BTN SHCS BLKZC w/Patch	2	\$1.58