


# Discontinued Angle Adjustable Non Locking Tubular Flip Up Armrest

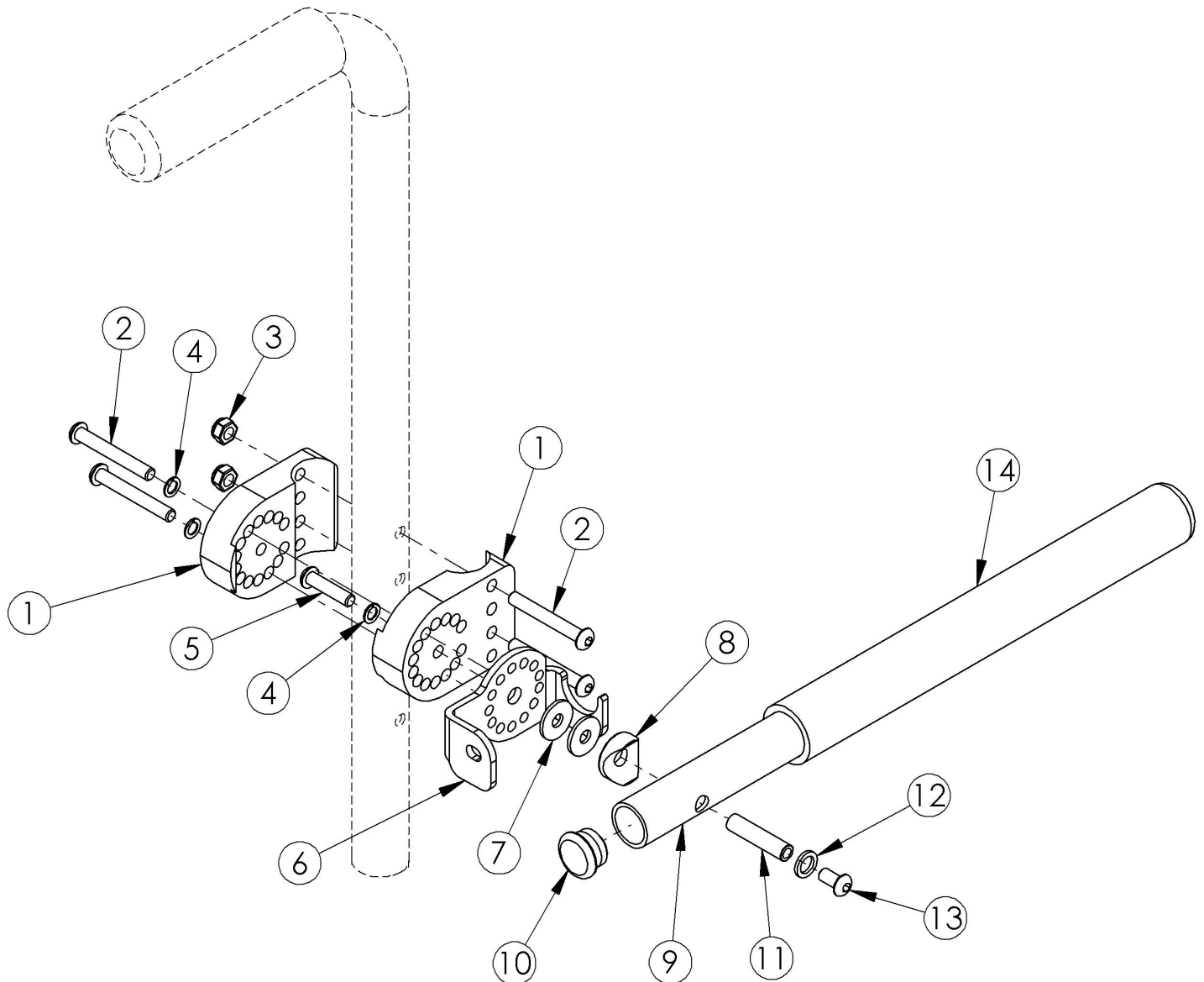

*This Angle Adjustable Non Locking Tubular Flip Up Armrest is discontinued. Please contact Customer Service for more information.*

Item Number	Part Number	Description	Quantity	Retail
1-5, 6b, 7-14	101209	(Discontinued) Non Locking Flip Up Arm Assm Left	1	\$214.68
1-5, 6a, 7-14	101210	(Discontinued) Non Locking Flip Up Arm Assm Right	1	\$214.68
1	000901	Flip Up Armrest Clamp	2	\$54.48
2	100670	M6 x 45 BTN SHCS BLK ZC	4	\$1.58

Item Number	Part Number	Description	Quantity	Retail
3	100658	M6 Nylock Nut Blk Zn	2	\$1.05
4	101835	M6 Lock Washer Blk Zn	3	\$0.53
5	101593	M6 x 25 BTN SHCS BLKZC	1	\$1.58
6a	000672R	Flip Up Armrest Pivot Right	1	\$55.70
6b	000672L	Flip Up Armrest Pivot Left	1	\$55.70
7	101098	Flip Up Armrest Washer	2	\$1.11
8	000697	Saddle 7/8"	1	\$1.15
9, 14	501022	Arm Tube Assm Non Locking Flip	1	\$37.60
10	100803	Tube End Plug 7/8"	2	\$1.58
11	000347	M6 x 1.0 Threaded Barrel	1	\$9.06
12	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	1	\$2.63
13	100667	M6 x 12 BTN SHCS BLKZC w/Patch	1	\$1.58