


Click Wheel Lock Clamps

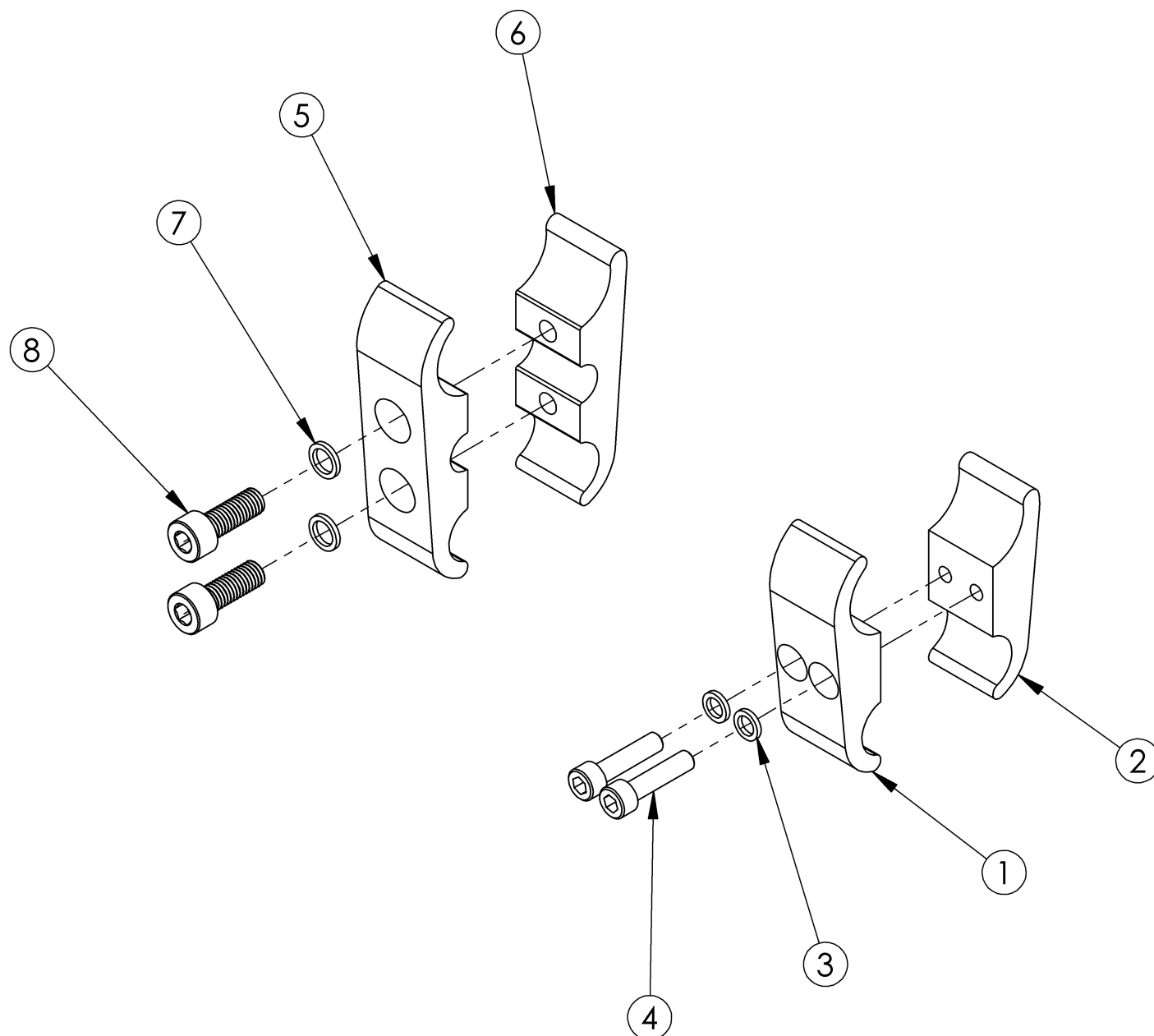

Item Number	Part Number	Description	Quantity
1a, 2a, 3, 4	502977	Clamp Assembly 1.125" Tubing, CLIK	1
1b, 2b, 3, 4	503460	Clamp Assembly 1.125" Tubing, Long, CLIK <i>Used with Offset Frame</i>	1
5 - 8	502976	Clamp Assembly 1.125" Tubing Extra Long CLIK <i>Used as needed or if Height Adjustable Low T-Arms are present</i>	1
1a	002541	Clamp 1.125 to .75 C'Bored	1

Item Number	Part Number	Description	Quantity
1b	002831	Wheel Lock Clamp 1.125 to .75 C'Bored Long Clik <i>Used with Offset Frame</i>	1
2a	002542	Clamp 1.125 to .75 Threaded	1
2b	002832	Wheel Lock Clamp 1.125 to .75 Threaded Long Clik <i>Used with Offset Frame</i>	1
3	102721	Washer .250 ID x .375 OD x .060 THK	2
4	100780	M6 x 25 SHCS ZC	2
5	002814	Wheel Lock Clamp 1.125 to .75 C'Bored Extra Long Clik <i>Used as needed or if Height Adjustable Low T-Arms are present</i>	1
6	002815	Wheel Lock Clamp 1.125 to .75 Threaded Extra Long Clik <i>Used as needed or if Height Adjustable Low T-Arms are present</i>	1
7	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	2
8	102864	M8 x 25mm SHCS, BLK ZC	2