


Catalyst E Caster Housing - Heavy Duty

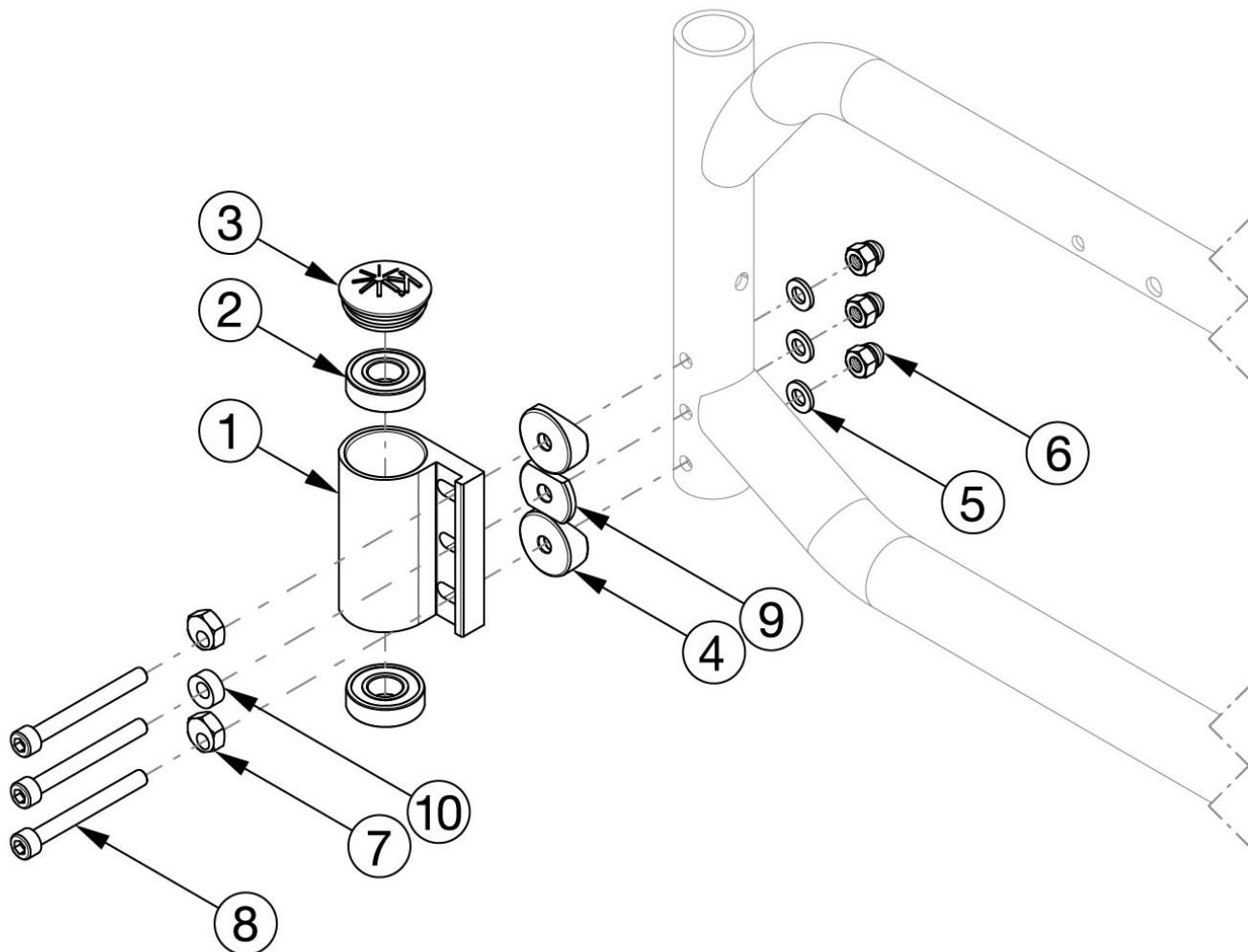

Please Note: The shown orientation of the caster housing above is in the "Forward" position.

Item Number	Part Number	Description	Quantity
1a, 2 - 10	115830	HD Caster Housing Assembly, Black, Left, No Transit	1
1b, 2 - 10	115831	HD Caster Housing Assembly, Black, Right, No Transit	1
1a, 1b, 2 - 10	115832	HD Caster Housing Assembly, Black, PAIR, No Transit	1
1a, 2	100526	Caster Housing Left, Black, CAT/Liberty	1
1b,2	100527	Caster Housing Right, Black, CAT/Liberty	1
3	102005	Ki Caster Housing Cap	1
4	000056	Saddle 1 1/8"	2

Item Number	Part Number	Description	Quantity
5	100746	M6 Flat Washer Blk Zn	3
6	100558	M6 Nyl Ins Domed Cap L/N Zc	3
7	100510	Eccentric Washer	2
8	111814	SHCS, M6x1.0X55, CLS10.9, PT, BLZ,DIN912, Custom	3
9	111813	1 1/8" Half Saddle	1
10	111544	Eccentric Washer, 0.50" Diameter	1