


Rogue2 Rear Anti-Tip

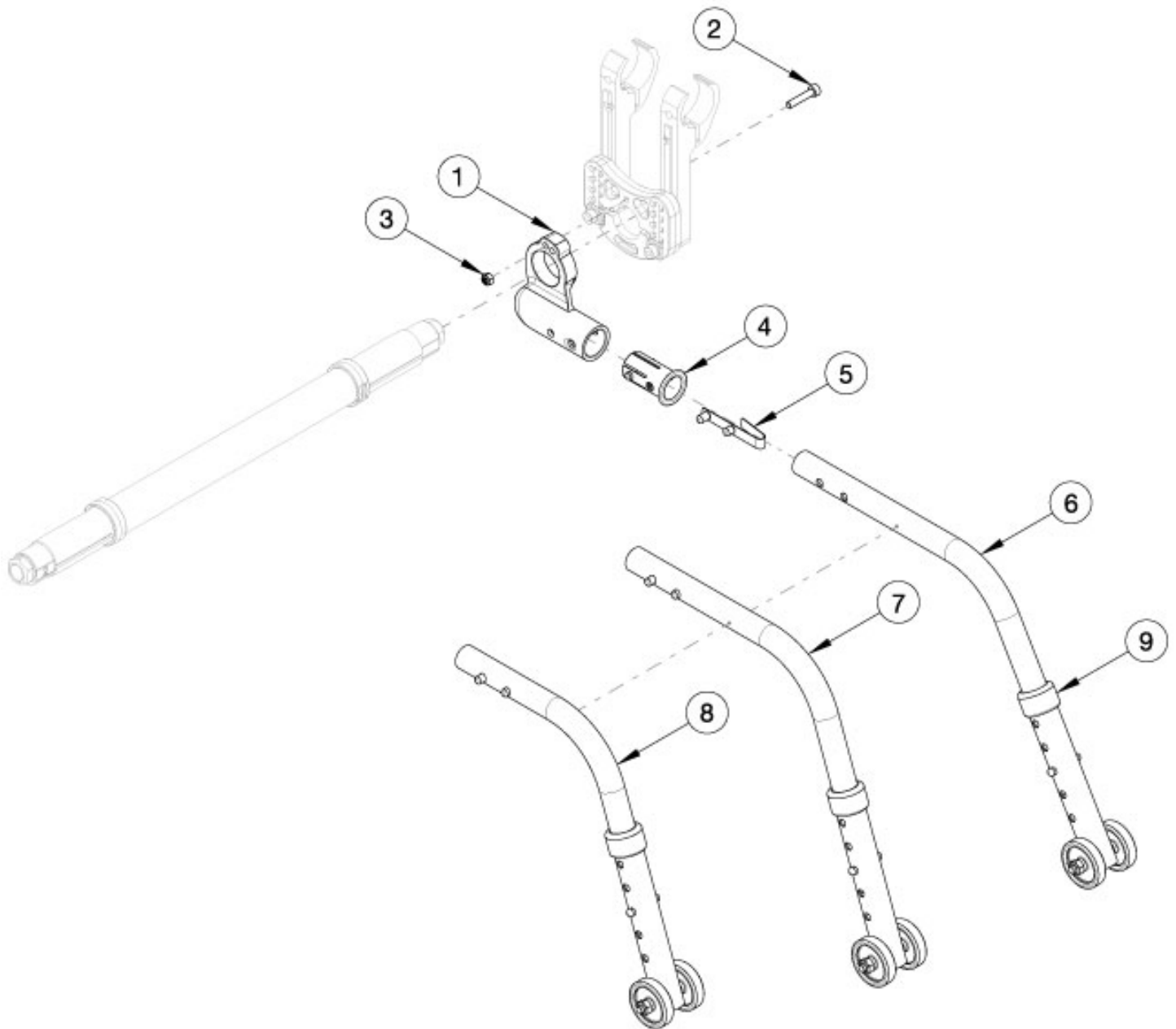

If retrofitting, Anti-Tip Receivers will be needed.

Item Number	Part Number	Description	Quantity
1a, 4	117465	Standard Anti Tip Receiver with Sleeve, Right Side, Rogue2	1
1b, 4	117466	Standard Anti Tip Receiver with Sleeve, Left Side, Rogue2	1
5, 8, 9	117467	Standard Anti Tip Lower Assembly, 18-20" Wheels, Rogue2	1
5, 7, 9	117468	Standard Anti Tip Lower Assembly, 22" Wheels, Rogue2	1

Item Number	Part Number	Description	Quantity
5, 6, 9	117469	Standard Anti Tip Lower Assembly, 24-26" Wheels, Rogue2	1
1a, 2-5, 8, 9	117470	Standard Anti Tip Receiver Complete Assembly, 18-20" Wheels, Right Side, Rogue2	1
1b, 2-5, 8, 9	117471	Standard Anti Tip Receiver Complete Assembly, 18-20" Wheels, Left Side, Rogue2	1
1a, 2-5, 7, 9	117472	Standard Anti Tip Receiver Complete Assembly, 22" Wheels, Right Side, Rogue2	1
1b, 2-5, 7, 9	117473	Standard Anti Tip Receiver Complete Assembly, 22" Wheels, Left Side, Rogue2	1
1a, 2-6, 9	117474	Standard Anti Tip Receiver Complete Assembly, 24-26" Wheels, Right Side, Rogue2	1
1b, 2-6, 9	117475	Standard Anti Tip Receiver Complete Assembly, 24-26" Wheels, Left Side, Rogue2	1
1a	113997	Rogue Standard Tipper Mount, Right	1
1b	114001	Rogue Standard Tipper Mount, Left	1
2	116550	M5 x 0.8 x 30, SHCS CLS 10.9, PT, BLZ, DIN 912	1
3	100657	M5 Nylock Nut Blk Zn	1
4	114018	Standard Tipper Sleeve, Rogue	1
5	100533	Detent Button	1
6	200524	Anti Tip Extra Long 24-26 RG Cat w/Reclining Back	1
7	100228	Anti Tip Long 25/26CAT 22ROGUE 22-26RG ALX/LW XP 26ETHOS	1
8	100226	Anti Tip Medium 22/24CAT 18/20RIGID 22/24/25ETHOS	1
9	004306	Anti-Tip Anti Rattle Bushing	1