


Ethos Rear Base Frame

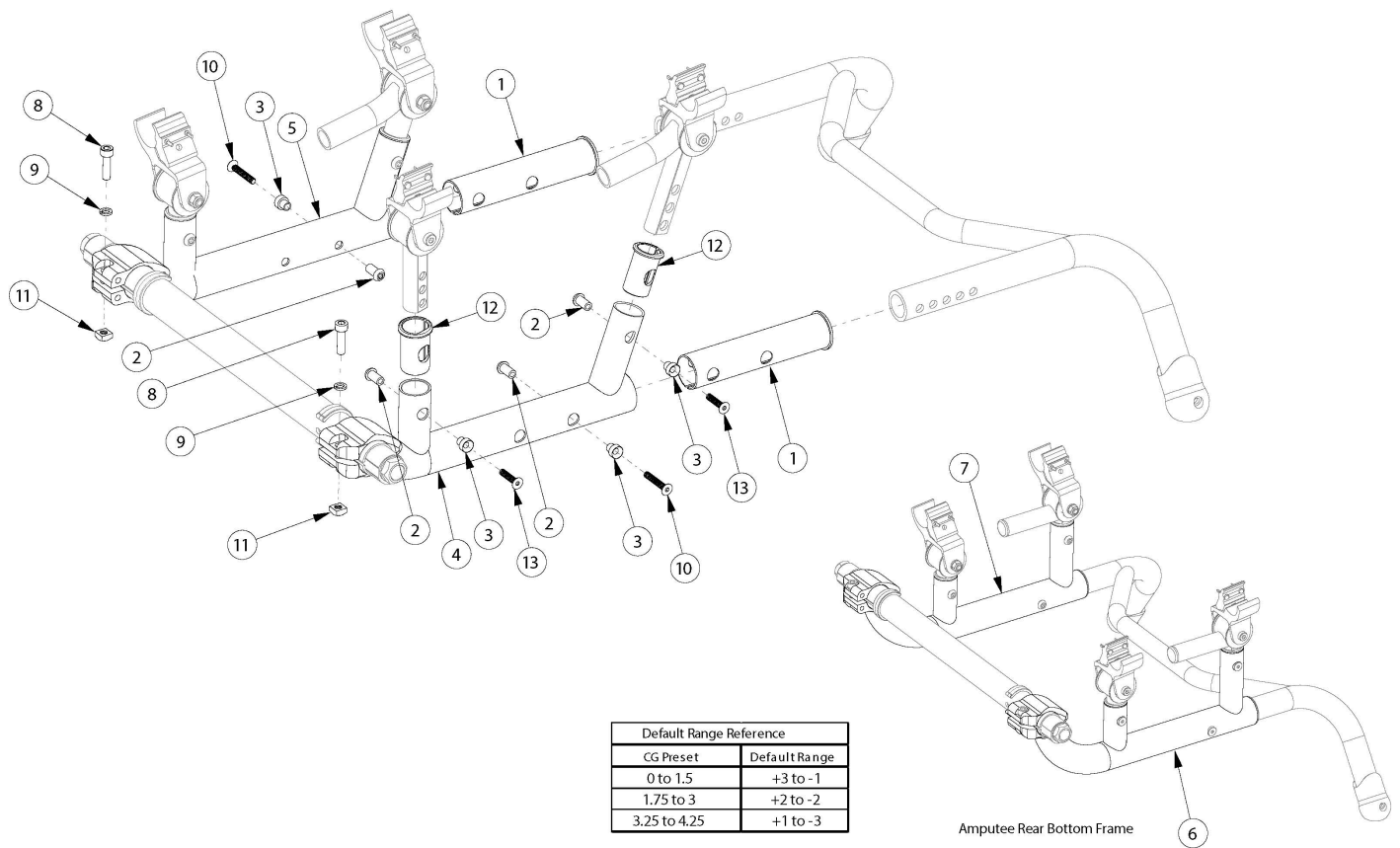

Replacement Rear Base Frames can be quoted but must be ordered through Customer Service

Item Number	Part Number	Description	Quantity
1a	003868	Base Frame Bushing, Black	2
1b	107711_ _ _	Base Frame Bushing, Highlight Color	2
2	003869	M5 Barrel Nut	6
3	003870	Barrel, M5 FHCS	6
8	101845	M6 x 22 SHCS Blk Zn	2
9	102810	M6 Lockwasher, Hi-Collar, ZC	2
10	103321	M5 x 30 FHCS Blk Zc	2
11	103505	M6 x 1.0 Square Nut Blk Zc	2
12a	003867	Vertical Upright Bushing With Flat, Black	4

Item Number	Part Number	Description	Quantity
12b	108060_ _ _	Vertical Upright Bushing, Highlight Color	4
13	103322	M5 x 20 FHCS Blk Zc	4