



## Q300 M MOTOR BUMPER ADJUSTMENT





Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

### **WARNING!**

#### **Motor bump stop adjustment**

Depending on the configuration of the drive wheels, the upper motor bump stop must be adjusted to the setting illustrated in Fig 1 and Fig 2.

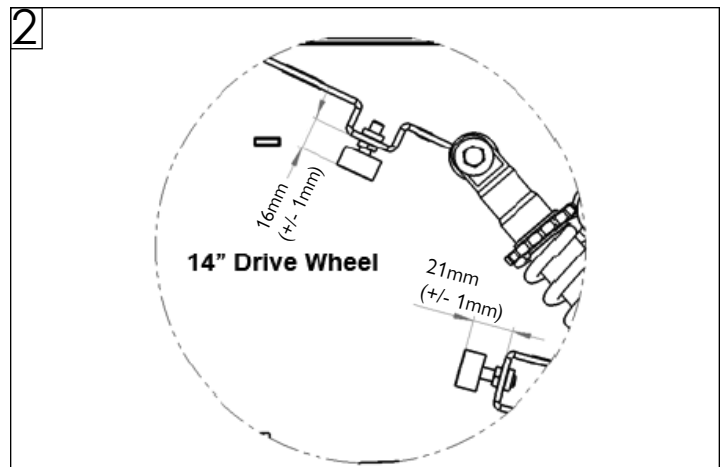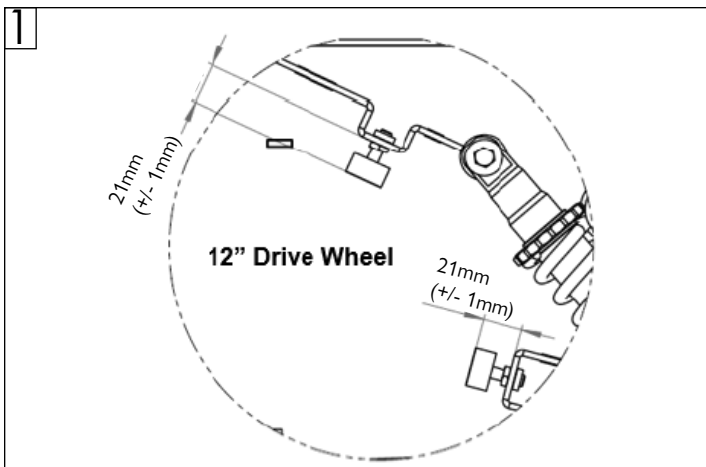

### **DEALER/TECHNICIAN WARNING**

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury.

Find more information and important warnings see the wheelchair owner's manual or at: [www.SunriseMedical.com](http://www.SunriseMedical.com).

**Sunrise Medical 2842 N. Business Park Ave Fresno, CA 93727 USA**

**In Canada (800) 263-3390**



**Customer service: +1(800) 333-4000  
or visit [www.SunriseMedical.com](http://www.SunriseMedical.com)**

© 2021 Sunrise Medical (US) LLC  
06/2021 255308 Rev. A  
Page 1 of 1