


# Catalyst Tension Adjustable Back Upholstery - Extra Tall Backposts

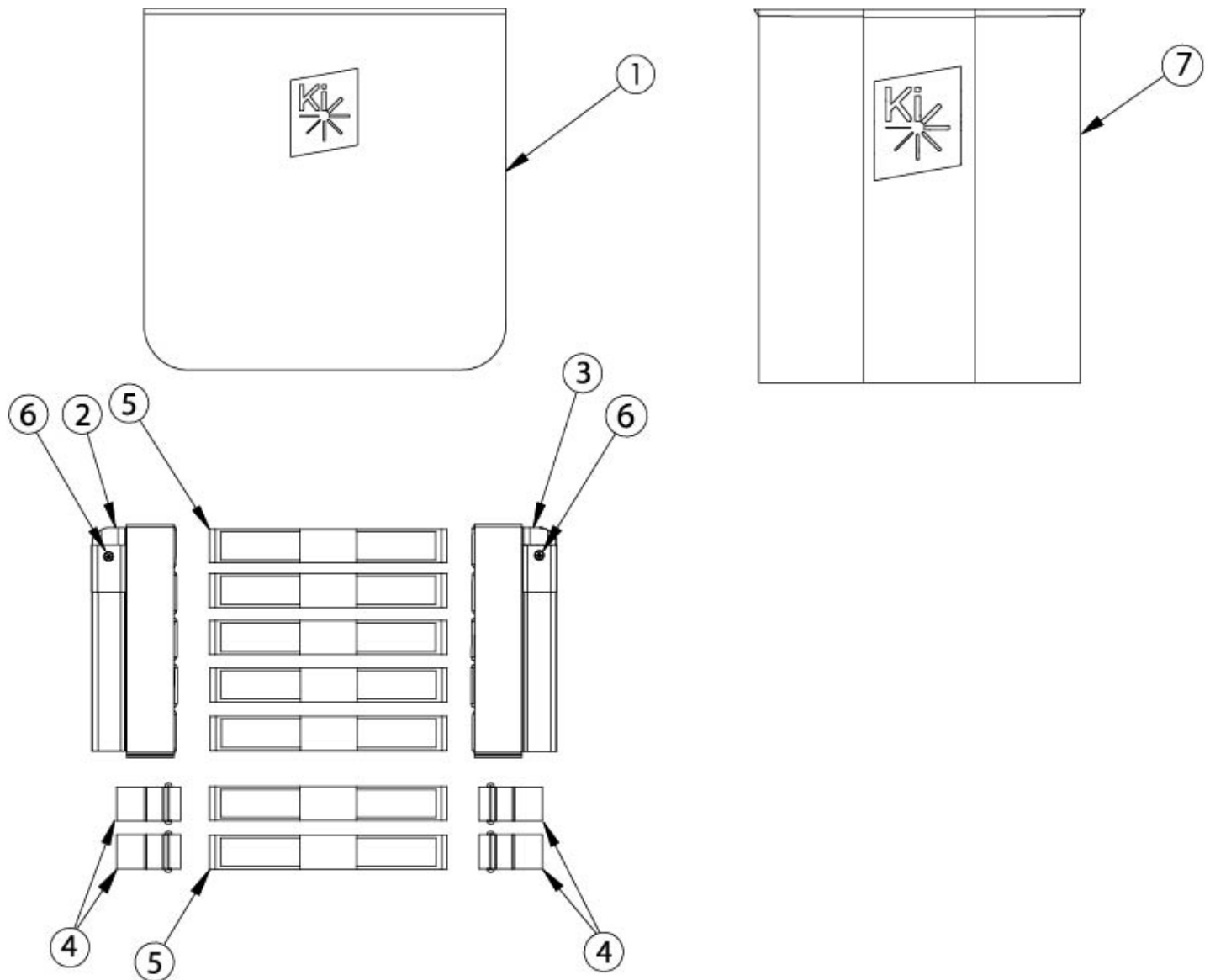

Backrest Range and Height	Wings	Total Strap Qty	Total D-Ring Qty
Extra Tall 21"	003096 003097	9	8
Extra Tall 22" - 23"	003096 003097	10	10
Extra Tall 24"	003096 003097	11	12

Assemblies contain the cover, amount of straps and d-rings used for the tallest height of 24" Attaching hardware is not included with the back upholstery assemblies

Item Number	Part Number	Description	Quantity
1a, 2-4, 5a	506794	Back Upholstery Assembly Tension Adjustable with Standard Cover 16W, 21" - 24", Extra Tall Backposts	1
1b, 2-4, 5b	506795	Back Upholstery Assembly Tension Adjustable with Standard Cover 17W, 21" - 24", Extra Tall Backposts	1
1c, 2-4, 5c	506796	Back Upholstery Assembly Tension Adjustable with Standard Cover 18W, 21" - 24", Extra Tall Backposts	1
1d, 2-4, 5d	506797	Back Upholstery Assembly Tension Adjustable with Standard Cover 19W, 21" - 24", Extra Tall Backposts	1
1e, 2-4, 5e	506798	Back Upholstery Assembly Tension Adjustable with Standard Cover 20W, 21" - 24", Extra Tall Backposts	1
1f, 2-4, 5f	506799	Back Upholstery Assembly Tension Adjustable with Standard Cover 21W, 21" - 24", Extra Tall Backposts	1
1g, 2-4, 5g	506800	Back Upholstery Assembly Tension Adjustable with Standard Cover 22W, 21" - 24", Extra Tall Backposts	1
2-4, 5a, 7a	108028	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16W, 21" - 24", Extra Tall Backposts	1
2-4, 5b, 7b	108029	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17W, 21" - 24", Extra Tall Backposts	1
2-4, 5c, 7c	108030	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18W, 21" - 24", Extra Tall Backposts	1
2-4, 5d, 7d	108031	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19W, 21" - 24", Extra Tall Backposts	1
2-4, 5e, 7e	108032	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20W, 21" - 24", Extra Tall Backposts	1
2-4, 5f, 7f	108033	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 21W, 21" - 24", Extra Tall Backposts	1
2-4, 5g, 7g	108034	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 22W, 21" - 24", Extra Tall Backposts	1
1a	003237	Backrest Cover 16" CAT/Liberty x 21-24"	1
1b	003047	Backrest Cover 17" CAT/Liberty x 21-24"	1
1c	003053	Backrest Cover 18" CAT/Liberty x 21-24"	1
1d	003059	Backrest Cover 19" CAT/Liberty x 21-24"	1
1e	003065	Backrest Cover 20" CAT/Liberty x 21-24"	1
1f	003071	Backrest Cover 21" CAT/Liberty x 21-24"	1
1g	003075	Backrest Cover 22" CAT/Liberty x 21-24"	1

Item Number	Part Number	Description	Quantity
2	003097	Tension Adjustable Upholstery Wing 9.5" Height Left	1
3	003096	Tension Adjustable Upholstery Wing 9.5" Height Right	1
4	100973	Back Upholstery D-Ring	2
5a	002533	Back Upholstery Strap 19.3" <i>Used on 16" chair widths</i>	5
5b	002534	Back Upholstery Strap 21.3" <i>Used on 17" chair widths</i>	5
5c	002535	Back Upholstery Strap 23.3" <i>Used on 18" chair widths</i>	5
5d	002536	Back Upholstery Strap 25.3" <i>Used on 19" chair widths</i>	5
5e	002537	Back Upholstery Strap 27.3" <i>Used on 20" chair widths</i>	5
5f	002538	Back Upholstery Strap 29.3" <i>Used on 21" chair widths</i>	5
5g	002539	Back Upholstery Strap 31.3" <i>Used on 22" chair widths</i>	5
6	100734	M5 x 16 PHMS ZC	2
7a	003908	Air Mesh Backrest Cover 16C 15R Width x 21-24" Height	1
7b	003917	Air Mesh Backrest Cover 17C 16R Width x 21-24" Height	1
7c	003923	Air Mesh Backrest Cover 18C 17R Width x 21-24" Height	1
7d	003929	Air Mesh Backrest Cover 19C 18R Width x 21-24" Height	1
7e	003935	Air Mesh Backrest Cover 20C 19R Width x 21-24" Height	1
7f	003941	Air Mesh Backrest Cover 21C 20R Width x 21-24" Height	1
7g	003945	Air Mesh Backrest Cover 22C 21R Width x 21-24" Height	1