


## Catalyst 5VX Vertical Axle Plate

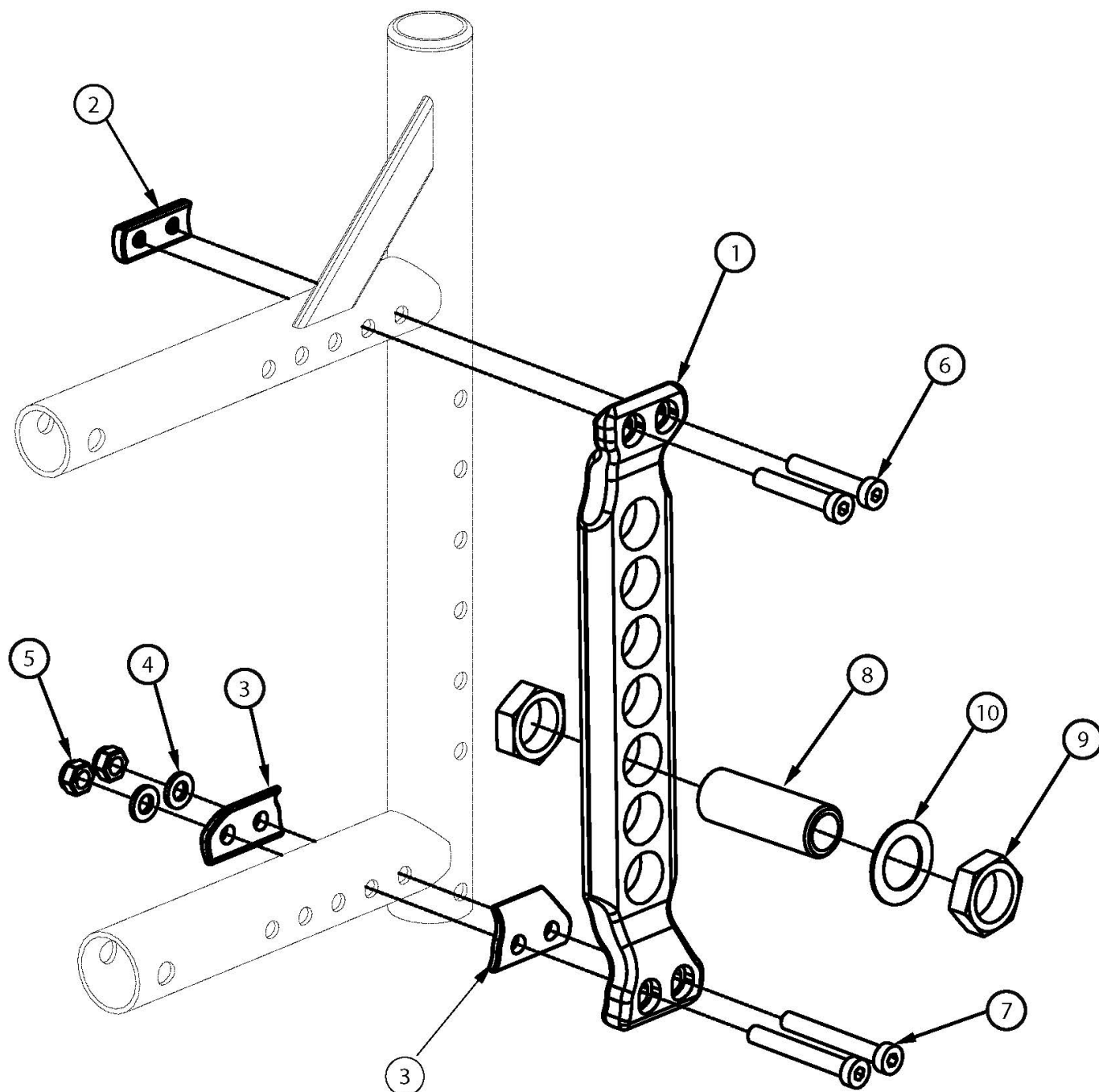

*This design effective January 22nd, 2018 for most Catalyst 5VX chairs. All Catalyst 5VX chairs received this design February 20th, 2019. <p>*

*Standard Camber will have the camber spacer mounted on the outside of the rear frame. Adjustable Camber will have the camber spacer mounted on the inside of the rear frame for later use. <p>*

*If retrofitting to add camber spacers and the discontinued rear frames and discontinued vertical axle plate are present, new rear frames and new vertical axle plates are required.*

Item Number	Part Number	Description	Quantity	Retail
1a, 2, 4-6, 7a	506127	Vertical Axle Plate Assembly, Black, 0° Camber <i>This kit is only for one side</i>	1	\$90.73
1b, 2, 4-6, 7a	506127_ _ _	Color Anodized Vertical Axle Plate Assembly, 0° Camber <i>This kit is only for one side</i>	1	\$148.51
1a, 2 - 6, 7b	506128	Vertical Axle Plate Assembly, Black, 1° Camber <i>This kit is only for one side</i>	1	\$93.88
1b, 2 - 6, 7b	506128_ _ _	Color Anodized Vertical Axle Plate Assembly, 1° Camber <i>This kit is only for one side</i>	1	\$151.66
1a, 2 - 6, 7b	506129	Vertical Axle Plate Assembly, Black, 2° Camber <i>This kit is only for one side</i>	1	\$97.03
1b, 2 - 6, 7b	506129_ _ _	Color Anodized Vertical Axle Plate Assembly, 2° Camber <i>This kit is only for one side</i>	1	\$154.82
1a, 2 - 6, 7c	506130	Vertical Axle Plate Assembly, Black, 3° Camber <i>This kit is only for one side</i>	1	\$114.93
1b, 2 - 6, 7c	506130_ _ _	Color Anodized Vertical Axle Plate Assembly, 3° Camber <i>This kit is only for one side</i>	1	\$172.72
1a, 2 - 6, 7c	506131	Vertical Axle Plate Assembly, Black, 4° Camber <i>This kit is only for one side</i>	1	\$118.09
1b, 2 - 6, 7c	506131_ _ _	Color Anodized Vertical Axle Plate Assembly, 4° Camber <i>This kit is only for one side</i>	1	\$175.87
1a, 8-10	506132	Vertical Axle Plate, Black, with Receiver Assembly 1/2" <i>This kit is only for one side</i>	1	\$85.23
1b, 8-10	506132_ _ _	Color Anodized Vertical Axle Plate, with Receiver Assembly 1/2" <i>This kit is only for one side</i>	1	\$143.01
8-10	114172	Axle Receiver Hardware <i>This kit is only for one side</i>	1	\$21.81
1a	003519INST	VX Axle Plate Black with Instruction	1	\$63.41
1b	506281_ _ _	Color Anodized VX Axle Plate with Instruction	1	\$121.20
2	003520	Threaded Axle Nut Plate	1	\$10.51
3	003521	1 Degree Camber Spacer <i>Use Qty 1 per each 1° of camber</i>	1	\$3.15
4	100746	M6 Flat Washer Blk Zn	2	\$1.58
5	100658	M6 Nylock Nut Blk Zn	2	\$1.05
6	103370	M6 x 35 Low SHCS BLK ZC w/Patch	2	\$4.20

Item Number	Part Number	Description	Quantity	Retail
7a	101123	M6 x 45 Low SHCS BLK ZC <i>Used with 0° camber</i>	2	\$1.58
7b	101124	M6 x 50 Low SHCS BLK ZC <i>Used with 1° or 2° camber</i>	2	\$1.58
7c	103371	M6 x 60 CLS8.8 DIN7984 Low SHCS BLKZC <i>Used with 3° or 4° camber</i>	2	\$8.94
8	100756	Axle Receiver 1/2"	1	\$17.68
9	100514	Axle Receiver Nut M18 x D12 x 8mm	2	\$1.54
10	100583	Axle Plate Washer 18mm	1	\$1.05

# Color Anodizing Samples

