

## INSTALLATION INSTRUCTIONS

PLEASE NOTE: THIS UNIT IS MADE FOR USE ON TUBS WITH WALLS UP TO 6" WIDE. IF YOUR TUB WALLS ARE THICKER THAN THIS, DO NOT INSTALL AS THE UNIT WILL NOT PROVIDE STABILITY AND WILL BE DANGEROUS.

FOR MAXIMUM SUPPORT THIS ITEM MUST BE INSTALLED CORRECTLY. ALWAYS TEST THE BATH SAFETY BAR BEFORE USING. INCORRECT INSTALLATION MAY CAUSE THE BATH SAFETY BAR TO LOOSEN AND NOT PROVIDE STABILITY.

1. Clean mounting site of all dust, dirt and soap build-up. Be sure surface is completely clean and dry.
2. Position the Bath Safety Bar on side of tub with rotating plate inside tub.
3. Position base of bolt in circular flange and tighten bolt with a wrench until Bath Safety Rail is securely attached to tub.
4. Check to see that the base of the bolt is still in the circular flange and that all 3 rubber pads are flat against the tub walls. If not, loosen and reinstall with all 3 pads flat against the tub walls.
5. You are now ready to use. CAUTION, YOU MUST TEST THE RAIL BEFORE USING AND IT SHOULD BE CHECKED PERIODICALLY FOR TIGHTNESS.
6. As this is a Safety Support item, we advise anyone having any difficulty following these instructions to use the services of a qualified installer.

Manufactured for:  
JOBAR INTERNATIONAL  
Carson CA USA

ITEM # JB2993

MADE IN TAIWAN