




Flip / Arc Tubular Flip Up Armrest

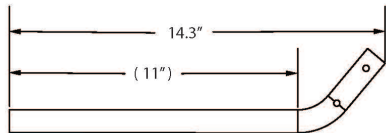

Standard Flip Up Armrest

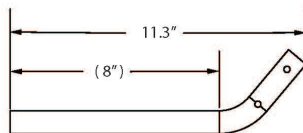

Short Flip Up Armrest

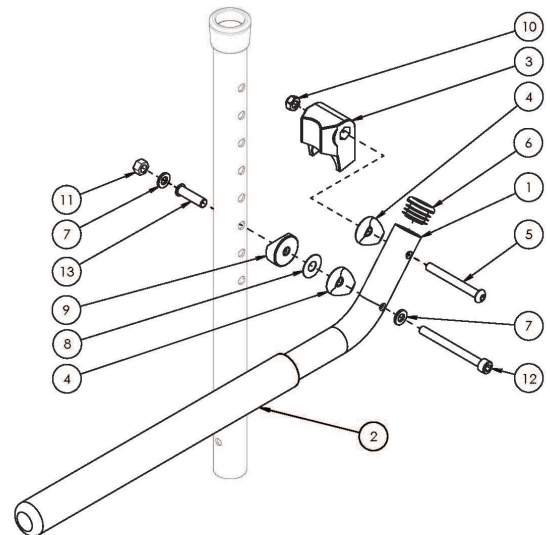

Item Number	Part Number	Description	Quantity
1a, 2 - 13	506133	Tubular Flip Up Armrest Assembly, Flip	1
1a, 2 - 13	506134	Tubular Flip Up Armrest Assembly, Flip (Pair)	1
1b, 2 - 13	506135	Tubular Flip Up Armrest Assembly Short, Flip <i>Used if chair depth is 12"-14" and Therafin Tray is not present</i>	1
1b, 2 - 13	506136	Tubular Flip Up Armrest Assembly Short, Flip (Pair) <i>Used if chair depth is 12"-14" and Therafin Tray is not present</i>	1
1a, 2, 6	501250	Tubular Flip Up Armrest Tube Assembly	1
1b, 2, 6	501251	Tubular Flip Up Armrest Tube Assembly Short <i>Used if chair depth is 12"-14" and Therafin Tray is not present</i>	1
3	501391	Tubular Flip Up Armrest Stop Block	1
4	000054	Saddle 7/8"	2
5	101125	M6 x 55 BTN SHCS BLKZC	1
6	100803	Tube End Plug 7/8"	1
7	100746	M6 Flat Washer Blk Zn	2
8	000428	Wear Washer	1
9	000055	Saddle 1"	1

Item Number	Part Number	Description	Quantity
10	100658	M6 Nylock Nut Blk Zn	1
11	101456	M6 Ovl Top L/N ZC	1
12	101849	M6 x 65 SHCS Blk Zn	1
13	002914	Sleeve .266 OD x .236 ID x .97 LG	1