


Catalyst 5 Universal Axle Plate

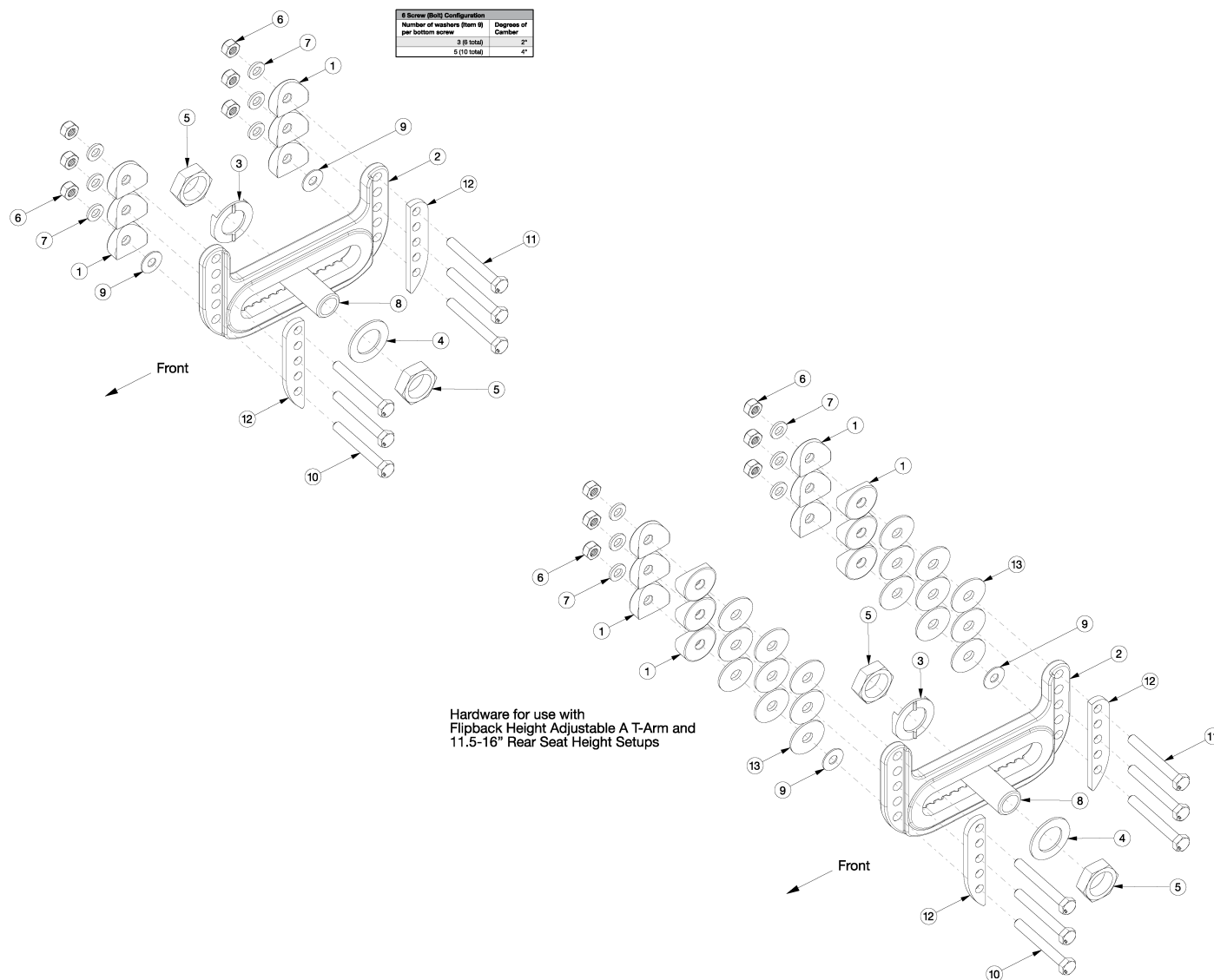





If chair was invoiced prior to November 14, 2018 and retrofitting to the universal axle plate and anti-tips are present, new anti-tip receivers are required.

Item Number	Part Number	Description	Quantity
1, 2a, 6, 7, 10a, 11a, 11b, 11c, 13	107413	Universal Axle Plate, Black Anodized with Hardware Kit, Curved/Titanium Rear Frame	1
1, 2a, 6, 7, 10c, 10d, 10f, 10g, 11a, 11c, 11d, 11f, 12, 13	114605	Universal Axle Plate, Black Anodized with Hardware Kit, Curved/Titanium Rear Frame, Heavy Duty	1
1, 2b, 6, 7, 10a, 11a, 11b, 11c, 13	107414_ _ _	Color Anodized/Color Accent Universal Axle Plate with Hardware Kit, Curved / Titanium Rear Frame	1

Item Number	Part Number	Description	Quantity
1, 2b, 6, 7, 10c, 10d, 10f, 10g, 11a, 11c, 11d, 11f, 12, 13	114606_ _ _	Color Anodized/Color Accent Universal Axle Plate with Hardware Kit, Curved / Titanium Rear Frame, Heavy Duty	1
1, 2a, 6, 7, 10b, 11a, 11c, 11d, 11e, 13	107415	Universal Axle Plate, Black Anodized with Hardware Kit, Depth Adjustable Rear Frame	1
1, 2a, 6, 7, 10a, 10c - 10e, 10g, 11a, 11b, 11d - 11f, 12, 13	114607	Universal Axle Plate, Black Anodized with Hardware Kit, Depth Adjustable Rear Frame, Heavy Duty	1
1, 2b, 6, 7, 10b, 11a, 11c, 11d, 11e, 13	107416_ _ _	Color Anodized/Color Accent Universal Axle Plate with Hardware Kit, Depth Adjustable Rear Frame	1
1, 2b, 6, 7, 10a, 10c - 10e, 10g, 11a, 11b, 11d - 11f, 12, 13	114608_ _ _	Color Anodized/Color Accent Universal Axle Plate with Hardware Kit, Depth Adjustable Rear Frame, Heavy Duty	1
1, 6, 7, 10a, 10b, 11a-11e, 13	107419	Universal Axle Plate Hardware Kit (One Side) <i>Contains hardware for curved, titanium and depth adjustable rear frames</i>	1
1, 6, 7, 10a, 10c - 10f, 11a - 11f, 12, 13	114609	Universal Axle Plate Hardware Kit, Heavy Duty (One Side) <i>Contains hardware for curved, titanium and depth adjustable rear frames with Heavy Duty Option</i>	1
2a, 3-5, 8b	117322	Universal Axle Plate Assembly 2.5", Black Anodized <i>Used with Quick Release and Fixed Axles</i>	1
2b, 3-5, 8b	117323_ _ _	Color Anodized/Color Accent Universal Axle Plate Assembly 2.5" <i>Used with Quick Release and Fixed Axles</i>	1
3-5, 8b	117325	Axle Receiver Assembly 2.5" <i>Used with Quick Release and Fixed Axles</i>	1
1	000056	Saddle 1 1/8" <i>Used with Transit or Swing Away Arm Bracket</i>	6
2a	107417	Universal Axle Plate, Black Anodized with Instruction	1
2b	107418_ _ _	Color Anodized/Color Accent Universal Axle Plate with Instruction	1
3	004444	Axle Plate Index Washer	1
4	004445	Axle Plate Flat Washer	1
5	100514	Axle Receiver Nut M18 x D12 x 8mm	2
6	100658	M6 Nylock Nut Blk Zn	6
7	100746	M6 Flat Washer Blk Zn	6

Item Number	Part Number	Description	Quantity
8b	114480	Axle Receiver, 2.5" Length <i>Used with Quick Release and Fixed Axles</i>	1
9	100763	Washer #12 Thin	10
10a	100674	M6 x 45 CLS10.9 DIN931 HHCS ZC	6
10b	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	6
10c	101826	M6 x 50 HHCS Blk Zn	6
10d	100676	M6 x 55 CLS10.9 DIN931 HHCS ZC	6
10e	100677	M6 x 60 CLS10.9 DIN931 HHCS ZC	6
10f	100679	M6 x 65 CLS 8.8 DIN931 HHCS ZC	6
10g	100680	M6 x 70 CLS 8.8 DIN931 HHCS ZC	6
11a	101826	M6 x 50 HHCS Blk Zn	6
11b	100677	M6 x 60 CLS10.9 DIN931 HHCS ZC	6
11c	100679	M6 x 65 CLS 8.8 DIN931 HHCS ZC	6
11d	100676	M6 x 55 CLS10.9 DIN931 HHCS ZC	6
11e	100674	M6 x 45 CLS10.9 DIN931 HHCS ZC	6
11f	100680	M6 x 70 CLS 8.8 DIN931 HHCS ZC	6
12	111543	Reinforcement Plate, Universal Axle Plate <i>Used with Heavy Duty on chairs starting with serial numbers C40043964, C50099247 or CVX0099946</i>	2
13	100881	1" Washer Plastic <i>If Height Adjustable Flip Back T-Arms are present in Axle Plate position 3, use qty 12. If Height Adjustable Flip Back T-Arms are present in Axle Plate position 2, use qty 15. If Height Adjustable Flip Back T-Arms are present in Axle Plate position 4, use qty 18.</i>	18
2a, 3-5, 8a	107412	Universal Axle Plate Assembly 1/2", Black Anodized <i>Only Used with Quad Quick Release Axles</i>	1
2b, 3-5, 8a	107424_ _ _	Color Anodized/Color Accent Universal Axle Plate Assembly 1/2" <i>Only Used with Quad Quick Release Axles</i>	1
3-5, 8a	500504	Axle Receiver Assembly 1/2" <i>Only Used with Quad Quick Release Axles</i>	1
8a	100756	Axle Receiver 1/2" Diameter <i>Only Used with Quad Quick Release Axles</i>	1

Color Anodizing Samples

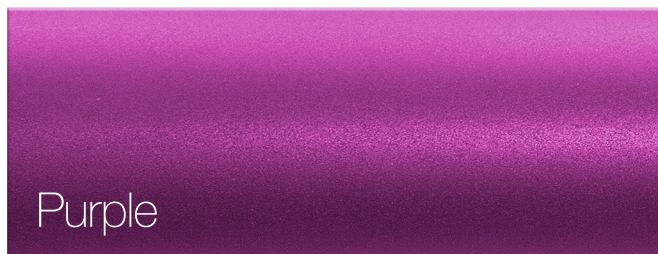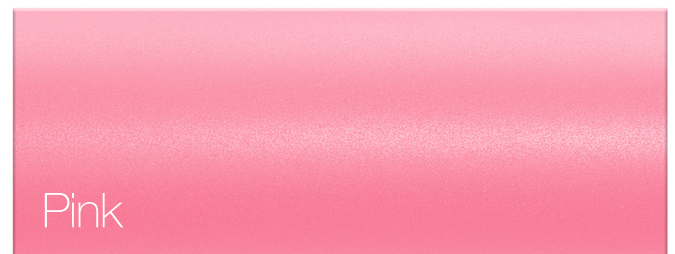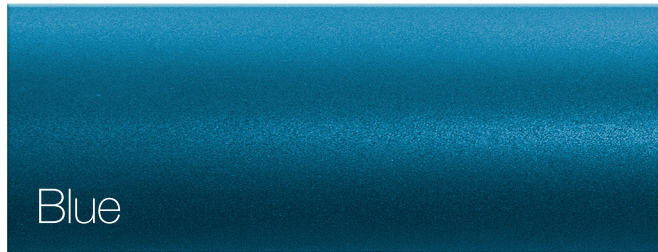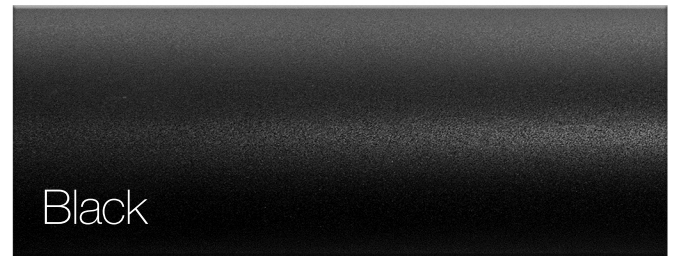

Wheel Size / Tire Profile / Rear Seat Height

Select Rear Seat Height in row corresponding to Wheel Size and Tire Profile. 14" Seat Depth not available with 24", 25" or 26" Rear Wheels.

☐ Requires curved axle plate ☐ Any axle plate can be used (Curved, Standard, Amputee, One Arm Drive)

Wheel Size	Tire Profile	Rear Seat Height									
20"	Low	☐ 11.5	☐ 12 ¹	☐ 12.5	☐ 13 ¹	☐ 13.5	☐ 14	☐ 14.5	☐ 15	☐ 15.5	☐ 16
	Full	☐ 12	☐ 12.5 ¹	☐ 13	☐ 13.5 ¹	☐ 14	☐ 14.5	☐ 15	☐ 15.5	☐ 16	☐ 16.5
22"	Low	☐ 12.5	☐ 13 ¹	☐ 13.5	☐ 14 ¹	☐ 14.5	☐ 15	☐ 15.5	☐ 16	☐ 16.5	☐ 17
	Full	☐ 13	☐ 13.5 ¹	☐ 14	☐ 14.5 ¹	☐ 15	☐ 15.5	☐ 16	☐ 16.5	☐ 17	☐ 17.5
24"	Low	☐ 13.5	☐ 14 ¹	☐ 14.5	☐ 15 ¹	☐ 15.5	☐ 16	☐ 16.5	☐ 17	☐ 17.5	☐ 18
	Full	☐ 14	☐ 14.5 ¹	☐ 15	☐ 15.5 ¹	☐ 16	☐ 16.5	☐ 17	☐ 17.5	☐ 18	☐ 18.5
25"	Low	☐ 14	☐ 14.5 ¹	☐ 15	☐ 15.5 ¹	☐ 16	☐ 16.5	☐ 17	☐ 17.5	☐ 18	☐ 18.5
26"	Low	☐ 14.5	☐ 15 ¹	☐ 15.5	☐ 16 ¹	☐ 16.5	☐ 17	☐ 17.5	☐ 18	☐ 18.5	☐ 19
	Full	☐ 15	☐ 15.5 ¹	☐ 16	☐ 16.5 ¹	☐ 17	☐ 17.5	☐ 18	☐ 18.5	☐ 19	☐ 19.5

¹ Not available with Amputee Axle Plate

CATALYST 5 CURVED REAR FRAME		DEPTH ADJUSTABLE REAR FRAME	
AXLE PLATE, FRAME	100674	AXLE PLATE, FRAME	100681
AXLE PLATE, FRAME, ANTI-TIP	101826	AXLE PLATE, FRAME, ANTI-TIP	100674
AXLE PLATE, FRAME, TRANSIT	101826	AXLE PLATE, FRAME, TRANSIT	101826
AXLE PLATE, FRAME, SWING AWAY	101826	AXLE PLATE, FRAME, SWING AWAY	101826
AXLE PLATE, FRAME, ANTI-TIP, TRANSIT	100677	AXLE PLATE, FRAME, ANTI-TIP, TRANSIT	100676
AXLE PLATE, FRAME, SWING AWAY, TRANSIT	100677	AXLE PLATE, FRAME, SWING AWAY, TRANSIT	100676
AXLE PLATE, FRAME, SWING AWAY, ANTI-TIP	100677	AXLE PLATE, FRAME, SWING AWAY, ANTI-TIP	100676
AXLE PLATE, FRAME, SWING AWAY, TRANSIT AND ANTI-TIP	100679	AXLE PLATE, FRAME, SWING AWAY, TRANSIT AND ANTI-TIP	100679