


ISO Towers

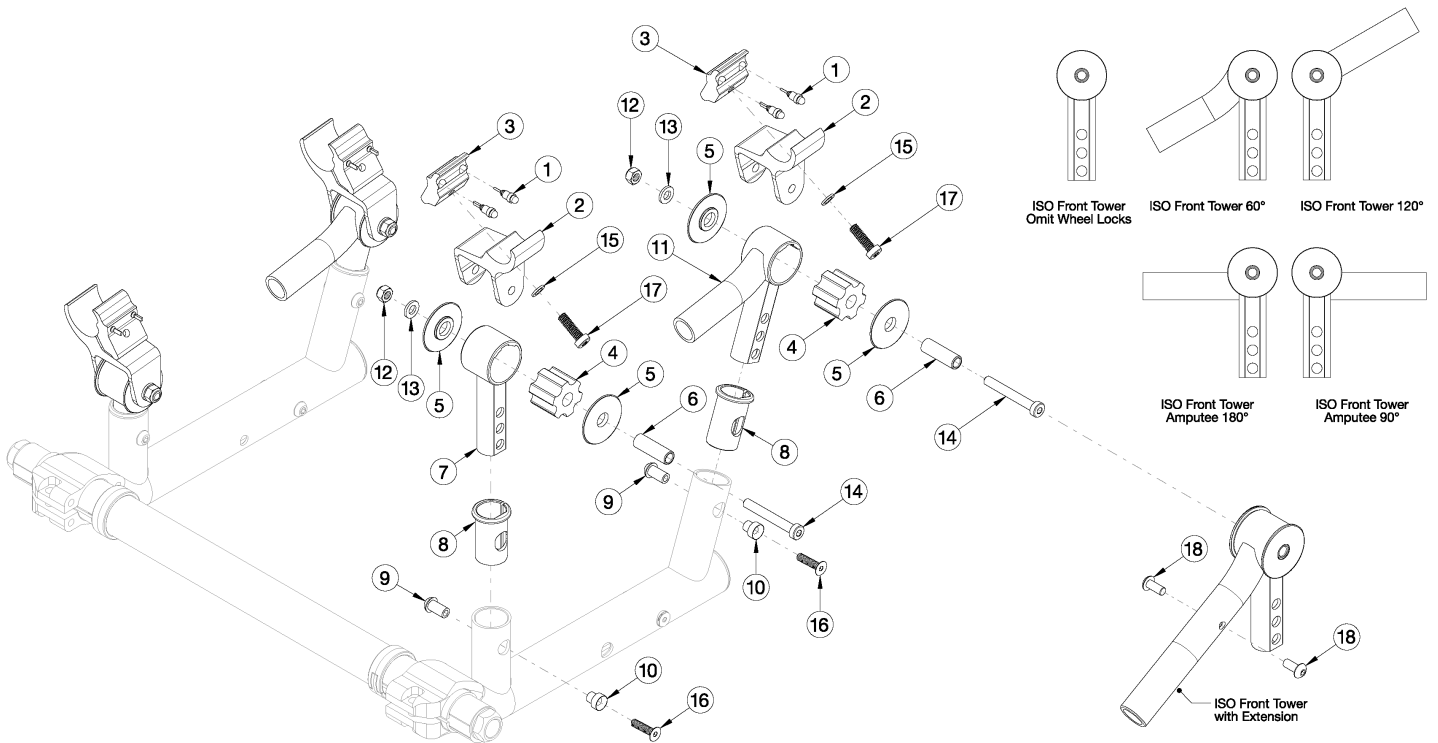

ISO Towers are modified per order and must be ordered through Customer Service. Serial number required upon order. <p>Customer Service reference DCN0511

Item Number	Part Number	Description	Quantity	Retail
1-3, 15, 17	108077	Rear ISO Clamp Assembly with Hardware Kit	1	\$28.69
2, 3, 15, 17	108078	Front ISO Clamp Assembly with Hardware Kit	1	\$23.59
1, 3	108076	ISO Clamp Inside with Button	2	\$13.79
2	003850	ISO Clamp Outside	4	\$12.84
3	003851	ISO Clamp Inside	4	\$11.24
4a	003859	Polymer Bushing 50 Durometer (Red)	4	\$5.66
4b	003860	Polymer Bushing 60 Durometer (Blue)	4	\$5.66
4c	004544	Polymer Bushing 70 Durometer (Black)	4	\$5.66
4d	004545	Polymer Bushing 80 Durometer (Green)	4	\$5.66
5	003861	Iso Strut Washer	8	\$5.01
6	003863	ISO Barrel	4	\$7.56
7	108183	Rear ISO Tower	2	\$18.36

Item Number	Part Number	Description	Quantity	Retail
8a	003867	Vertical Upright Bushing With Flat, Black	4	\$2.04
8b	108060_ _ _	Vertical Upright Bushing, Highlight Color	4	\$2.04
9	003869	M5 Barrel Nut	4	\$18.21
10	003870	Barrel, M5 FHCS	4	\$8.67
11a	108187	Front ISO Tower 120°, Right	1	\$22.44
11b	110040	Front ISO Tower with Extension 120°, Right	1	\$93.88
11c	108189	Front ISO Tower 120°, Left	1	\$22.44
11d	110041	Front ISO Tower with Extension 120°, Left	1	\$93.88
11e	108188	Front ISO Tower 60°, Right	1	\$22.44
11f	110042	Front ISO Tower with Extension 60°, Right	1	\$93.88
11g	108190	Front ISO Tower 60°, Left	1	\$22.44
11h	110043	Front ISO Tower with Extension 60°, Left	1	\$93.88
11i	108184	Front ISO Tower, Omit Wheel Locks	2	\$18.36
11j	108185	Front ISO Tower, Amputee 90°	2	\$28.34
11k	110038	Front ISO Tower with Extension, Amputee 90°	2	\$99.78
11l	108186	Front ISO Tower, Amputee 180°	2	\$28.34
11m	110039	Front ISO Tower with Extension, Amputee 180°	2	\$99.78
12	100658	M6 Nylock Nut Blk Zn	4	\$1.05
13	100746	M6 Flat Washer Blk Zn	4	\$1.58
14	101123	M6 x 45 Low SHCS BLK ZC	4	\$1.58
15	102326	Flat Washer ID .254" x OD .396" Thickness .047" Zinc	4	\$0.53
16	103322	M5 x 20 FHCS Blk Zc	4	\$1.02
17	103396	M6 x 20 LHCS Blk Zc	4	\$1.53
18	102621	M5 x 8 BHCS BLK ZC	2	\$0.53