


(Discontinued 1) Ethos Backrest Mount and Hardware

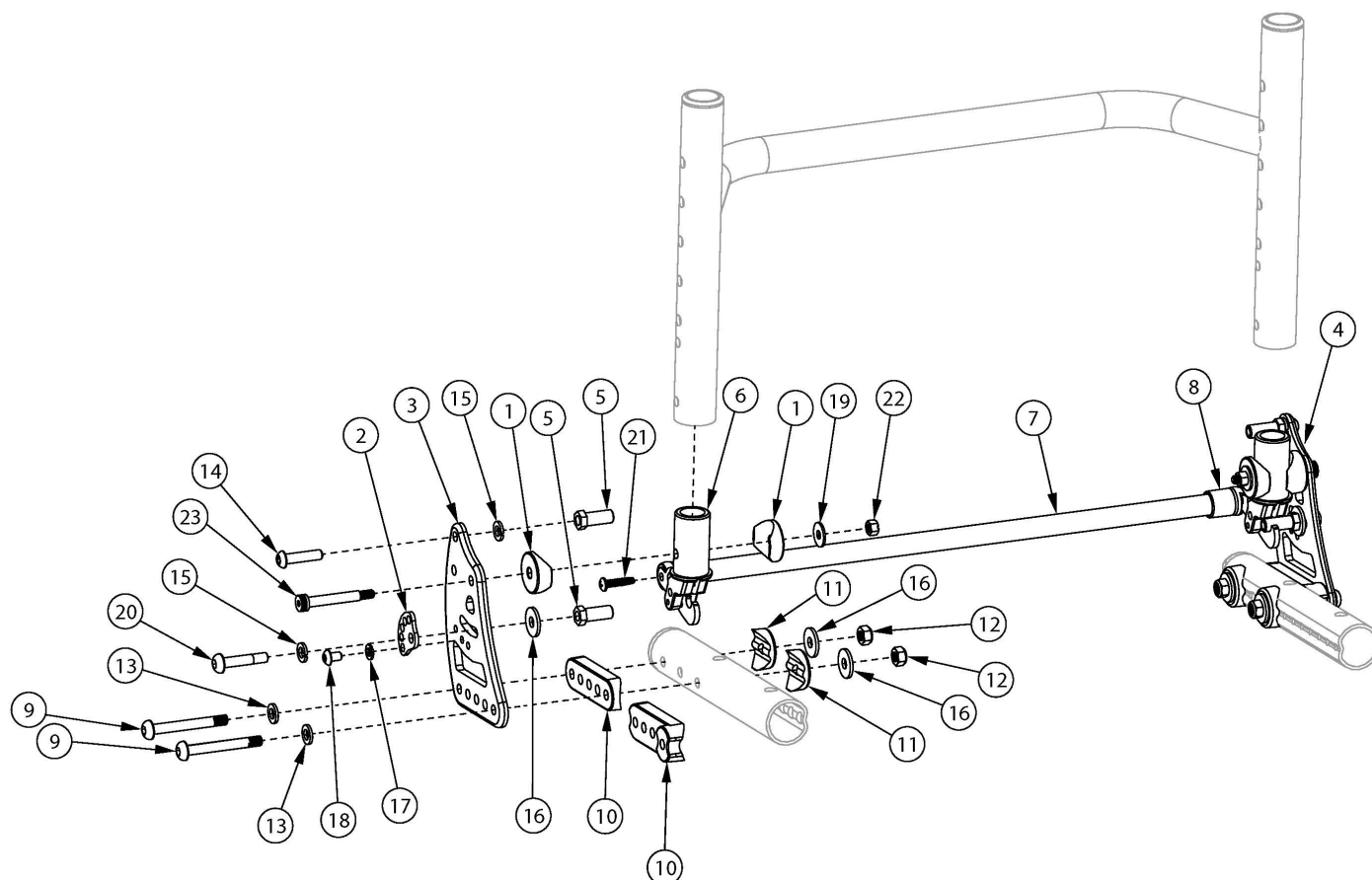





If chair has Rigid Locking Fold Down Back, please see Ethos Backrest Mount and Hardware page.

Item Number	Part Number	Description	Quantity
1a	000055	Saddle 1" <i>If swing away arms are not present, use quantity 4. Use quantity 2 if swing away arms are present.</i>	4
1b	004117	Saddle M6 ID x 1 OD x .25 T <i>If swing away arms are present, use quantity 2 of 000055 and quantity 2 of 004117</i>	2
2	001762	Arc Adjustment Cam	2
3a	003711	Back Bracket Right, Black Anodized	1
3b	003711_ _ _	Color Anodized Back Bracket, Right	1
3c	109296	Back Bracket with Transit, Black, Rigid	1
4a	003712	Back Bracket Left, Black Anodized	1
4b	003712_ _ _	Color Anodized Back Bracket, Left	1

Item Number	Part Number	Description	Quantity
4c	109296	Back Bracket with Transit, Black, Rigid	1
5	003713	Strike Hex	4
7a	003739	Backrest Release Bar 12" Wide, Black Anodized	1
7b	003740	Backrest Release Bar 13" Wide, Black Anodized	1
7c	003741	Backrest Release Bar 14" Wide, Black Anodized	1
7d	003742	Backrest Release Bar 15" Wide, Black Anodized	1
7e	003743	Backrest Release Bar 16" Wide, Black Anodized	1
7f	003744	Backrest Release Bar 17" Wide, Black Anodized	1
7h	003746	Backrest Release Bar 19" Wide, Black Anodized	1
7i	003747	Backrest Release Bar 20" Wide, Black Anodized	1
7j	109642_ _ _	Ethos Backrest Release Bar 12" Wide, Painted	1
7k	109643_ _ _	Ethos Backrest Release Bar 13" Wide, Painted	1
7l	109644_ _ _	Ethos Backrest Release Bar 14" Wide, Painted	1
7m	109645_ _ _	Ethos Backrest Release Bar 15" Wide, Painted	1
7n	109646_ _ _	Ethos Backrest Release Bar 16" Wide, Painted	1
7o	109647_ _ _	Ethos Backrest Release Bar 17" Wide, Painted	1
7p	109648_ _ _	Ethos Backrest Release Bar 18" Wide, Painted	1
7q	109649_ _ _	Ethos Backrest Release Bar 19" Wide, Painted	1
7r	109650_ _ _	Ethos Backrest Release Bar 20" Wide, Painted	1
9a	003846	M6 x 50 BHCS, Custom, Blk Zc <i>Used if Swing Away Arms are not present</i>	4
9b	003232	M6 x 55 BHCS Blk Zc <i>Used if Swing Away Arms are present</i>	4
10a	003848	Back Bracket Saddle 1 1/8" <i>Used if Swing Away Arms are not present</i>	2
11	003849	Saddle For Dimple 1 1/8"	4
12	100658	M6 Nylock Nut Blk Zn	4
13	100746	M6 Flat Washer Blk Zn	4
14	101593	M6 x 25 BTN SHCS BLKZC	2

Item Number	Part Number	Description	Quantity
15	101835	M6 Lock Washer Blk Zn	4
16	101853	M6 Washer Blk Zn	2
17	102614	M5 Split Lock Washer BLK ZC	2
18	102621	M5 x 8 BHCS BLK ZC	2
19	103022	M5 Washer, Blk Zn	2
20	103227	M6 x 30 Long Shoulder BHCS BZ	2
21	103279	#8-16 Tri-Lobular Plastic Screw BZ	2
22	103283	M5 Top Lock Nut BZ	2
23a	103284	M6 x 35 Shoulder Bolt BZ <i>Used if Swing Away Arms are not present</i>	2
23b	103397	M6 x 40 Shoulder Screw BLK ZC <i>Used if Swing Away Arms are present</i>	2

Color Anodizing Samples

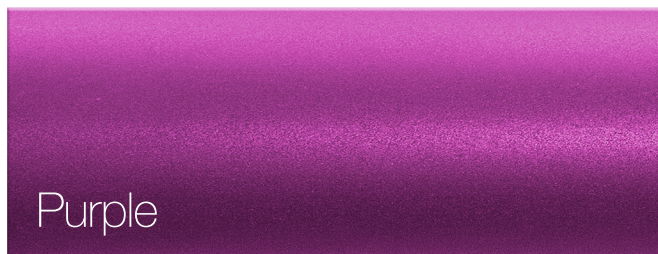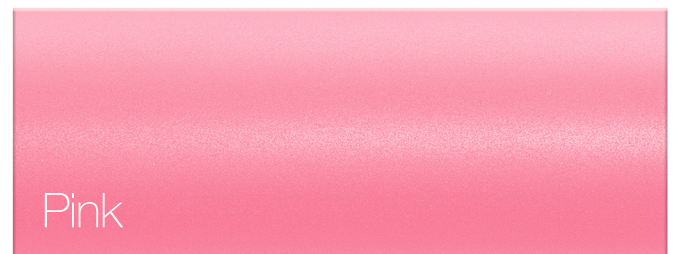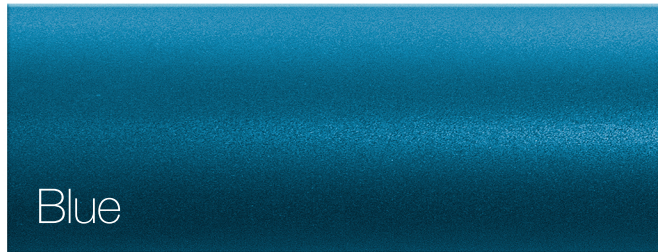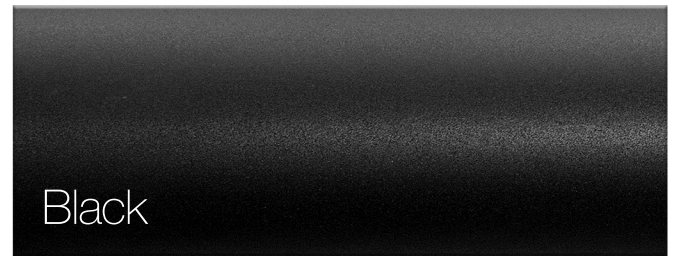

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer