


Catalyst 5Vx Hemi Wheel Lock

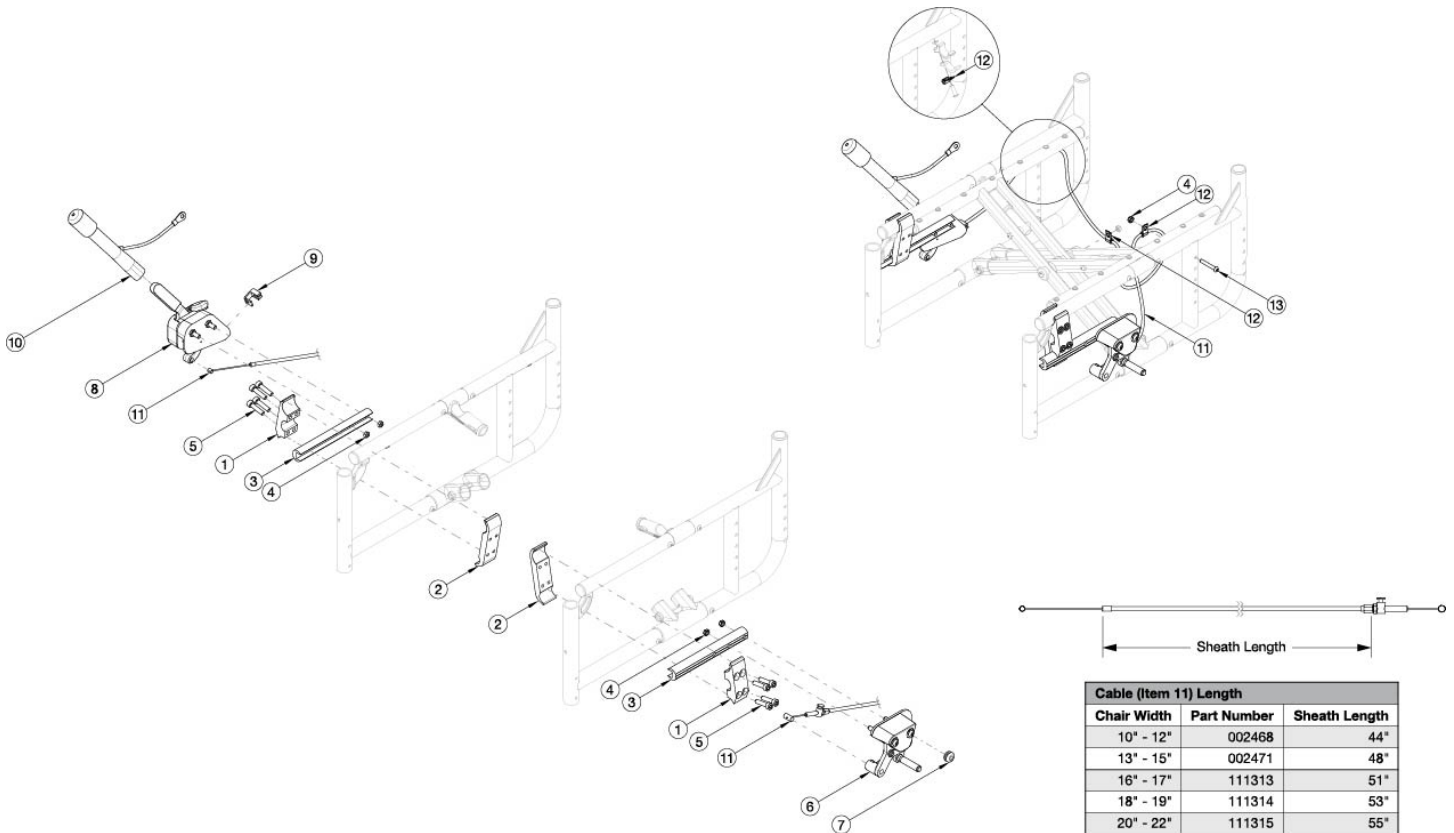

Cable (Item 11) Length		
Chair Width	Part Number	Sheath Length
10" - 12"	002468	44"
13" - 15"	002471	48"
16" - 17"	111313	51"
18" - 19"	111314	53"
20" - 22"	111315	55"

Item Number	Part Number	Description	Quantity
1-5, 6a, 7, 8a, 9, 10, 11a, 12, 13	122930	Hemi Wheel Lock Assembly, Width 10-12, Right Side Mounted, Catalyst 5Vx	1
1-5, 6a, 7, 8a, 9, 10, 11b, 12, 13	122932	Hemi Wheel Lock Assembly, Width 13-19, Right Side Mounted, Catalyst 5Vx	1
1-5, 6a, 7, 8a, 9, 10, 11c, 12, 13	122934	Hemi Wheel Lock Assembly, Width 20-22, Right Side Mounted, Catalyst 5Vx	1
1-5, 6b, 7, 8b, 9, 10, 11a, 12, 13	122931	Hemi Wheel Lock Assembly, Width 10-12, Left Side Mounted, Catalyst 5Vx	1
1-5, 6b, 7, 8b, 9, 10, 11b, 12, 13	122933	Hemi Wheel Lock Assembly, Width 13-19, Left Side Mounted, Catalyst 5Vx	1
1-5, 6b, 7, 8b, 9, 10, 11c, 12, 13	122935	Hemi Wheel Lock Assembly, Width 20-22, Left Side Mounted, Catalyst 5Vx	1
1	000143	Wheel Lock Clamp Extended Top CAT	1
2	000144	Wheel Lock Clamp Extended Bottom CAT	1
3	001321	Wheel Lock Mount Bar Extended	1

Item Number	Part Number	Description	Quantity
4	100658	M6 Nylock Nut Blk Zn	5
5	100780	M6 x 25 SHCS ZC	4
6a	111591	Wheel Lock Follower Assembly without Mount, Left	1
6b	111592	Wheel Lock Follower Assembly without Mount, Right	1
7	115743	Wheel Lock Retaining Grommet	1
8a	111593	Hemi Wheel Lock, Master Assembly, Right without Mount	1
8b	111594	Hemi Wheel Lock, Master Assembly, Left without Mount	1
9	115742	Hemi Wheel Lock Cable Retaining Clip	1
10	200369	Wheel Lock Extension	1
11a	122726	Drum Brake / Hemi Wheel Lock Short Cable <i>Used with Seat Widths 10 - 12"</i>	1
11b	122727	Drum Brake / Hemi Wheel Lock Medium Cable <i>Used with Seat Widths 13 - 19"</i>	1
11c	122728	Drum Brake / Hemi Wheel Lock Long Cable <i>Used with Seat Widths 20 - 22"</i>	1
12	001239	P-Clip 3/16"	3
13	100797	M6 x 40 BHCS BLK ZC	1