


Catalyst Heavy Duty Caster Forks and Stems

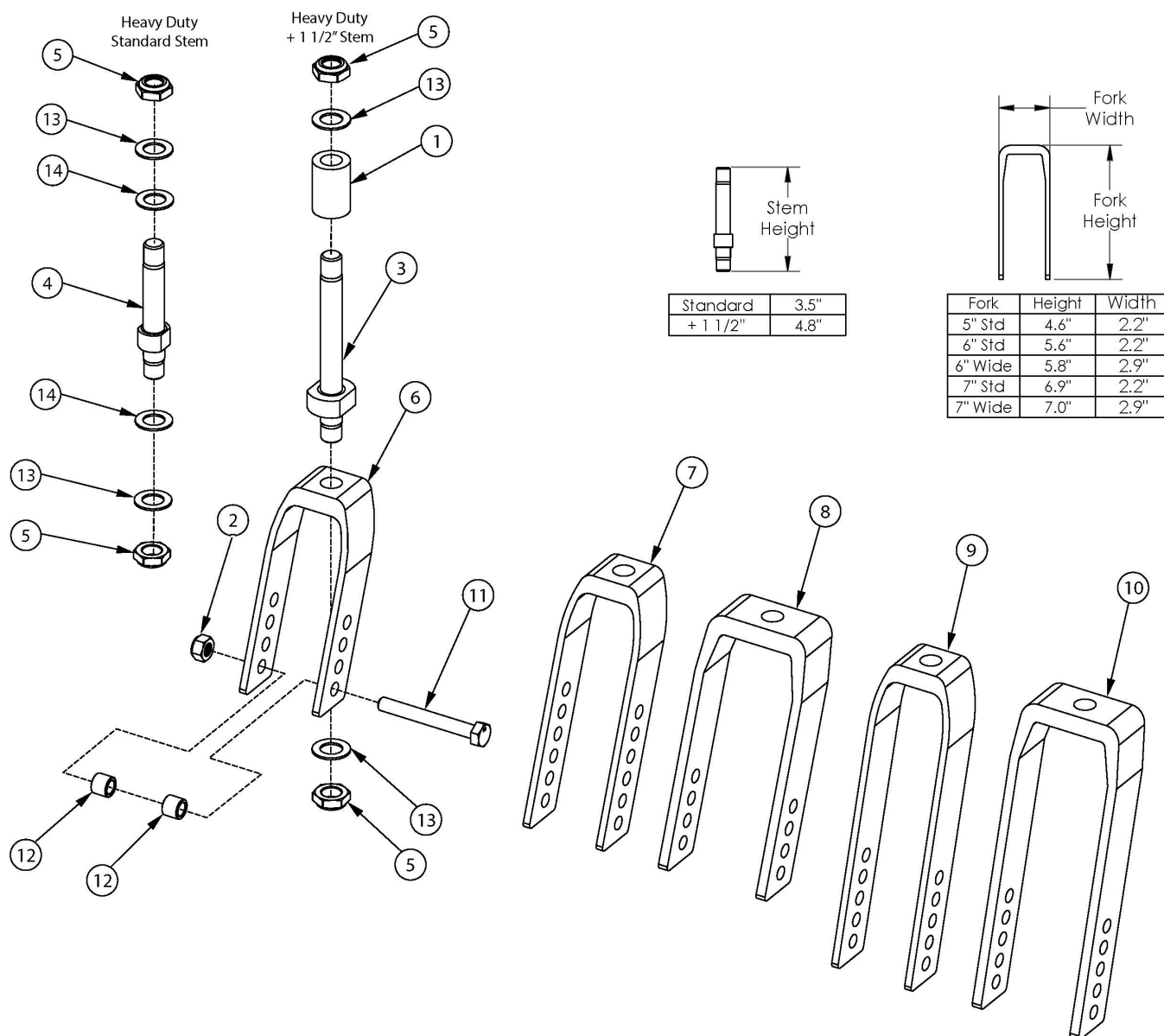

This Heavy Duty fork stem design effective starting with serial numbers C40043964, C50099247, CVX0099946 with the 350 lbs Weight Capacity option. If changing to 2" casters, wide forks and hardware needed.

Item Number	Part Number	Description	Quantity
4, 5, 6a, 13, 14	114577	Heavy Duty Fork Assembly, Black, 5" Fork, Std Stem, Catalyst	1
4, 5, 6b, 13, 14	114577_ _ _	Heavy Duty Fork Assembly, Color Anodized/Color Accent, 5" Fork, Std Stem, Catalyst	1
4, 5, 7a, 13, 14	114578	Heavy Duty Fork Assembly, Black, 6" Fork, Std Stem, Catalyst	1
4, 5, 7b, 13, 14	114578_ _ _	Heavy Duty Fork Assembly, Color Anodized/Color Accent, 6" Fork, Std Stem, Catalyst	1

Item Number	Part Number	Description	Quantity
4, 5, 8, 13, 14	114579	Heavy Duty Fork Assembly, Black, 6" Wide Fork, Std Stem, Catalyst <i>Only used with 2" wide casters</i>	1
4, 5, 9a, 13, 14	114580	Heavy Duty Fork Assembly, Black, 7" Fork, Std Stem, Catalyst	1
4, 5, 9b, 13, 14	114580_ _ _	Heavy Duty Fork Assembly, Color Anodized/Color Accent, 7" Fork, Std Stem, Catalyst	1
4, 5, 10, 13, 14	114581	Heavy Duty Fork Assembly, Black, 7" Wide Fork, Std Stem, Catalyst <i>Only used with 2" wide casters</i>	1
1, 3, 5, 6a, 13	114582	Heavy Duty Fork Assembly, Black, 5" Fork, + 1 1/2" Stem, Catalyst	1
1, 3, 5, 6b, 13	114582_ _ _	Heavy Duty Fork Assembly, Color Anodized/Color Accent, 5" Fork, + 1 1/2" Stem, Catalyst	1
1, 3, 5, 7a, 13	114583	Heavy Duty Fork Assembly, Black, 6" Fork, + 1 1/2" Stem, Catalyst	1
1, 3, 5, 7b, 13	114583_ _ _	Heavy Duty Fork Assembly, Color Anodized/Color Accent, 6" Fork, + 1 1/2" Stem, Catalyst	1
1, 3, 5, 8, 13	114584	Heavy Duty Fork Assembly, Black, 6" Wide Fork, + 1 1/2" Stem, Catalyst <i>Only used with 2" wide casters</i>	1
1, 3, 5, 9a, 13	114585	Heavy Duty Fork Assembly, Black, 7" Fork, + 1 1/2" Stem, Catalyst	1
1, 3, 5, 9b, 13	114585_ _ _	Heavy Duty Fork Assembly, Color Anodized/Color Accent, 7" Fork, + 1 1/2" Stem, Catalyst	1
1, 3, 5, 10, 13	114586	Heavy Duty Fork Assembly, Black, 7" Wide Fork, + 1 1/2" Stem, Catalyst <i>Only used with 2" wide casters</i>	1
2, 11a, 12a	502653	Caster Attachment Hardware Kit CAT/FCS	1
2, 11b, 12b	502654	Caster Attachment Hardware Kit for 6" and 7" Wide Fork, CAT/FCS <i>Only used with 2" wide casters</i>	1
1	002476	Spacer, 3/4" OD x 1/2" ID x 1.23	1
2	100659	M8 Nylock Nut Blk Zn	1
4, 5, 13, 14	114587	Heavy Duty Caster Stem Standard, Catalyst	1
1, 3, 5, 13	114588	Heavy Duty Caster Stem + 1 1/2", Catalyst	1
5	100529	M12 x 1.25mm Lock Nut Black	2
6a, 5	114589	Heavy Duty Caster Fork, Black, 5" Catalyst	1
6b, 5	114589_ _ _	Heavy Duty Caster Fork, Color Anodized/Color Accent, 5" Catalyst	1
7a, 5	114590	Heavy Duty Caster Fork, Black, 6" Catalyst	1
7b, 5	114590_ _ _	Heavy Duty Caster Fork, Color Anodized/Color Accent, 6" Catalyst	1

Item Number	Part Number	Description	Quantity
8, 5	114591	Heavy Duty Caster Fork, Black, 6" Wide Catalyst <i>Only used with 2" wide casters</i>	1
9a, 5	114592	Heavy Duty Caster Fork, Black, 7" Catalyst	1
9b, 5	114592_ _ _	Heavy Duty Caster Fork, Color Anodized/Color Accent, 7" Catalyst	1
10, 5	114593	Heavy Duty Caster Fork, Black, 7" Wide Catalyst <i>Only used with 2" wide casters</i>	1
11a	101829	M8 x 65 HHCS Blk Zn	1
11b	101831	M8 x 80 HHCS Blk Zn <i>Only used with 2" wide casters</i>	1
12a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer <i>Used with all casters except 6x2 and 8x2</i>	2
12b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Only used with 6" x 2" casters</i>	2
13	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	2
14	002593	Caster Stem Washer Heavy Duty <i>Only used with standard stem</i>	2

Color Anodizing Samples

