



Catalyst 5 Standard Caster Housing for Fixed Front Frame

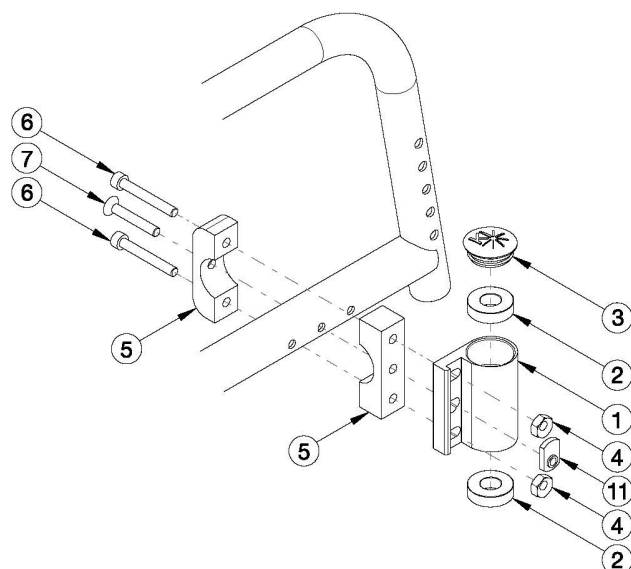

Fixed Front without Transit

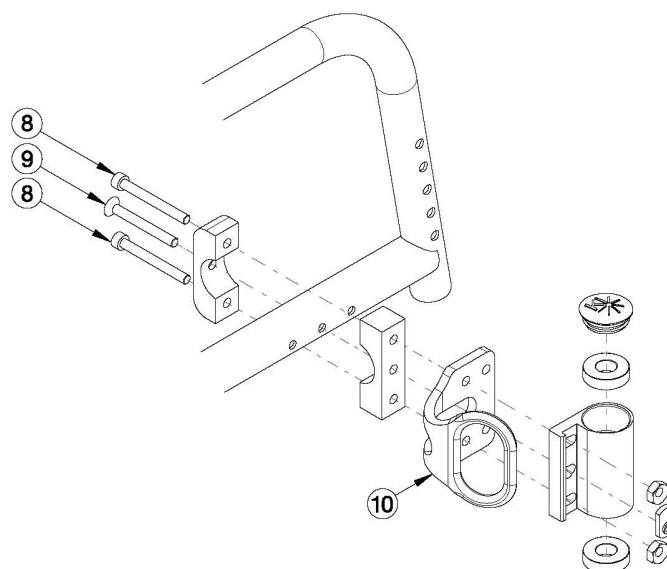

Fixed Front with Transit

This design effective July 12th, 2021 Please Note: The shown orientation of the caster housing above is in the "Forward" position.

Item Number	Part Number	Description	Quantity
1a, 2-7, 11	114444	Caster Housing Assembly, Fixed Front, Black, Left, Catalyst 5	1
1b, 2-7, 11	114444_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, Left, Catalyst 5	1
1c, 2-7, 11	114445	Caster Housing Assembly, Fixed Front, Black, Right, Catalyst 5	1
1d, 2-7, 11	114445_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, Right, Catalyst 5	1
1a, 1c, 2-7, 11	114446	Caster Housing Assembly, Fixed Front, Black, PAIR, Catalyst 5	1
1b, 1d, 2-7, 11	114446_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, PAIR, Catalyst 5	1
1a, 2-5, 8, 9, 11	114447	Caster Housing Assembly, Fixed Front with Transit, Black, Left, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1
1b, 2-5, 8, 9, 11	114447_ _ _	Color Anodized Caster Housing Assembly, Fixed Front with Transit, Left, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1
1c, 2-5, 8, 9, 11	114448	Caster Housing Assembly, Fixed Front with Transit, Black, Right, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1
1d, 2-5, 8, 9, 11	114448_ _ _	Color Anodized Caster Housing Assembly, Fixed Front with Transit, Right, Catalyst 5	1

Item Number	Part Number	Description	Quantity
		<i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	
1a, 1c, 2-5, 8, 9, 11	114449	Caster Housing Assembly, Fixed Front with Transit, Black, PAIR, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1
1b, 1d, 2-5, 8, 9, 11	114449_ _ _	Color Anodized Caster Housing Assembly, Fixed Front with Transit, PAIR, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1
1a, 2	001429	Caster Housing Fixed Front Left, Black, Catalyst	1
1b, 2	001429_ _ _	Color Anodized Caster Housing Fixed Front, Left, Catalyst	1
1c, 2	001430	Caster Housing Fixed Front Right, Black, Catalyst	1
1d, 2	001430_ _ _	Color Anodized Caster Housing Fixed Front, Right, Catalyst	1
2	100530	Bearing M28 x 1/2" 6001RS	2
3	102005	Ki Caster Housing Cap	1
4	100745	Eccentric nut (threaded)	2
5	100528	Caster Housing Clamp	1
6	101472	M6 x 45 SHCS BLK ZC	2
7	101130	M6 x 55 FSHCS P/THD BLKZC	1
8	102473	M6 X 50 CLS12.9 SHCS BLK ZC	2
9	103068	M6 x 60mm FHCS Blk Zn	1
11	102092	M6 Weld Nut Offset	1

Color Anodizing Samples

