


# Catalyst E Vertical Axle Plate

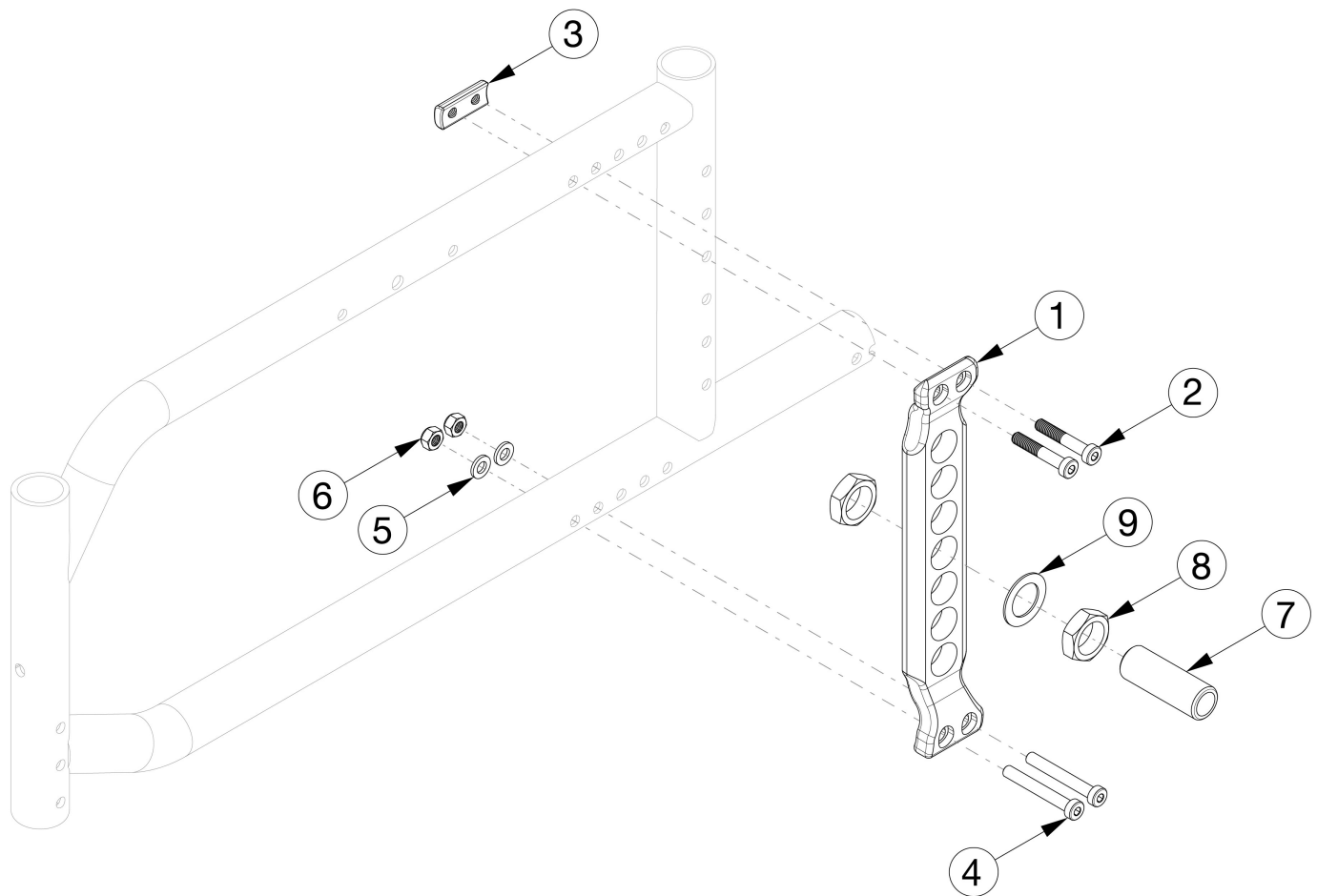

Item Number	Part Number	Description	Quantity
1-6	506127	Vertical Axle Plate Assembly, Black, 0° Camber <i>This kit is only for one side</i>	1
1, 7a-9	506132	Vertical Axle Plate, Black, with Receiver Assembly 1/2" <i>This kit is only for one side. Used with Quad Quick Release Axles.</i>	1
1, 7b-9	117102	Vertical Axle Plate, Black, with Receiver Assembly 2.5" <i>This kit is only for one side. Used with Quick Release and Fixed Axles.</i>	1
7a-9	114172	Axle Receiver Assembly, Catalyst 5Vx <i>This kit is only for one side. Used with Quad Quick Release Axles.</i>	1
7b-9	117101	Axle Receiver Assembly, Catalyst 5Vx & E <i>This kit is only for one side. Used with Quick Release and Fixed Axles.</i>	1
1	003519INST	VX Axle Plate Black with Instruction	1

Item Number	Part Number	Description	Quantity
2	103370	M6 x 35 Low SHCS BLK ZC w/Patch	2
3	003520	Threaded Axle Nut Plate	1
4	101123	M6 x 45 Low SHCS BLK ZC	2
5	100746	M6 Flat Washer Blk Zn	2
6	100658	M6 Nylock Nut Blk Zn	2
7a	100756	Axle Receiver 1/2" Diameter <i>Used with Quad Quick Release Axles</i>	1
7b	114480	Axle Receiver, 2.5" Length <i>Used with Quick Release and Fixed Axles</i>	1
8	100514	Axle Receiver Nut M18 x D12 x 8mm	2
9	100583	Axle Plate Washer 18mm	1