


# Flip Rear Anti-Tip

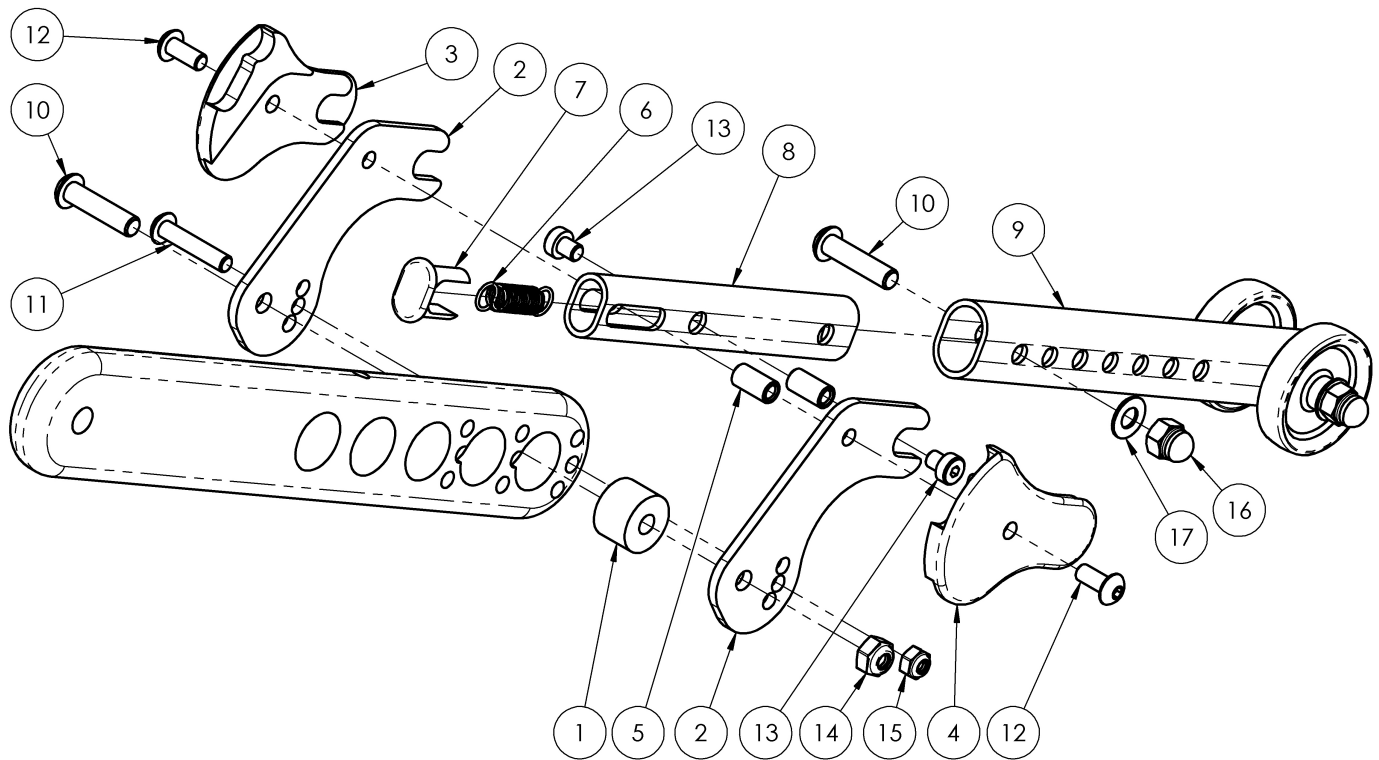

Item Number	Part Number	Description	Quantity
1-7, 8a, 9a, 10-17	505580	Anti-Tip Kit, 12" FLIP EACH	1
1-7, 8a, 9a, 10-17	502886	Anti-Tip Kit, 12" FLIP PAIR	1
1-7, 8a, 9b, 10-17	505581	Anti-Tip Kit, 16" FLIP EACH	1
1-7, 8a, 9b, 10-17	502887	Anti-Tip Kit, 16" FLIP PAIR	1
1-7, 8b, 9b, 10-17	505582	Anti-Tip Kit, 20-24" FLIP EACH	1
1-7, 8b, 9b, 10-17	502888	Anti-Tip Kit, 20-24" FLIP PAIR	1
1	001906	Bushing, Anti-tip	1
2	001907	Anti-Tip Bracket	2
3	001908	Cover, Anti-Tip, R	1
4	001909	Cover, Anti-Tip, L	1
5	001910	Spring Anchor, Index, Anti-Tip	2
6	001911	Spring, Ext, .31OD x 1.16LG x .038W	1

Item Number	Part Number	Description	Quantity
7	001912	Cap, Tube, Anti-Tip	1
8a	001913	Tube, Anti-Tip, Short <i>Commonly used with 12" or 16" rear wheels</i>	1
8b	001914	Tube, Anti-Tip, Long <i>Commonly used with 20" - 24" rear wheels</i>	1
9a	001917	Extension Assembly, Anti-Tip, Short <i>Commonly used with 12" rear wheels</i>	1
9b	119675	Extension Assembly, Anti-Tip, Long <i>Includes entire lower anti-tip.</i>	1
10	101593	M6 x 25 BTN SHCS BLKZC	2
11	102616	M5 X 25 BHCS BLK ZC	1
12	102799	M5 x 12, BHCS, BZ w/Patch	2
13	102796	M5 x 6, Low SHCS, Blk Zn	2
14	100658	M6 Nylock Nut Blk Zn	1
15	100657	M5 Nylock Nut Blk Zn	1
16	100558	M6 Nyl Ins Domed Cap L/N Zc	1
17	102803	Washer Flat .250 X .500 X .027 Hx Blz	1