


Rogue2 Armrest - Height Adjustable Tall T-Arm

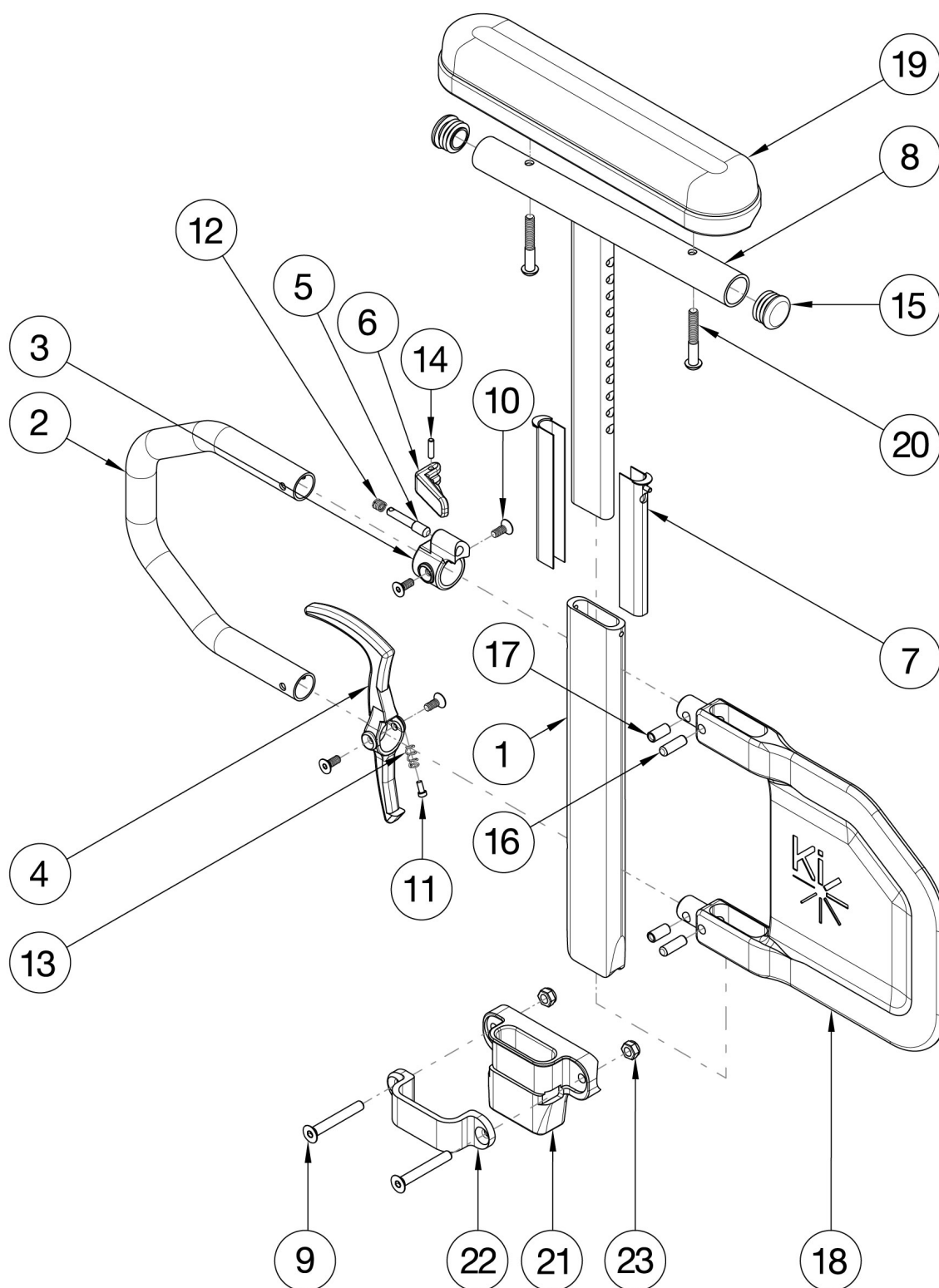

Height Adjustable Tall T-Arms cannot be retro-fitted.

Item Number	Part Number	Description	Quantity
1-7, 8a, 9-18, 19b, 20-23	116178	Height Adj Tall T-Arm Assm w/ Short Side Shield and Std Pad Desk, 1-3/8" Receiver, RG ALX/RG/LW XP	1
1-7, 8b, 9-18, 19a, 20-23	116179	Height Adj Tall T-Arm Assm w/ Short Side Shield and Std Pad Full, 1-3/8" Receiver, RG ALX/RG/LW XP	1
1-7, 8a, 9-18, 19b, 20-23	116180	Height Adj Tall T-Arm Assm w/ Short Side Shield and Std Pad Desk, 1-3/8" Receiver, RG ALX/RG/LW XP, Pair	1
1-7, 8b, 9-18, 19a, 20-23	116181	Height Adj Tall T-Arm Assm w/ Short Side Shield and Std Pad Full, 1-3/8" Receiver, RG ALX/RG/LW XP, Pair	1
1-7, 8a, 10-18, 19b, 20	116162	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Std Pad Desk	1
1-7, 8b, 10-18, 19a, 20	116164	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Std Pad Full	1
1-7, 8a, 10-18, 19b, 20	116163	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Std Pad Desk, Pair	1
1-7, 8b, 10-18, 19a, 20	116165	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Std Pad Full, Pair	1
1-7, 10-14, 16-18	114150	Height Adjustable T-Arm Short Lower Assembly, Tall <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1
1-7, 8a, 10-18	114432	Height Adjustable T-Arm Short Side Shield, Desk, Tall <i>Does not include arm pad or arm pad screws</i>	1
1-7, 8b, 10-18	114433	Height Adjustable T-Arm Short Side Shield, Full, Tall <i>Does not include arm pad or arm pad screws</i>	1
3, 5, 6, 10, 12, 14	116378	Height Adjustable Tall T-Arm Upper Latch Assembly	1
4, 10, 11, 13	116375	Height Adjustable Tall T-Arm Release Lever Assembly	1
9, 21-23	501731	Height Adj T-Arm Receiver Assembly 1-3/8", RG ALX/LW XP	1
16-18	116377	Height Adjustable Tall T-Arm Short Shield Assembly	1
1	113137	Height Adjustable T-Arm Post, Tall	1
2	113138	Height Adjustable T-Arm Transfer Tube, Tall	1
7	000090	Height Adjustable T Armrest Shim	2
8a, 15	116379	Height Adjustable Tall T Armrest Upper Desk w/ Tube End Plug	1
8b, 15	116380	Height Adjustable Tall T Armrest Upper Full w/ Tube End Plug	1
9	101130	M6 x 55 FSHCS P/THD BLKZC	2
10	103522	M5 x 0.8 x 12 FHCS CLS 10.9 Nyl Blk Zn DIN7991	4

Item Number	Part Number	Description	Quantity
11	000152	Height Adjustable T Armrest Release Spring Button	1
13	000135	Height Adjustable T Armrest Release Lever Spring	1
15	100803	Tube End Plug 7/8"	2
16	003641	.25" x .75" Dowel Pin	2
17	003457	M5 x 14mm Barrel Nut	2
19a	100500	Arm Pad Full	1
19b	100501	Arm Pad Desk	1
20	100696	M5 x 30mm PPH Capscrew	2
21	004578	Height Adjustable T-Arm Receiver 1.375"	1
22	000088	Height Adjustable T Armrest Bracket	1
23	100658	M6 Nylock Nut Blk Zn	2