


Catalyst Straight Extra Tall Backrest with Push Handle

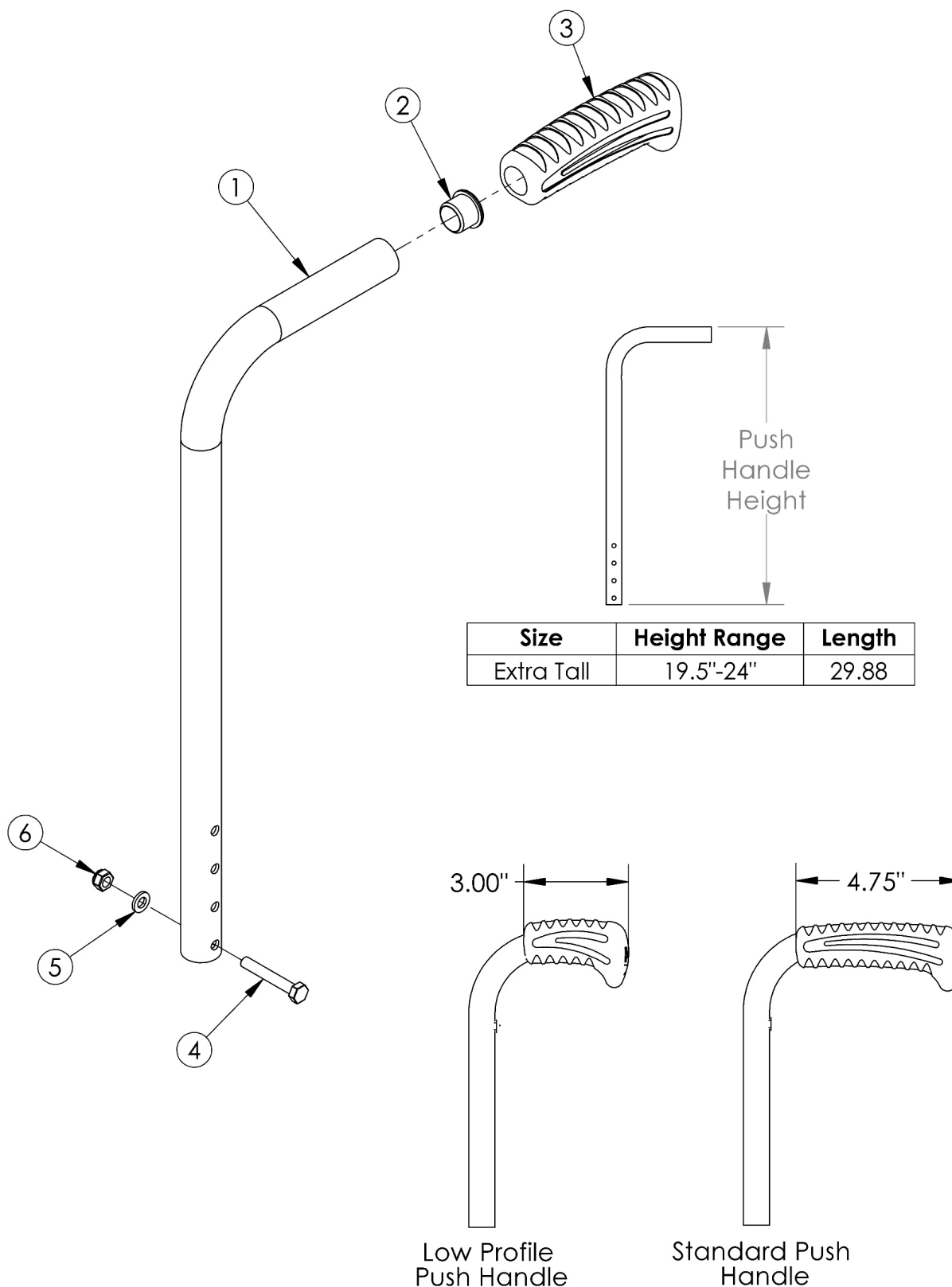

Low Profile Push Handles are not available with Heavy Duty Option If Anti-Rollback is present, see Back Tubes for Anti-Rollback System

Item Number	Part Number	Description	Quantity
1, 2, 3a	101134	Back Post Assembly Extra Tall Straight with Standard Push Handle	1
1, 2, 3b	125244	Back Post Assembly Extra Tall Straight with Low Profile Push Handle	1
2	100803	Tube End Plug 7/8"	1
3a	109433	Ergo Push Handle Grip 7/8" Standard with Instruction	1
3b	109431	Ergo Push Handle Grip 7/8" Low Profile with Instruction	1
4	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	1
5	100746	M6 Flat Washer Blk Zn	1
6	100658	M6 Nylock Nut Blk Zn	1