


# (Discontinued) Rogue Adjustable Height Backrest with Adjustable Height Rigidizer Bar

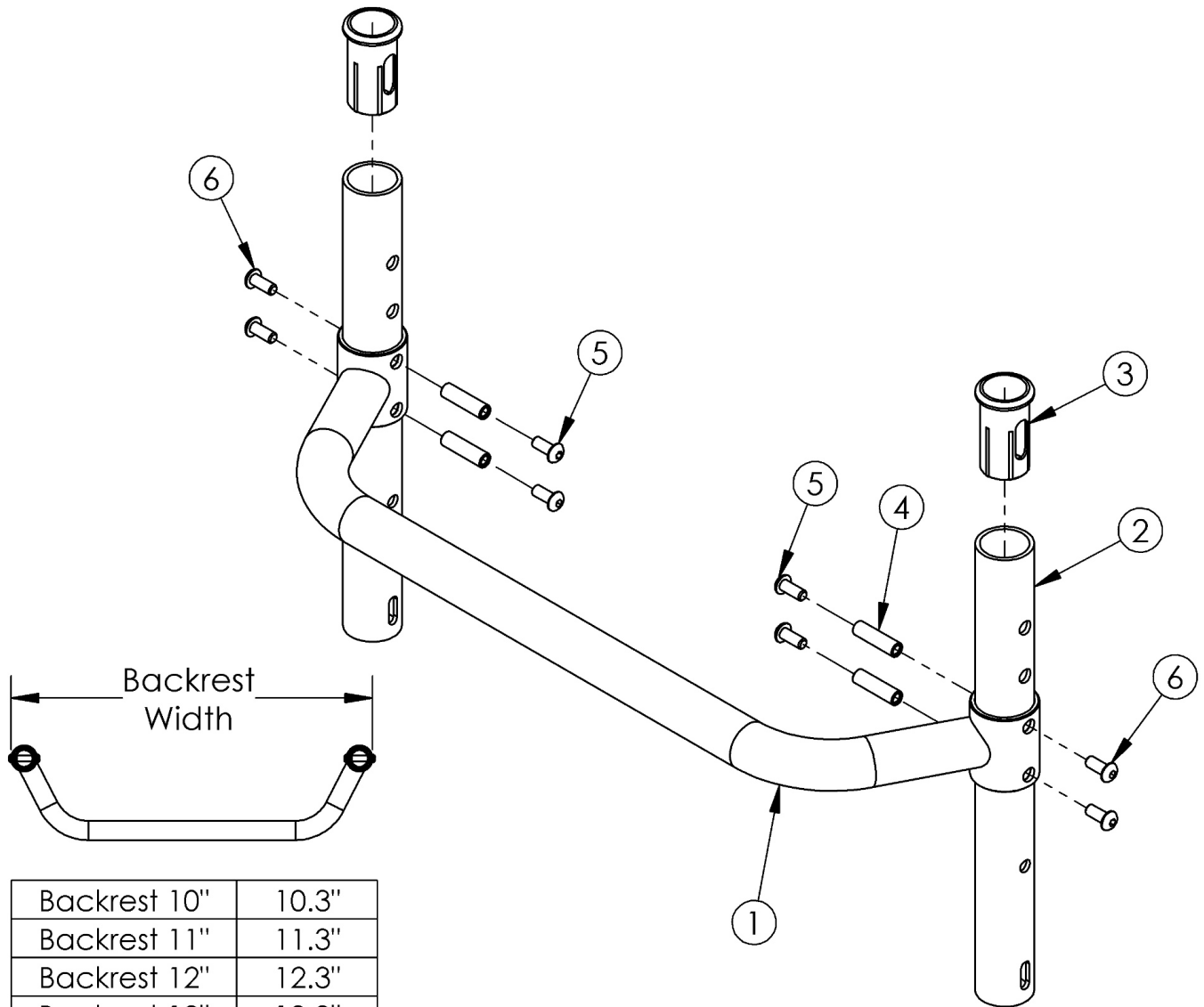

Backrest 10"	10.3"
Backrest 11"	11.3"
Backrest 12"	12.3"
Backrest 13"	13.3"
Backrest 14"	14.3"
Backrest 15"	15.3"
Backrest 16"	16.3"
Backrest 17"	17.3"
Backrest 18"	18.3"
Backrest 19"	19.3"
Backrest 20"	20.3"
Backrest 21"	21.3"
Backrest 22"	22.3"

*This Rogue Adjustable Height Backrest with Adjustable Height Rigidizer Bar is discontinued and was used on serial numbers prior to RG0016612.*

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>	<b>Retail</b>
1a	000527	Adjustable Height Rigidizer Bar 12"	1	\$82.67
1b	000528	Adjustable Height Rigidizer Bar 13"	1	\$82.67
1c	000185	Adjustable Height Rigidizer Bar 14"	1	\$82.67
1d	000186	Adjustable Height Rigidizer Bar 15"	1	\$82.67
1e	000187	Adjustable Height Rigidizer Bar 16"	1	\$82.67
1f	000188	Adjustable Height Rigidizer Bar 17"	1	\$82.67
1g	000189	Adjustable Height Rigidizer Bar 18"	1	\$82.67
1h	000190	Adjustable Height Rigidizer Bar 19"	1	\$82.67
1i	000191	Adjustable Height Rigidizer Bar 20"	1	\$82.67
2	000177	Rigid Backrest Tube, 9.25"	2	\$23.67
3	000034	Sleeve 3/4" x 1 1/2"	2	\$3.66
4	000073	M5 x 24 Threaded Barrel	4	\$3.78
5	100669	M5 x 10 BTN SHCS BLKZC w/Patch	4	\$1.58
6	103524	M5 x 10 BTN SHCS BLK ZN	4	\$0.53